

## Medcare Lift, Stand & Car Extractor Cleaning Instructions

Wipe all metal, plastic and padding down with a damp cloth using warm water and mild detergent. When higher levels of disinfection are required; use Accelerated Hydrogen Peroxide (AHP) with 0.5% hydrogen peroxide which is found in Accel TB or Oxivir TB or 70% Isopropyl Alcohol. These have been approved and tested on Medcare floor machines. Other disinfectants with equivalent chemical makeup can be used.

NOTE: Do not use petroleum-based solvents, trichloroethylene or similar products in cleaning the surface Medcare's floor machines, as they may damage the plastic. If a hot air dryer is used, the temperature must not exceed 175° F.