


MEDCARE[®]

PRODUCTS

Repositioning Sling

Hook Up Instructions


Note: The color of loops used can vary from person to person depending on weight distribution. Adjust the color of the loops according to the length that is needed to ensure that the patient remains level during the transfer.

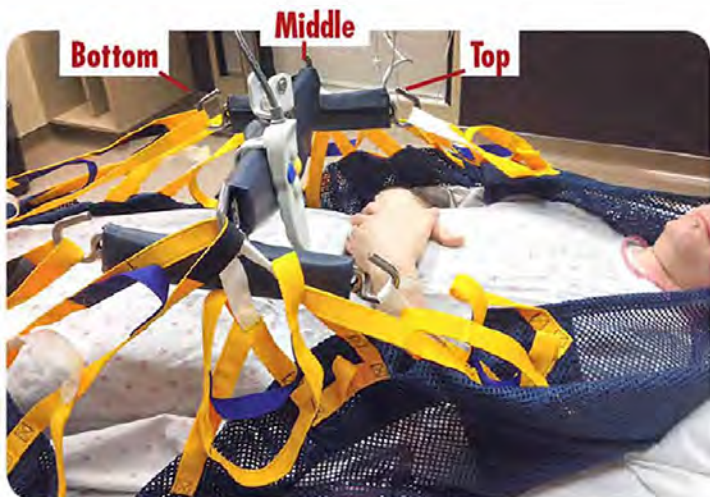

Yellow
Black
Blue
White



Black
Blue
White

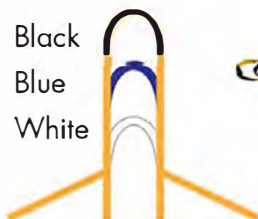

Standard loop colors to keep patient level with 4-point (Medcare or EZ Way) spreader bar. See hook-up options below per each hanger bar:

Option 1 (Standard hook-up):

1. **White** loop on head (1st) strap to top hook
2. **White** loop on shoulder (2nd) strap to middle hook
3. **Black** loop on leg (3rd) strap to middle hook
4. **Yellow** loop on foot (4th) strap to bottom hook

Option 2:

1. **Blue** loop on head (1st) strap to top hook
2. **Blue** loop on shoulder (2nd) strap to middle hook
3. **Black** loop on leg (3rd) strap to middle hook
4. **Yellow** loop on foot (4th) strap to bottom hook

Option 3:

1. **Black** loop on head (1st) strap to top hook
2. **Black** loop on shoulder (2nd) strap to middle hook
3. **Black** loop on leg (3rd) strap to middle hook
4. **Yellow** loop on foot (4th) strap to bottom hook



MEDCARE[®]

PRODUCTS

Repositioning Sling


Hook Up Instructions

Note: The color of loops used can vary from person to person depending on weight distribution. Adjust the color of the loops according to the length that is needed to ensure that the patient remains level during the transfer.

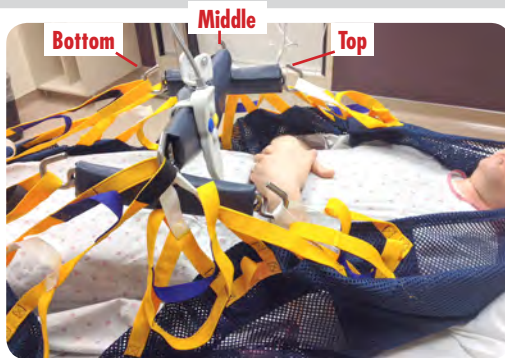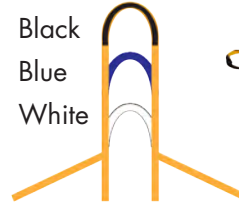

Standard loop colors to keep patient level with 4 point Medicare spreader bar


- White loop on head (1st) strap to top hook
- White loop on shoulder (2nd) strap to middle hook
- Black loop on leg (3rd) strap to middle hook
- Yellow loop on foot (4th) strap to bottom hook

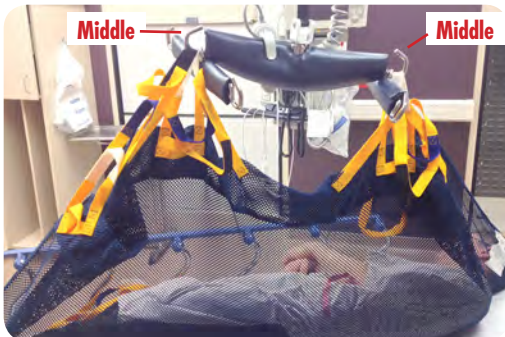

Turning hook up for 4 point spreader bar

Used for Turning or Rolling a patient. Position the hanger bar so that the main (large) carry bar is parallel with the patient's body. Attach shortest loops possible when turning.

- Hook head & shoulder (1st & 2nd) loops to middle hook
- Hook leg & foot (3rd & 4th) to opposite middle hook

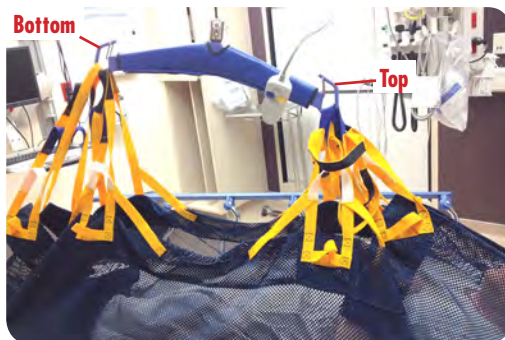

Standard loop colors to keep patient level with 2 point spreader bar

- Blue loop on head (1st) strap to top hook
- Blue loop on shoulder (2nd) strap to top hook
- Black loop on leg (3rd) strap to bottom hook
- Yellow loop on foot (4th) strap to bottom hook

Turning hook up for 2 point spreader bar (Not Pictured)

Used for Turning or Rolling a patient. Position the hanger bar so that the bar is parallel with the patient's body. Attach shortest loops possible when turning.

- Hook head & shoulder (1st & 2nd) loops to top hook
- Hook leg & foot (3rd & 4th) to opposite bottom hook