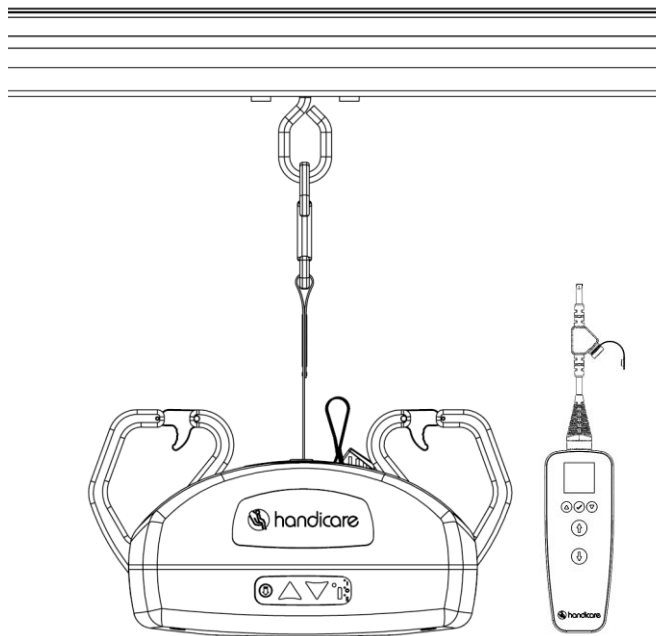
AP300/AP450
Handicare

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Manual: 753160 Rev H

Contents

Cautions	3
Use and Care Introduction	4
Overview of AP300/450 Lift System	4
Turn On the Lift	7
Turn Off the Lift	8
Hand Control Operation	8
Lift Features	8
Raise or Lower the Lift	9
Mounting the Lift to the Track	10
Move the Lift on the Track	11
Transfer an Individual	11
Hand Control Information and Diagnostics	14
Emergency Stop	14
Emergency Lowering	14
Clean and Disinfect the Lift System	15
Contraindications	15
Troubleshooting	16
General Inspection and Preventive Maintenance	17
Recurrent Performance and Safety Testing	19
Reference	20
AP300/450 Physical Overview	20
AP300 Lift Specifications	21
AP450 Lift Specifications	22
Hand Control Icons	23
Français	29

Cautions

- The AP300/450 **must be installed** only by persons authorized by Handicare. The installation guide is available to and for use by authorized dealers only.
- Under no circumstance should the AP300/450 track, lift, sling, or the entire system be put in control of a person who has not been properly trained in the use and care of this equipment. Failure to adhere to this warning may result in serious injury to the operator, and/or the individual being lifted/transferred.
- The AP300/450 lift, and associated track and slings are not toys. Do not use it for unsafe practices. Do not allow children to play with the lift or any of its parts.
- The manufacturer's **warranty is void** if persons unauthorized by Handicare perform work on the AP300/450 lift system.
- There are no user serviceable parts inside the cover. Do not remove cover screws, or open the lift unit, as this may result in serious injury and will **VOID THE WARRANTY**. Contact your local Handicare dealer or your local Handicare business development manager for service.
- In facilities where more than one operator will be responsible for using the AP300/450 lift and associated track and slings it is imperative that all such individuals be trained in its proper use. A training program should be established by the facility to acquaint new operators with this equipment. Contact your local Handicare dealer or your local Handicare business development manager for training assistance.
- Never expose the AP300/450 lift or Hand Control directly into water. Warranty does not cover any misuse or abuse of the lift system.
- To maintain optimum function, the AP300/450 should be inspected and maintained on a regular basis. See the section titled "General Inspection and Maintenance" See page 17.
- Any accessories used with the AP300/450 including but not limited to track and slings, should be checked to ensure that they are in good working order. Check for signs of wear or fraying prior to each use. Report any unusual wear, or damage immediately to your local Handicare dealer or your local Handicare business development manager.
- The AP300/450 lift and associated accessories, track, and slings are intended only for lifting, repositioning and transferring of a person. Handicare will not be responsible for any damage caused by the misuse, neglect or purposeful destruction of the lift, or its associated parts. Do not attempt to modify/alter the AP300/450 lifts.
- Do not in any circumstance exceed the maximum allowable load of this lift. Refer to the "Specifications" section of this manual, or the labels on the lift.
- The installation of the lift, track, accessories, and sling are certified to a maximum load. Do not exceed the maximum rated load of any of the parts.
- There is a risk of explosion if the lift is used in the presence of flammable anesthetics.
- Ensure that a clear space is maintained around the lift and track. Move all curtain material and other obstacles out of the way before performing a transfer.
- The charger must be located outside the patient vicinity at all times. The patient vicinity is the space with surfaces likely contacted by the patient or an attendant who can touch the patient.
- AP300/450 lifts should be decommissioned/disposed of after recommended 10 year service life (or 11,000 cycles, whichever comes earlier) in accordance with regional component specific disposal recommendations.

Use and Care Introduction

To ensure safe operation of your AP300/450 lift, before using this equipment, carefully read this entire manual, especially the section on "Cautions". The AP300/450 is designed to be used in conjunction with Handicare/Prism Medical lift track, compatible competitor tracks, accessories, and slings; Refer to the user guides supplied with these parts before using them and refer to them while reviewing this manual.

Be sure you understand the contents of this manual completely before using this equipment.

Failure to comply with warnings in this manual or proper lifting instructions may result in injury to the operator or the individual being lifted or transferred, as well as damage to the lift and related parts.

If you have any questions after reviewing this manual, Contact your local Handicare dealer or your local Handicare business development manager.

Store this manual with the documents included with the lift system, sling, and all related parts. Contents of this manual are subject to change without prior written notice.

Overview of AP300/450 Lift System

The AP300/450 is a lifting aid used by health care professionals and caregivers in homes, institutions, and long-term care facilities to lift, position and transfer clients or a disabled family member.

The AP300/450 is one part of a ceiling lift system that lifts an individual directly from above, making it possible to transfer and reposition mobility impaired individuals safely, with dignity, and in comfort while minimizing both strain and risk to the caregiver. The ceiling lift also does not take up valuable floor space as most traditional methods do.

The AP300/450 lift is one of the four major parts that make up a ceiling lift system:

- **Lift** - Motorized unit that raises and lowers an individual directly from above
- **Track** - Custom tailored and securely mounted to a ceiling structure, wall or floor provides the path for the lift to traverse
- **Trolley** – Mounted inside the track, the trolley has wheels that allow the lift to move along the track. The lift attaches to the trolley via the lift's carabiner and the eyelet of the trolley.
- **Sling** - Specially designed fabric accessory that attaches to the lift's carry bar to hold and support an individual for positioning or transfer

Refer to any user guides supplied with the accessories and reference them while reviewing this lift guide.

AP300/450 Ceiling Lifts

The AP300/450 is designed to raise and transfer an individual along a track from one surface to another surface. The lift remains on the lift track when in use.

The AP300/450 is a portable ceiling lift. It is designed to be easily moved from one track location to another. It is moved along the track manually with the aid of a caregiver.

Functions such as lifting up or down are controlled by pressing buttons on a digital Hand Control attached to the lift.

Lift System

The following is included with your new AP300/450 lift system:

- AP300/450 lift
- Digital Hand control
- Lift Charger
- Owner's Manual

Additional parts may include:

- **Trolley** - If a trolley has been supplied with the lift, contact your local authorized dealer to ensure proper installation
- **Slings** - If a sling has been supplied with the lift, refer to the instructions included with the sling.
- **Reacher Arm** - If a reacher arm has been supplied with the lift, refer to the instructions included with the reacher arm.
- **Accessories** - If additional accessories such as a turntable or transition gate system have been supplied with the lift, refer to the instructions included with those items.

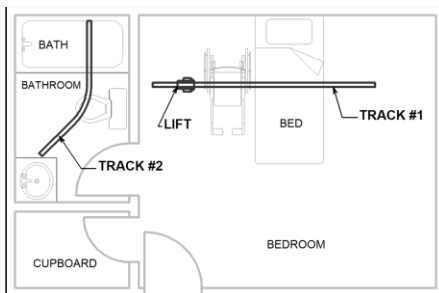
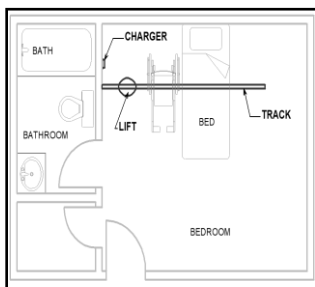
Figure 1: Installed Lift System Sample Plans

Charge the Lift

-  Before initial use, the lift unit must be charged for 5 hours.
-  Charge the batteries regularly. Handicare recommends the lift be left on charge when not in operation, and at the end of each day, which will maximize the life cycle of the batteries.
-  The lift may remain connected to the charger indefinitely since the charger has a built-in regulator to eliminate the danger of overcharging.

The ceiling lift charges through the barrel connection on the hand control cable. While charging, the lift indicator light located on the bottom cover membrane is orange; it turns green once the lift is completely charged. **The lift can only be charged while the emergency stop switch is in the ON position.** See section "Charge through the Hand Control" for further information.

Figure 2: Charger with Barrel Connector

To charge AP300/450 lifts through the hand control:

1. Ensure the hand control is plugged into the AP300/450 lift. See section "Connect the Hand Control"
2. Pull out the protective cap from the Hand Control connector.
3. Align the charger's barrel connector with the opening on the hand control's charger connection point.
4. Insert the charger's barrel connector into the Hand Control.
5. Once the charger is connected properly, the lift indicator located on the bottom cover membrane lights orange and turns green when the lift is completely charged. **The lift can only be charged while the emergency stop switch is in the ON position.**

The lift will not raise or lower when the Hand Control charger is connected; Emergency lowering still functions.

Do not position the lift where it is difficult to disconnect the charger in an event of an emergency. The unit can be isolated electrically from "Mains" by unplugging the charger.

Low Battery

If the batteries of the lift are low (25%) and require charging, the indicator located on the bottom cover membrane of the lift lights orange, a beep audible alarm sounds when button is pressed, and the Hand Control displays the message **Low Battery**. If a transfer is in progress, complete the transfer (the Emergency Down also continues to operate) and move the lift to a location where it can be charged.

- The lift may be used after one hour of charging from low battery; however, Handicare recommends overnight charging or minimum of 5 hours.

If the batteries are fully discharged and require charging, the indicator on the lift control panel lights red a beep audible alarm sounds when button is pressed, and the Hand Control displays the message **Very low battery**.

When the battery is discharged the UP function is disabled. The Down, and Emergency Down continue to operate.

Power supply is considered a part of the AP300/450 system. Use only the charger that was supplied with the lift. Use of any other charger will void all warranties and may cause damage to the lift.

Do not position the lift where it is difficult to disconnect the charger in an emergency.

The unit can be isolated electrically from “Mains” by unplugging the charger.

Plug Types

Plug types vary by geographical location. Refer to local electrical guidelines on the type of plug or voltage which can be used (connection to the charger is an IEC 60320-C7 plug).

Attach the Hand Control to the Lift

A sturdy ladder may be required to access the hand control connection to re-attach the Hand Control to the lift. Use caution whenever using a ladder.

- Should you have any concerns or questions contact your local Handicare dealer or your local Handicare business development manager.
- Should the Hand Control disconnect from the side of the lift it must be re-connected in order for the hand control to work. The lift can still be lifted and lowered using the membrane located on the top cover while the hand control is disconnected.

The Hand Control of the AP300/450 is designed with a locking connector to withstand normal day-to-day usage. However, the connector can become disconnected if:

- The lift is pulled along the track by the Hand Control cord
- The Hand Control cord gets wrapped around an object while a lift or transfer is being performed
- Hand Control cord is accidentally pulled down by the caregiver or the individual being lifted

The Hand Control is connected to a lift connector located on the side of the lift.

The small latch arm on the Hand Control connector holds the Hand Control cord to the lift connector in a specific orientation.

Connect the Hand Control

Both the male connector on the cord and the female connector have a latch. Make sure that the male connector is inserted into the lift connector in the proper orientation.

1. Line up the latch and latch opening as shown in Figure 3.
2. The latch has to be aligned with the cavity of the connector.
3. Insert the cord connector completely into the connector on underside of the lift until you hear a click.
4. Pull lightly on the connector to confirm proper connection.

Figure 3: Hand Control connector with latch being inserted into lift connector

Test the Hand Control

Perform a brief test to ensure proper connectivity:

1. Turn on the lift as shown on page .
2. Press any button on the Hand Control. The Hand Control screen lights up.
3. Raise and lower the lift using the Up button  and Down button .
4. *If the lift moves up and down, the Hand Control is properly connected.*
5. *If the lift does not move correctly, double-check:*
 - That the lift is on
 - That the Hand Control connector is properly connected
 - That the Hand Control does not display **Slack Detected**

If there are still problems with the lift, contact your local Handicare authorized dealer for service.

Mount the Hand Control

While the lift is not in-use, the hand control can be mounted on to the top of the lift using the angled slider. Align the hand control with the slider on the cover, and slide the hand control down. To remove, pull the hand control up and over until it is disengaged from the slider.

Turn On the Lift

Before using the AP300/450 lift system, check the lift, track, and sling for any unusual wear or damage. Refer to the owner's manual for each piece of additional equipment to determine what should be checked. If anything looks unusual, contact your local Handicare dealer or your local Handicare business development manager before use. Failure to check the lift could result in damage to the lift or serious injury to the operator or the individual being lifted.

1. Press any button on the Hand Control to turn the lift on. The indicator light on lift membrane lights green and the display screen on Hand Control displays **[Ready]**.
2. *If the lift batteries require charging, the indicator on the membrane lights orange, a beep audible alarm sounds when the button is pressed, and the Hand Control displays the message **[Low Battery]**.*
3. *If the lift batteries are discharged to 25% of capacity, the Hand Control displays the message **[Low Battery]** and the lift continues to function until the battery status reaches 10-15%. The Battery indicator LED lights **blinking green**.*
4. *If the lift batteries are discharged to 10-15% of capacity, the Hand Control displays the message **[Very low battery]**, the indicator on the lift membrane lights red, and a beep audible alarm sounds when the button is pressed. The lift will not raise, but will lower and the emergency lowering will still function.*

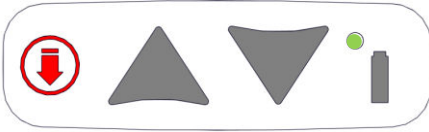
Figure 4: Battery Indicator LED on lift membrane

Turn Off the Lift

The lift automatically shuts off after approximately 3 minutes of non-use.

In an emergency, the lift can also be shut off by switching the Emergency Stop Switch to “Off.” See “Emergency Stop” (page 14) for details on Emergency Stop and restarting the lift after an Emergency Stop.

Hand Control Operation

The AP300/450 lift hand control:

- Controls lift motion
- Presents active lift status display
- Provides usage data and diagnostics.

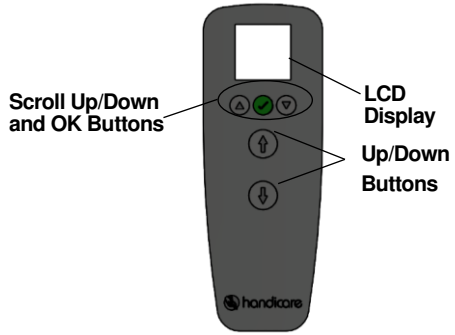
Figure 5: AP300/450 Hand Control Button Layout

Hand Control Display Functionality

The AP300/450 lift Hand Control works in two modes:

1. **Control Mode** to display command icons and active lift status display
2. **Diagnostic/Information Mode** to display lift usage and diagnostic information

See “Hand Control Icons” on page 23 for details on the display information and what it means.

Lift Features

Pivoting Hooks

The AP300/450 Ceiling lift has hooks that can be turned upwards for storage and transport. The hooks must always be in the “DOWN” position when performing a lift. (Figure 5).

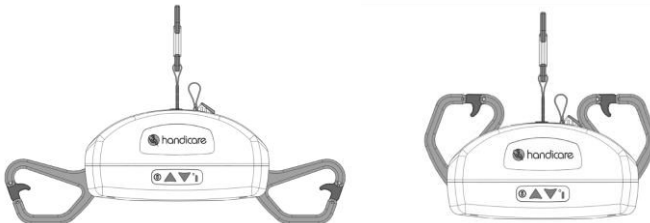
Figure 6: Left – Hooks down for lifting. Right – Hooks up for storage.

Tilt Control

The AP300/450 is equipped with a tilt sensor that activates once the lift tilts past a certain angle, restricting upward movement until the lift is adjusted. *Tilt should be avoided as much as possible during the lifting/transferring process.*

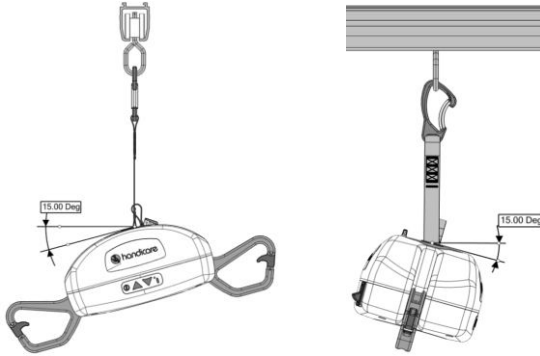
Figure 7: Two Direction Tilt Sensor

Front Cover Membrane

Along with the hand control, the lift can be controlled with the membrane on the front cover of the lift. The membrane includes Up/Down and Emergency Down buttons, as well as the indicator light.

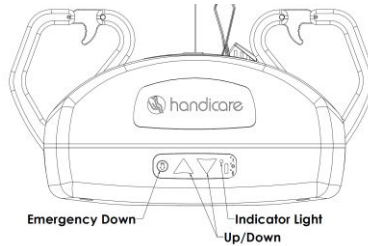
Figure 8: Front Cover Membrane

Raise or Lower the Lift

1. Hold the Hand Control in one hand.
2. If possible, hold the lift in the other hand so that it will not accidentally sway and knock into an individual or nearby object.
3. *To raise the lift*, press the Up button .
4. *To lower the lift*, press the Down button .

Mounting the Lift to the Track

The AP300/450 lift is designed to be easily mounted and removed from the trolley. There are four possible connections between the ceiling lift and trolley.

Figure 9: Left to Right - Standard Connection | Reacher Arm Connection
Lanyard Connection | Reacher arm and Lanyard Connection.

 **ALWAYS ENSURE CARABINER IS PROPERLY POSITIONED BEFORE ATTACHING TO TROLLEY OR REACHER ARM. CARABINER TYPE MAY VARY FOR DIFFERENT MANUFACTURING YEARS.**

Use the following information to properly connect the lift to the track in the four configurations.

1. *Standard Connection* – Open the latch on the lift's carabiner. Insert the carabiner through the eyelet of the trolley. Ensure the carabiner closes fully to lock the lift onto the trolley.
2. *Reacher Arm Connection* – Refer to the Reacher Arm User Guide for proper installation.
3. *Lanyard Connection* - Open the latch on the Lanyard's carabiner. Insert the carabiner through the eyelet of the trolley. Ensure the carabiner closes fully to lock the Lanyard onto the trolley. Open the latch on the Lift's carabiner. Insert the Lift's carabiner through the Lanyard's carabiner. Ensure the carabiner closes fully to lock the lift onto the trolley.
4. *Lanyard and Reacher Arm Connection* – Open the latch on the Lanyard's carabiner. Insert the carabiner through the eyelet of the trolley. Ensure the carabiner closes fully to lock the Lanyard onto the trolley. Refer to the Reacher Arm User Guide for proper installation of the Reacher Arm onto the Lanyard.

Move the Lift on the Track

The lift is normally parked at the end of the track when not in use. Since the AP300/450 lift is a manual ceiling lift, you must move the lift along the track to a position directly above the person to be lifted. See Figure 8 for the various connection types.

Manual Traversing Lift

1. Lower the lift to a comfortable height so you can hold it easily.
2. Grab hold of the lift and move it along the track by gently pushing the lift or the individual in the sling.

Never drag the lift along the track by pulling with the Hand Control.

Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.

Transfer an Individual

The following steps are intended to generally illustrate the procedure of using the lift to transfer an individual from one location to another.

Configurations vary by installation. Prior to attempting these steps, review the instructions for the sling and additional accessories you are using with the lift.

Contact your local Handicare dealer or your local Handicare business development manager if you have any questions or concerns.

To transfer an individual using the lift,

1. Complete the “Each Use” inspection on page 17 of the General Inspection and Preventive Maintenance instructions.
2. Move the lift close to the individual to be transferred.
3. Follow the procedures for up and down and moving along the track as described in “Raise or Lower the Lift” on page 9 and “Move the Lift on the Track” on page 11.

Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.

4. Prepare the individual being transferred with the appropriate sling.
5. Refer to the instructions supplied with the sling.

Prior to lifting, make sure that the sling is correctly fitted and adjusted around of the individual for maximum comfort and safety.

6. Position the lift directly over the individual.
7. Lower the Lift.

Always check to ensure that the lift is correctly positioned directly above the person to be lifted. The lift straps may fray over time, if repeatedly positioned improperly.

Ensure that the lift hooks have no cuts, dents, or sharp edges that may come in contact with the straps of the sling and cause damage to them. Report any concerns to your Handicare dealer or your local Handicare business development manager.

8. Attach the sling loops to the AP300/450 hooks. Be sure the hooks are in the DOWN position prior to lifting.
9. The straps on each side of the sling are generally attached to the corresponding hooks on the lift.
10. Double-check to ensure:
 - The straps are properly attached to the hooks.
 - The individual being lifted is properly positioned in the sling prior to lifting.

Prior to lifting an individual, ensure the Hand Control cord is free of any catch points.

Prior to lifting an individual, make sure that the straps of the sling are securely placed on the hooks of the lift and that the red latches are fully closed. Ensure the straps will not come off. (Figure 9).

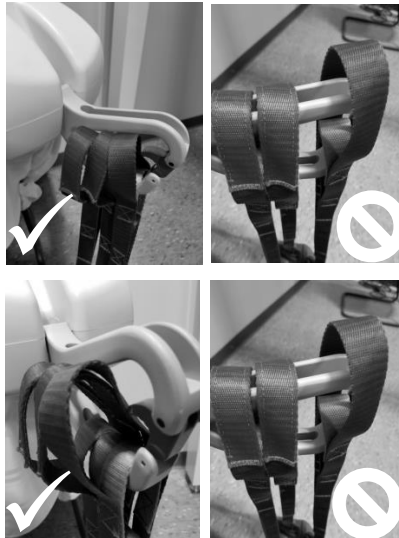
Figure 10: Sling Straps Properly and Improperly Placed

11. On the Hand Control, press the Up button . The lift raises the individual.
 -  **Make sure the individual is lifted to an appropriate height for a safe transfer that avoids any obstructions that may cause injury to the individual in the sling or damage to the lift.**
12. Move the individual along the track to the desired location.
13. Lower the lift to a comfortable height so you can hold it easily.
14. Grab hold of the lift and move it along the track by gently pushing the lift or the individual in the sling.
15. When the individual is at the desired location, press the Down button  to lower the individual to the correct height.
 -  **Before you remove the sling loops from the carry bar, check that the transferred individual is securely supported in the appropriate position.**
16. Press the Down button  to lower the lift so the sling straps can be easily removed from the lift hooks.
 -  **Make sure the lift does not contact the individual in the sling.**
17. Remove the sling straps from the lift's hooks.
18. Press the Up button  to raise the lift to a safe height.
19. Gently push the lift until it is out of the way of the patient.
20. Remove the sling from the individual.
21. Store the sling in a safe place.
22. Press the Up button  to raise the lift to height where it is out of the way while not in use.
23. Charge the lift as described in "Charge the Lift" on page 5.

WARNING: STRANGULATION HAZARD

On completion of Lift Cycle, ensure that the lift, strap, and hand control are raised up away from reach of children, toddlers, or anyone not intended to use the product.

Psychological, mental, and emotional state assessment is key to safe use of a ceiling lift; ensure that use and storage of the ceiling lift and accessories are safe for the patient's mental condition.

While raising a ceiling lift system with or without patient, ensure that the patient, caregiver, other individuals, furniture, and other items are not at risk of becoming entangled with the moving lift strap and hand control cable.

Traversing Beam Transfer

If you are using a Handicare X-Y system with a traversing beam, be sure you understand the traversing beam and how it works in transferring an individual. As shown in **Error! Reference source not found.10**, the X-Y system features two parallel support tracks with one traversing beam mounted across them.

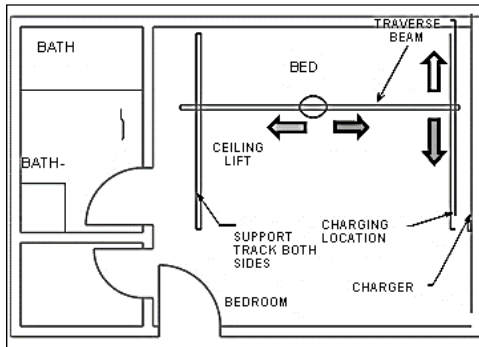
Figure 11: Handicare X-Y System Featuring Traversing Beam

The lift can be moved along the traversing beam, and the traversing beam itself can be moved along the two parallel support tracks. Because of this, an X-Y system provides greater movement and positioning ability for an individual since the coverage area is everywhere between the support tracks as opposed to just the area beneath a section of track, as in a single-track installation.

Since the AP300/450 is a manually traversing lift, the caregiver must move the beam along the support tracks manually to transport the beam, lift, and individual together.

1. Lower the lift to a comfortable height so you can grab it easily.
2. Grab hold of the lift and move it and traversing beam along the track by gently pushing the lift or the individual in the sling.

When moving the traversing beam, avoid any obstructions that may cause injury to the individual in the sling or damage to the lift or track.

Hand Control Information and Diagnostics

In addition to controlling the lift, the AP300/450 Hand Control is an interface that provides usage and diagnostics data from the lift.

To view data and setting through the Hand Control:

1. Press the  key of the Hand Control. The Menu screen displays.

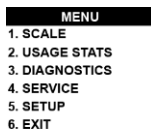
Figure 12: The Hand Control Menu Screen

2. Use the   keys to scroll through the menu options.
3. **The Scale menu** is used on a feature that is not available for AP300/450.
4. **The Usage Stats menu** is for reviewing usage information for the lift.
5. **The Diagnostics menu** is for service and technical support of the lift.
6. On the menu you want to review, press the  key. The selected menu displays.
7. If you select the wrong menu, at the bottom of the display:
 - Select **EXIT/GO BACK** and press the  key to return to the previous screen
or
 - Press the Up  or Down  lift buttons to return to the main screen.

Emergency Stop

The lift unit also has an Emergency Shut-off feature so that in an emergency the operator can completely cut the power to the lift.

To perform an Emergency Stop:

1. Press the Emergency Stop Switch located on the back side of the AP300/450 to OFF.
2. The lift immediately stops, and all power to the lift shuts off
 - *To lower the patient after an emergency stop*, press down on the Emergency Down button located on the lift's membrane.
3. All lift functions are disabled
4. The On indicator light and display turn off

Should the emergency stop function be used in an **emergency**, it must be inspected by an authorized dealer. Once the lift has been inspected and/or repaired, the lift can be turned back on. To turn the lift on after an Emergency Stop:

1. Inspect the lift.
2. Press the Emergency Stop Switch located on the back side of the AP300/450 to ON.
 The Emergency Stop Switch *must* be pushed back into "ON" in order to restore power to the lift.

Emergency Lowering

The Emergency Lowering System can be used to lower the lift should the Down button on the hand control and lift membrane fail. When pressed, an audible alarm will sound until the Emergency Down button is released. When pressed, the Emergency Down Button bypasses all other control circuitry of the lift to immediately lower an individual safely. To perform Emergency Lowering:

1. Locate the Emergency Down button on the lift's membrane (Figure).
2. Press and hold the Emergency Down button until the lift and individual are lowered to a safe location.

Clean and Disinfect the Lift System

Clean the Lift

To clean and disinfect the exterior of the lift:

1. Dampen a cloth with isopropyl alcohol.
2. Wipe down the entire exterior of the lift including the hooks.

Use only isopropyl alcohol to clean and disinfect the exterior of the Lift system. No other chemicals or liquids should be used to clean and disinfect this lift!

Ensure that no liquids get inside the lift unit. This lift unit is not watertight or drip proof; Failure to protect the lift from liquids may result in damage to the lift and result in personal injury.

Clean the Anti-Microbial Strap

The AP300/450 is equipped with a strap featuring specialized antimicrobial coating at the fiber level. The anti-microbial coating provides excellent hygiene and material protection, including:

- Preventing microbial development
- Providing excellent protection against Methicillin-resistant Staphylococcus aureus (MRSA)
- Preventing mold stains and algae infestation
- Providing rot-proof performance

Disinfect the strap using an accelerated hydrogen peroxide disinfecting product.

Clean the Lift Accessories

Follow the cleaning and disinfecting instructions included with the accessory.

Contraindications

There are no known “contraindications” associated with the use of AP300/450 lifts and their accessories, provided they are used according to manufacturer’s recommendations and guidelines.

Troubleshooting

If you encounter problems using the AP300/450, review the Troubleshooting chart.

Use the Fault column to locate the fault you've encountered, then complete the recommended solution in the adjacent column on the row.

If you cannot find the fault or the recommended solution does not correct the problem, immediately contact your local Handicare authorized dealer for service.

Fault	Recommended Solution
The Hand Control of the lift has become disengaged.	Refer to "Attach the Hand Control to the Lift" on page 6. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The Hand Control LCD does not Turn On even after pressing Hand Control buttons.	Ensure that Hand Control is properly connected (see Attach the Hand Control to the Lift" on page 6). Ensure the Emergency Stop Switch is set to ON Ensure the lift is charged; refer "Charge the Lift" on page 5 If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The Hand Control LCD turns On after pressing Hand Control buttons but lift does not move.	Ensure the Hand Control charger is not connected to the lift—the lift will not move when Hand Control charger is connected. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The lift does not operate up or down even when the Hand Control is properly connected and Hand Control LCD is On.	The indicator light on the control panel located on the front side of the lift should be green. Press the Up/Down arrow buttons or any colored button on the Hand Control. This should activate the lift and the indicator light turn green. If the lift still does not function, the batteries may be low and require charging. Refer to "Charge the Lift" on page 5 and charge the lift for at least one hour, then try to raise or lower the carry bar. If this does not correct the problem, contact your local authorized dealer immediately so that the lift can be checked to ensure proper continued operation.
The green light on the front side of the lift is On and Hand Control LCD is On but the lift does not operate in the DOWN direction.	There is a built-in slack tape detector in the lift. This may be sensitive. Apply weight to the lift while pressing the Down button. If this corrects the problem temporarily but not permanently, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The red indicator light on the front side of the lift turns red or a loud alarm sound is heard when an individual is raised.	The batteries are low and require charging. Refer to "Charge the Lift" on page 5. Charge the lift for at least one hour and then try to raise or lower the carry bar. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.

Fault	Recommended Solution
One side of the lift strap is starting to fray after continued use.	Discontinue using immediately and replace strap. Check to be sure that the lift is always directly above the individual being lifted, especially with power traversing lifts. Refer to "Transfer an Individual" on page 11 for correct lift positioning. If fraying still continues, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The lift does not pass through a track component such as a turntable or transition gate.	Refer to the Owner's Manual for the specific piece of equipment in question. If the recommended solution does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
No Power	Ensure the Emergency Stop Switch is set to ON.
Lift starts beeping while pushing Emergency Down button with lift not coming down	The AP300/450 lift is equipped with dual sensors for strap tension and position, the Emergency Down Button/Switch system only works with load on the strap. Emergency Down Button/Switch motion stops immediately after the lift strap is loose.
The lift is recommending preventive maintenance.	Complete the "Semi-Annually" maintenance procedure on page 18 prior to resetting the lift counter. Instructions to reset the lift counter are detailed in the AP300/450 Service/Spare Parts Manual.

General Inspection and Preventive Maintenance

Do not perform any inspection or maintenance while the lift is in use with a patient.

Each Use

(To be completed by User)

Prior to each use the AP300/450 lift and associated track, accessories, and slings must be visually inspected. Refer to the accessory and sling user guides for specific details regarding their inspection.

Should any of these items fail the inspection do not use the lift. Contact your local authorized dealer for service.

Visually check for the following:

- ✓ The lift strap shows NO signs of fraying or breaking along its entire length.
- ✓ The stitching on the lift strap where it connects to the carabiner shows no signs of fraying or breaking.
- ✓ If the lift strap wear indicators are grey, replace the lift strap regardless of number of cycles.
- ✓ The sling that will be used show no signs of unusual wear and tear. The straps of the sling that connect to the carry bar of the lift show no signs of fraying or breaking. Refer to specific sling instructions.
- ✓ The Hand Control cable is not kinked, twisted, knotted, disconnected, or damaged.
- ✓ All the functions on the Hand Control work correctly (e.g. Up/Down, etc.).
- ✓ There are no cuts, dents or sharp edges on the hooks that may damage the sling straps.
- ✓ The lift has no unusual sounds when it is moved up, down, left or right.
- ✓ Ensure that end stops are installed at each end of the track.

Monthly

(To be completed by User)

Monthly, the AP300/450 lift and associated track, accessories, and slings, must be inspected.

Should any of these items fail the inspection *do not use the lift*. Contact your local authorized dealer for service.

- ✓ Complete the visual inspection as noted in the "Each Use" section above.
- ✓ *With no one in the sling nor attached to the lift* check that the lift moves freely along the entire length of the track.

Semi-Annually

(To be completed by User)

In high frequency transfer areas (more than 1500 lifts per year) or situations where heavier than normal clients are regularly lifted, maintenance should also be completed every 6 months. It is recommended that this service history be documented.

Should any of these items fail the inspection, do not use the lift. Contact your local authorized dealer for service.

Check for the following:

- ✓ Check emergency stopping and emergency lowering functions using the red Emergency Stop Switch and Emergency Down button.
- ✓ Inspect lift hooks and exterior for damage. Ensure there are NO cuts, dents, or sharp edges that may damage the strap of the unit or sling.
- ✓ Lower the lift exposing the entire length of strap; verify there are no signs of fraying or breaking along its entire length.
- ✓ The stitching on the lift strap where it connects to the carabiner shows NO signs of fraying or breaking.
- ✓ Verify all functions on the Hand Control work correctly (Up, Down, etc.).
- ✓ Verify all functions on Control Membrane (Up, Down, Emergency Down)
- ✓ The lift makes no unusual sounds when moving up, down, left or right
- ✓ Press and hold the Up button until the lift is at the top. The motor should stop automatically.
- ✓ Using the Hand Control, let the strap out all the way to the ground. The motor should stop before the strap begins to wind backwards.
- ✓ While using either the up or down buttons, angle the strap more than 20 degrees. The motor should stop automatically.
- ✓ Engage the unit with charger and check that the unit is charging properly. The indicator on the lift should light orange.
- ✓ Check that the lift's LED light is green when on, dark when off, and orange when charging
- ✓ Check for cracks or wear in the case.

If the audible/visual preventive maintenance alert sounds, complete the Preventive Maintenance Form before resetting the lift counter. Instructions to reset the lift counter are detailed in the AP300/450 Service/Spare Parts Manual.

Annually

(To be completed by a lift technician)

This section to be only completed by a qualified service technician as authorized by Handicare.

Annually, the AP300/450 system must be inspected. Be sure to document this service history.

- ✓ Complete the inspection as noted in the "Monthly" section above.
- ✓ Complete the preventive maintenance procedure as outlined in the AP300/450 Technical Manual.




Recurrent Performance and Safety Testing

The following table highlights AP300/450's Essential Performance Parameters, with their subsequent testing method and frequency. If your AP300/450 does not meet the following criteria, contact your local Handicare authorized dealer for service.

Performance Criteria	Recommended Check	Frequency
The AP-300 / AP-450 should be capable of lifting 300lb (136kg) / 450 lb (204 kg).	Load test at full load AP-300 (300lb/136kg) AP-450 (450lb/204kg)	Annual
The AP300/450 will operate as intended throughout its service life, provided all recommended maintenance and usage recommendations are followed.	1. Load test at full load AP-300 (300lb/136kg) AP-450 (450lb/204kg) 2. Pre-lifting inspection by user care giver. 3. Preventive maintenance by trained tech.	1. Annual. 2. Every use 3. Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 should be capable of working in conjunction and compatibility with Handicare, Arjo, Guldmann, Milkyway Track, trollies and accessories (Carry bars and Slings).	1. Load test at full load AP-300 (300lb/136kg) AP-450 (450lb/204kg) 2. Pre-lifting inspection by user care giver. 3. Preventive maintenance by trained tech.	1. Annual. 2. Every use 3. Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 should have "Free Fall" protection system.	Preventive maintenance by trained tech.	Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 should have upper and lower limits.	Preventive maintenance by trained tech.	Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 strap should have a minimum safety factor of 6 in terms of break load.	1. Load test at full load AP-300 (300lb/136kg) AP-450 (450lb/204kg) 2. Pre-lifting inspection by user care giver. 3. Preventive maintenance by trained tech.	Annual Every use Semi-annually or annually, per usage frequency See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 should have visual "Error" status display.	Preventive maintenance by trained tech.	Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.
The AP300/450 hand control locking mechanism used as connections should be "Self-Locking".	Preventive maintenance by trained tech.	Semi-annually or annually, per usage frequency. See "General Inspection and Preventive Maintenance" on page 17.

Reference

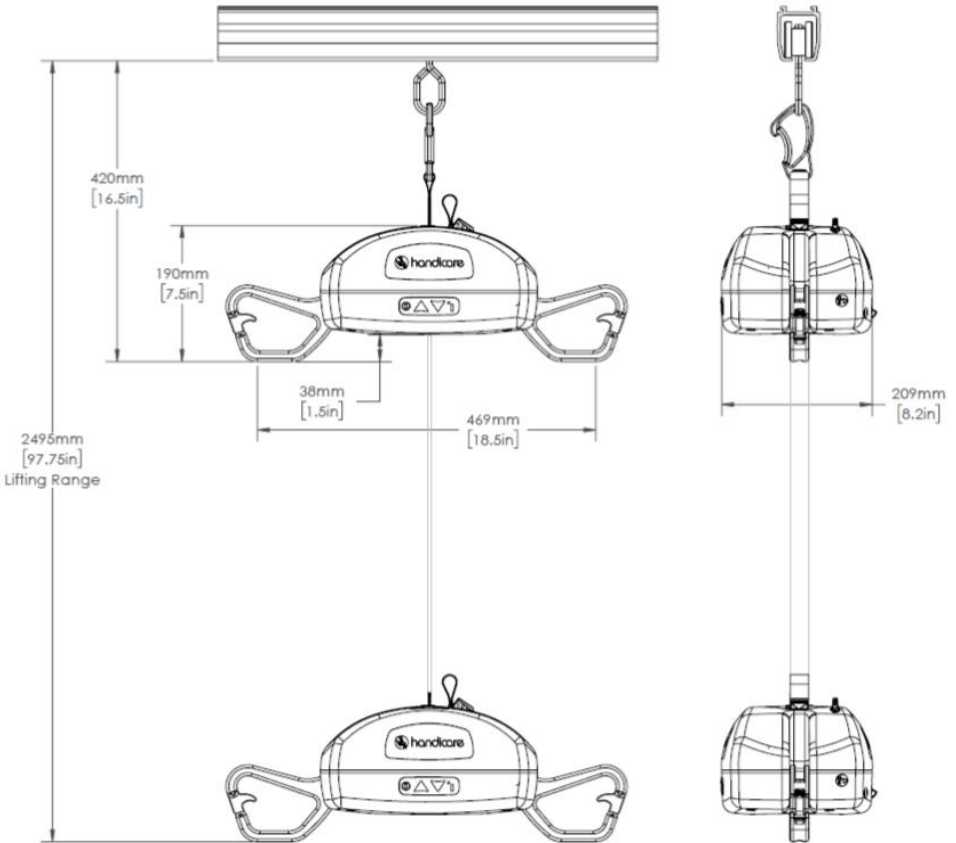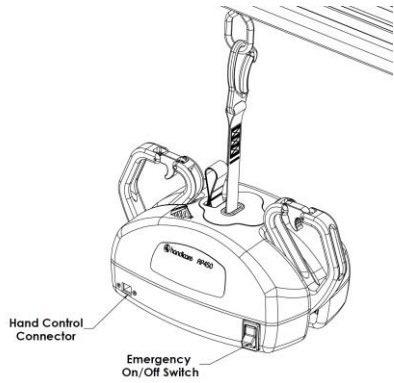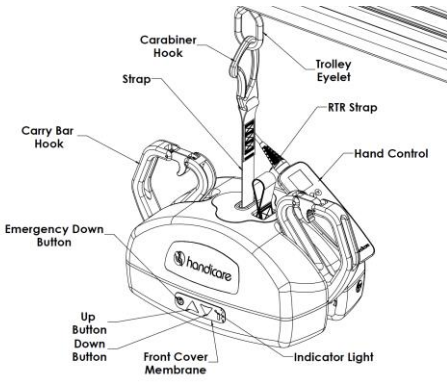AP300/450 Physical Overview

AP300 Lift Specifications

Lift Motor:	24 VDC
Charger:	Model: Soneil, PS2403SRM26
	Input: 100-240 VAC 0.35 Amps, 50-60 Hz
	Output: 24 VDC, 1.33 Amps
	Note: Only use the Soneil, PS2403SRM26 charger with the AP300 Lift. Other chargers, including the P-440 charger, are not compatible to the AP300.
Batteries:	24 VDC (20 x 1.2 VDC) 1.2 AH, Nickel Metal Hydride
Lift Case:	Flame Retardant ABS
Hand control:	Digital with IPX4
Main Unit IP Rating	IP23 = Protected against access to hazards parts by finger, protected against vertically falling as a spray up to 15 degrees from vertical
Lifting Range:	Up to 77" (1960mm)
Lift Weight:	12.2 lb (5.5 kg)
Maximum Safe Working Load:	300 lb / 136 kg Note: The maximum safe working load (SWL) of the installed lift is determined by referring to the product label located on the bottom of the lift.
Duty Cycle:	1 Min On—9 Mins Off
Max Sound Level:	Raising Max load 50 dB, Lowering Max load 50 dB on a lift in good working order.
Service Life:	11,000 cycles or 10 years, whichever comes earlier. As a precautionary measure, the lifting strap should be replaced every 5000 cycles, 3 years, or when the wear indicators of the lift strap are grey, whichever comes earlier. Higher usage lifts may necessitate more frequent replacement of the lifting strap. See "General Inspection and Preventive Maintenance" on page 16.
Usage Environmental Conditions:	Temperature: +10° to +35° C (50 to 95 °F)
	Relative Humidity: 10 to 85% RH
	Atmospheric Pressure: 700 to 1010 hPa
Storage Environmental Conditions	Temperature: -20° to +40° C (5 to 104 °F)
	Relative Humidity: 10 to 90% RH
	Atmospheric Pressure: 250 to 1010 hPa
Rated Performance:	48 lifts at 180 lb (82 kg), middle of lifting range (from 54" to 30" visible strap), 1 min ON - 9 min OFF duty cycle per charge
	22 lifts at 300 lb (136 kg), middle of lifting range (from 54" to 30" visible strap), 1 min ON - 9 min OFF duty cycle per charge

AP450 Lift Specifications

Lift Motor:	24 VDC
Charger:	Model: Soneil, PS2403SRM26
	Input: 100-240 VAC 0.35 Amps, 50-60 Hz
	Output: 24 VDC, 1.33 Amps
	Note: Only use the Soneil, PS2403SRM26 charger with the AP300/450 Lift. Other chargers, including the P-440 charger, are not compatible to the AP300/450.
Batteries:	24 VDC (20 x 1.2 VDC) 4.0 AH, Nickel Metal Hydride
Lift Case:	Flame Retardant ABS
Hand control:	Digital with IPX4
Main Unit IP Rating	IP23 = Protected against access to hazards parts by finger, protected against vertically falling as a spray up to 15 degrees from vertical
Lifting Range:	Up to 77" (1960mm)
Lift Weight:	12.2 lb (5.5 kg)
Maximum Safe Working Load:	450lb / 204.5kg Note: The maximum safe working load (SWL) of the installed lift is determined by referring to the product label located on the bottom of the lift.
Duty Cycle:	1 Min On—9 Mins Off
Max Sound Level:	Raising Max load 50 dB, Lowering Max load 50 dB on a lift in good working order.
Service Life:	11,000 cycles or 10 years, whichever comes earlier. As a precautionary measure, the lifting strap should be replaced every 5000 cycles, 3 years, or when the wear indicators of the lift strap are grey, whichever comes earlier. Higher usage lifts may necessitate more frequent replacement of the lifting strap. See "General Inspection and Preventive Maintenance" on page 17.
Usage Environmental Conditions:	Temperature: +10° to +35° C (50 to 95 °F)
	Relative Humidity: 10 to 85% RH
	Atmospheric Pressure: 700 to 1010 hPa
Storage Environmental Conditions	Temperature: -20° to +40° C (5 to 104 °F)
	Relative Humidity: 10 to 90% RH
	Atmospheric Pressure: 250 to 1010 hPa
Rated Performance:	110 lifts at 275 lb (125 kg), middle of lifting range (from 54" to 30" visible strap), 1 min ON - 9 min OFF duty cycle per charge
	55 lifts at 450 lb (204 kg), middle of lifting range (from 54" to 30" visible strap), 1 min ON - 9 min OFF duty cycle per charge

Hand Control Icons

Display Image	Details	Comments
	Boot Up Icon	Lift is turning On by pressing any button of Hand Control.
	Up motion Icon	Lift is going up. Real-time battery percentage also shown.
	Down motion Icon	Lift is going down. Real-time battery percentage also shown.
	Slack/Loose strap Icon	Strap is slack/loose. Real-time battery percentage also shown.
	Lower Limit Icon	Lower limit reached. Real-time battery percentage also shown.
	Upper Limit Icon	Upper limit reached. <i>Real-time battery percentage also shown.</i>
	Battery Very Low Icon	Battery is critically low. Unit will lower but not raise. <i>Real-time battery percentage also shown.</i>
	Battery Low Icon	Battery is low. Unit will still function. <i>Real-time battery percentage also shown.</i>
	Overload Icon	Lift is overloaded. <i>Real-time battery percentage also shown.</i>
	Service Due Icon	Lift has accumulated at least 1500 post PM cycles.
	Service Limit Exceeded	Lift has surpassed recommended cycles without servicing

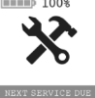
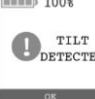
AP300/450 Lift Display Symbols

Symbol	Definition	Symbol	Definition
	Class II equipment		Type B applied part
	Caution risk of danger or Attention, consult accompanying Documents		Refer to instructions manual/booklet
	Certified by TUV		Off Power
	On Power		Keep On all Times
	Press Emergency Only		Emergency Down Button
	Status LED's for battery level and system error indication		Authorized Representative in European Community
	Manufacturer		Body Weight
	Date of Manufacturing		Model/Catalogue Number
	WEEE wheeled bin		Product Serial Number

Symbol	Definition	Symbol	Definition
	General Warning Sign		European Conformity

*CE Mark pending for AP300

LED, Hand Control LCD, and Audio Symbols

Lift Status	Lift Status LED Color	Audio Signals	Hand Control LCD	Available Functionality		
				Up	Down	E-Down
Lift fully charged	Solid Green 	None		None	✓	✓
Lift ready for operation	Solid Green 	None		✓	✓	✓
Charging hand control	Solid Orange 	None		None	✓	✓
Low Battery	Solid Orange 	Beep on button press		✓	✓	✓
Very Low Battery	Solid Red 	Beep on button press		None	✓	✓
Over Load	Blinking Orange 	Beep on button press		None	✓	✓
System Fault	Blinking Orange 	None		None	None	✓
Lift has over 1,000 cycles	Solid Orange 	None		✓	✓	✓
Lift has over 1,500 cycles	Blinking Red 	Beep on button press		✓	✓	✓
Tilt Detected	Solid Green 	Beep on button press		None	✓	✓

IEC 60601-1-2:2014–(Ed 4.0) EMC/EMI Compliance Guidelines

The **equipment** needs special precautions regarding EMC and needs to be installed and put into

service according to the EMC information provided in the accompanying documents.

- The portable and mobile RF equipment can affect the **equipment**.
- Use of accessories and cables other than those specified, with the exception of accessories and cables qualified and sold by the manufacturer of the equipment may result in increased emissions or decreased immunity of the equipment and may cause the system to be non-compliant with the requirements of IEC 60601-1-2:2014.
- The **equipment** should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the equipment should be observed to verify normal operation in the configuration in which it will be used.

The table below elaborates the emission compliance of the equipment's electronics.

Table C.1 – Example Emission Class and Group Compliance

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should		
Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The equipment is suitable for use in all establishments including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Complies	

IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

The table below elaborates the recommended electromagnetic working environment.

Table C.2 – Example Immunity Test Level Compliance

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV Contact ±15 kV Air	±8 kV Contact ±15 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV line to line & ±0.5 kV, ±1 kV, ±2 kV line to ground	±0.5 kV, ±1 kV line to line & ±0.5 kV, ±1 kV, ±2 kV line to ground	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% U_T (100% dip in U_T) for 0,5 cycle 0% U_T (100% dip in U_T) for 1 cycles 70% U_T (30% dip in U_T) for 25/30 cycles 0% U_T (100% dip in U_T) for 5 sec	0% U_T (100% dip in U_T) for 0,5 cycle 0% U_T (100% dip in U_T) for 1 cycles 70% U_T (30% dip in U_T) for 25/30 cycles 0% U_T (100% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation during power mains interruptions, it is recommended that the equipment be powered from an uninterruptible
Power Frequency Magnetic Field (50/60 Hz) IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

The table below elaborates the recommended electromagnetic working environment.

Table C.3 – Example Immunity Test Level Compliance

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the equipment including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2\sqrt{P}$ $d = 1.2\sqrt{P}$ $d = 2.3\sqrt{P}$ 80 MHz to 800 MHz 800 MHz to 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey should be less than the compliance level in each frequency range.
Radiated RF IEC 61000-4-3	6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	
	3 V/m 80 MHz to 2,7 GHz	3 V/m 80 MHz to 2,7 GHz	
	RF communication equipment inside 80 MHz to 6 GHz	RF communication equipment inside 80 MHz to 6 GHz	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re orienting or relocating the equipment.			

IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

The table below elaborates the recommended separation between the equipment from RF communication devices.

Table C.4 – Example Recommended Separation Distances

Recommended separation distances between portable and mobile RF communications equipment and the equipment in question			
The equipment is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the equipment can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the equipment as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

Warranty

This Warranty does not affect or in any way limit your Statutory Rights.

Handicare guarantees all equipment, which includes Ceiling Lifts, Floor Lifts, Slings, Service Parts and Track and accessories, supplied as new, against failure within the period of 1 year from date of installation or 18 months from date of manufacturing, whichever is shorter, by virtue of defects in material or workmanship.

Handicare guarantees all refurbished equipment supplied against failure within a period of three months from date of installation or six months from date of purchase whichever is shorter.

This guarantee does not apply to failure attributable to normal wear and tear, damage by natural forces, user neglect or misuse or to deliberate destruction, or to batteries more than 90 days after original purchase.

This guarantee shall be void if the equipment is not serviced by Handicare or its authorized service agents in accordance with the manufacturer's recommendations or if any unauthorized person carries out works on the equipment.

The liability of Handicare under the terms of this guarantee shall be limited to the replacement of defective parts and in no event shall Handicare incur liability for any consequential or unforeseeable losses.

Always make sure you have the correct version of the manual. The most recent version is available on our website, www.handicare.com.

For questions about the manufacture or operation of this equipment, contact your local Handicare dealer or your local Handicare business development manager.

About Handicare

Handicare offers solutions and support to increase the independence of the physically challenged, mobility impaired or elderly people and to enable them to live an active life—on their terms—and to facilitate for their care providers and family. The offering encompasses a comprehensive range of curved and straight stairlifts, transfer, lifting and repositioning aids, vehicle adaptations and medical equipment. Handicare is a global company with sales in more than 20 countries and is a market leader in this field. The head office is in Stockholm, Sweden and manufacturing is located at six sites distributed across North America, Asia and Europe.

A-Series

The A-Series Advantage

Built on the proven technology of the Handicare P-Series, the new AP300/450 ceiling lifts raise patient handling to the next level. Increase patient protection with antimicrobial materials, simplify patient transfers with its tilting design and easily access real-time data and diagnostics. The A-Series ceiling lifts give you the extra advantage in safe patient handling.

Always make sure you have the correct version of the manual

The most recent version of all manuals are available for downloading from our website, www.handicare.com.

For questions about the product and its use

Contact your local Handicare dealer or your local Handicare business development manager.

Please contact your local dealer or Handicare for any questions regarding electromagnetic compatibility.

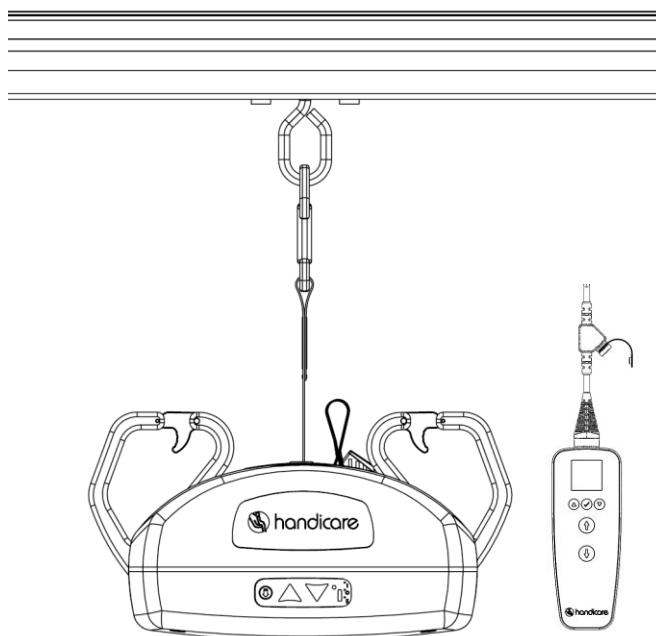
Handicare USA, Inc.
10888 Metro Court
St. Louis, MO 63043 USA
Tel: 314-692-9135
Fax: 314-692-7858
Toll Free: 1-866-276-5438
www.handicare.com

Handicare USA is quality certified in accordance
with ISO 13485

For the latest version of the User Manual please
visit www.handicareusa.com

Handicare Canada
81 Romina Drive
Concord, ON L4K 4Z9
Canada
Tel. : +1 416 260-2145
Fax : +1 416 260-0256
Toll free :
+1 877 304-5438
www.handicare.com

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