



Handicare Slings

Lift and Transfer Slings for Any Situation

Table of contents

3	The Art of Choosing	
4	Materials	
5	Sizing Guide	
6–22	Slings	
	Universal Sling	6–7
	Deluxe Hammock Sling	8–9
	Hammock Sling	10–11
	Comfort Care Sling	12–13
	Hygiene Sling	14
	Dual Access Sling	15
	Band Sling	16
	Positioning Sling	17
	Tri-Turner Sling	18
	Full Standing Support Sling	19
	Rehab Total Support System	20
	Sit-to-Stand Sling	21
	Thorax Sling	22
23	Situations Table	

The Art of Choosing


With over one million slings manufactured worldwide, Handicare offers a comprehensive line of styles, sizes, and materials that can be used with most ceiling or floor lifts. We also offer options for users that require a customized solution.

Several factors go into choosing the right sling, but it all begins with the user, their ability and need for support, and also the type of transfer being performed. After the initial assessment is made, the sling model is selected first, then the size, and the material. Finally the sling bar and any accessories that may be needed for optimal lifting are chosen.


The aim is to maximize safety and comfort for the user by choosing the right sling model for each situation.

Handicare offers solutions for many different transfer situations and for all users. Each of our slings are designed and produced for safety and comfort, both for the user and the caregiver.

Some models and materials work best for seated transfers, for example, between bed and chair, while others are adapted for situations such as lifting from the recumbent position, for hygiene, bathing, or showering.

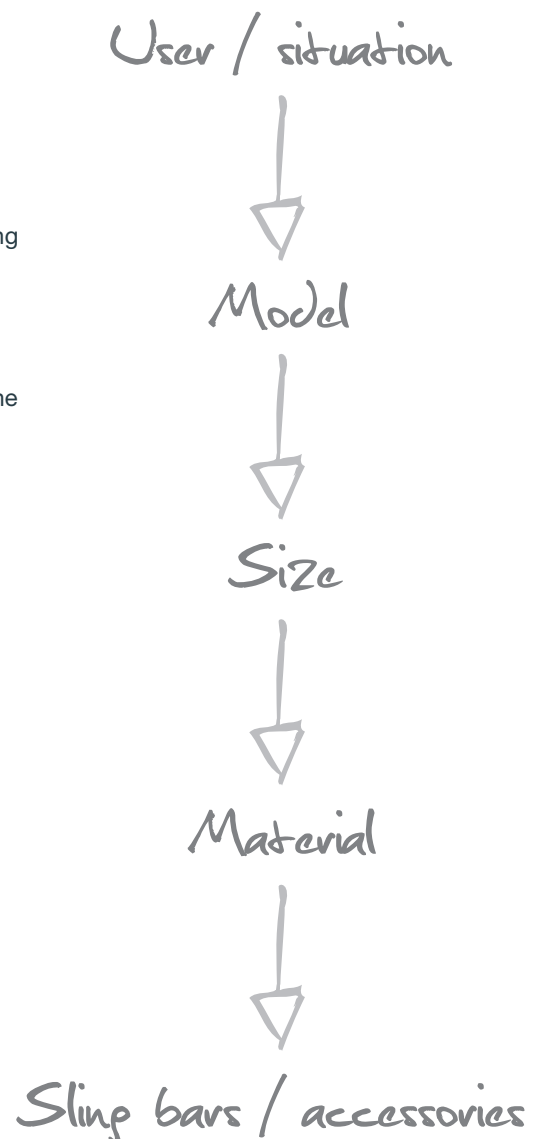

Materials

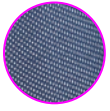

Mesh with Anti-Microbial Coating

- Highly durable, open-weave polyester blend
- Manufactured for strength
- Mesh slings have quick drying properties and are highly breathable
- Slings made of this fabric are washable



Polyester SlipFit (PolySlip)

- Smooth knitted polyester fabric that can stretch providing ease of application and added comfort
- Includes outer leg sections of 'slip fit' nylon material with low friction properties allowing it to be fitted / removed more easily



Polyester Mesh (Mesh / Poly) with Anti-Microbial Coating

- Polyester mesh is combined with smooth polyester layer and non-absorbent padding on inner leg sections
- Ideal for use in swimming pools and for showering while the sling is in position




Disposable

- Designed for single patient use, this non-woven, stitch-bond polyester is strong, highly breathable, and liquid repellent
- Can be spot cleaned of light soiling; disposable fabric slings are designed to be left in place
- Premium version includes padded leg and head support, increased SWL, and ability to position user in sling



Polyester (Poly) with Anti-Microbial Coating

- Highly durable, closed weave fabric manufactured for strength
- Slings made of this fabric are washable and have an anti-microbial coating
- Some of our Polyester slings offer open-cell foam padding in the leg supports and optional head supports.

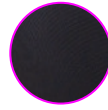

Spacer

- Multi-directional stretch
- Wicks away moisture
- May be suitable for those with skin integrity issues
- May provide a more comfortable fit for those in pain



Quilted Mesh (Quilted)

- Our Quilted Mesh fabric slings have a unique blend of quilted open cell foam and polyester that provides increased comfort for users and reduces bunching under the legs
- Highly durable and constructed to be permeable, allowing for air and water to pass through
- Easily washable when needed
- Seated style transfers slings have the body of Mesh material and the leg supports of Quilted Mesh material



Padded with Anti-Microbial Coating

- Padded polyester leg bands with increased comfort

Sizing Guide

Remember to use your best clinical judgment when choosing a sling size which best fits your patient / user. People come in all different shapes and sizes and two people may weigh the same or be the same height but require different sized slings. If the patient / user weighs over 400 lb (181 kg) a sling consultation is recommended.

All Seated Slings

Child	less than 3 feet less than 91 cm	less than 70 lb less than 32 kg
Junior	3 - 4 feet 91 - 121 cm	70 - 110 lb 32 - 50 kg
S	4 - 5'6 feet 121 - 152 cm	95 - 150 lb 43 - 68 kg
M	5 - 6 feet 152 - 183 cm	125 - 250 lb 57 - 113 kg
L	6 - 7 feet 183 - 213 cm	250 - 400 lb 113 - 181 kg
XL	6 - 7 feet 183 - 213 cm	400 - 480 lb 181 - 218 kg
XXL	more than 6.5 feet more than 198 cm	480 - 1,000 lb 218 - 454 kg

Rehab Total Support System

Junior	25 - 32 in 64 - 81 cm
S	32 - 36 in 81 - 91 cm
M	37 - 52 in 94 - 132 cm
L	53 - 60 in 135 - 152 cm
XL	61 - 68 in 155 - 173 cm
XXL	69 - 76 in 175 - 193 cm

Stand Aid and Full Standing Support Slings

Junior	23 - 28 in 58 - 71 cm
S	29 - 44 in 74 - 112 cm
M	36 - 51 in 91 - 130 cm
L	44 - 59 in 112 - 150 cm
XL	52 - 67 in 132 - 170 cm
XXL	61 - 76 in 155 - 193 cm

Spacer fabric slings sizing is located on the tag.

Thorax Sling

S	37" 94 cm
L	49" 124 cm
XL	57" 146 cm

Tri-Turner Sling

Standard	39 in 99 cm	19 in 48 cm	10 in 25 cm
Large	44 in 111 cm	24 in 60 cm	13 in 33 cm
Bariatric	49 in 124 cm	30 in 76 cm	16 in 40 cm

Band Sling

Standard	28 in 71 cm	9 in 23 cm
Bariatric	34 in 86 cm	9 in 23 cm

Universal Sling






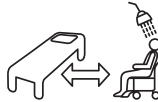



Multi-Purpose Seated Transfer Sling

- Provides support around the hips and sacral area and under the thighs up to the shoulders with optional head support
- Generous lateral trunk support
- Multiple loop and strap options can accommodate various seated and semi-reclined positions
- Divided leg supports can be applied and removed in a seated or supine position
- Leg straps can be applied in various positions for different levels of support
- Built-in caregiver handles on each side add an additional level of control when moving and transferring user
- Medium commode opening

Materials

 Mesh / Poly Opt. Head Support ✓	 Quilted Opt. Head Support ✓	 Mesh W. Head Support ✓	 PolySlip Opt. Head Support ✓
 Spacer Opt. Head Support ✓	 Padded Opt. Head Support ✓	 Disposable Integrated Head Support ✓	 Premium Disposable Integrated Head Support ✓

Applications

 Up from the floor <small>*Requires Head Support</small>	 Higher up in bed <small>*Requires Head Support</small>	 From bed to wheelchair	 From bed to commode	 From wheelchair to toilet
 Transfer to tub	 Positioning in chair			

Universal Sling

Part Numbers

Mesh / Poly

Part No.	Size	SWL
8A2700	Child	450 lb (204 kg)
8A2600	Junior	450 lb (204 kg)
8A2500	S	625 lb (283 kg)
8A2400	M	625 lb (283 kg)
8A2300	L	625 lb (283 kg)
8A2200	XL	1,000 lb (454 kg)

Quilted

Part No.	Size	SWL
517305	Junior	1,000 lb (454 kg)
517310	S	1,000 lb (454 kg)
517320	M	1,000 lb (454 kg)
517330	L	1,000 lb (454 kg)
517340	XL	1,000 lb (454 kg)
517350	XXL	1,000 lb (454 kg)

Mesh with Head Support

Part No.	Size	SWL
517405	Junior	800 lb (363 kg)
517410	S	800 lb (363 kg)
517420	M	800 lb (363 kg)
517430	L	800 lb (363 kg)
517440	XL	800 lb (363 kg)
517450	XXL	800 lb (363 kg)

Mesh / Poly with Head Support

Part No.	Size	SWL
8A2730	Child	450 lb (204 kg)
8A2630	Junior	450 lb (204 kg)
8A2530	S	625 lb (283 kg)
8A2430	M	625 lb (283 kg)
8A2330	L	625 lb (283 kg)
8A2230	XL	1,000 lb (454 kg)

Quilted with Head Support

Part No.	Size	SWL
517505	Junior	1,000 lb (454 kg)
517510	S	1,000 lb (454 kg)
517520	M	1,000 lb (454 kg)
517530	L	1,000 lb (454 kg)
517540	XL	1,000 lb (454 kg)
517550	XXL	1,000 lb (454 kg)

PolySlip

Part No.	Size	SWL
8A1700	Child	450 lb (204 kg)
8A1600	Junior	450 lb (204 kg)
8A1500	S	625 lb (283 kg)
8A1400	M	625 lb (283 kg)
8A1300	L	625 lb (283 kg)
8A1200	XL	1,000 lb (454 kg)

Spacer

Part No.	Size	SWL
8A4700	Child	625 lb (283 kg)
8A4600	Junior	625 lb (283 kg)
8A4500	S	625 lb (283 kg)
8A4400	M	625 lb (283 kg)
8A4300	L	625 lb (283 kg)
8A4200	XL	625 lb (283 kg)

Disposable

Part No.	Size	SWL
547050-10	S	625 lb (283 kg)
547060-10	M	625 lb (283 kg)
547070-10	L	625 lb (283 kg)
547080-10	XL	625 lb (283 kg)

PolySlip with Head Support

Part No.	Size	SWL
8A1720	Child	450 lb (204 kg)
8A1620	Junior	450 lb (204 kg)
8A1520	S	625 lb (283 kg)
8A1420	M	625 lb (283 kg)
8A1320	L	625 lb (283 kg)
8A1220	XL	1,000 lb (454 kg)

Spacer with Head Support

Part No.	Size	SWL
8A4730	Child	625 lb (283 kg)
8A4630	Junior	625 lb (283 kg)
8A4530	S	625 lb (283 kg)
8A4430	M	625 lb (283 kg)
8A4330	L	625 lb (283 kg)
8A4230	XL	625 lb (283 kg)

Premium Disposable

Part No.	Size	SWL
547051-10	S	1,000 lb (454 kg)
547061-10	M	1,000 lb (454 kg)
547071-10	L	1,000 lb (454 kg)
547081-10	XL	1,000 lb (454 kg)

Padded

Part No.	Size	SWL
527350	XXL	1,000 lb (454 kg)

Padded with Head Support

Part No.	Size	SWL
527550	XXL	1,000 lb (454 kg)


Deluxe Hammock Sling



Multi-Purpose Seated Transfer Sling

- Provides support around the hips and sacral area and under the thighs up to the shoulders with optional head support
- Multiple loop and strap options can accommodate various seated and semi-reclined positions
- Divided leg supports that can be applied and removed in a seated or supine position
- Leg straps can be applied in various positions for different levels of support
- Built-in caregiver handles on each side add an additional level of control when moving and transferring user
- Small to medium commode opening

Materials

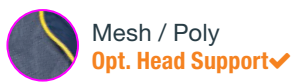

Mesh / Poly
Opt. Head Support ✓



PolySlip
Opt. Head Support ✓



Spacer
Opt. Head Support ✓

Applications




Up from the floor
*Requires Head Support



Higher up in bed
*Requires Head Support



From bed to wheelchair

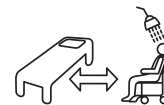

From bed to commode



From wheelchair to toilet



Transfer to tub



Positioning in chair

Deluxe Hammock Sling

Part Numbers

Mesh / Poly

Part No.	Size	SWL
8H2700	Child	450 lb (204 kg)
8H2600	Junior	450 lb (204 kg)
8H2500	S	625 lb (283 kg)
8H2400	M	625 lb (283 kg)
8H2300	L	625 lb (283 kg)
8H2200	XL	1,000 lb (454 kg)

PolySlip

Part No.	Size	SWL
8H1700	Child	450 lb (204 kg)
8H1600	Junior	450 lb (204 kg)
8H1500	S	625 lb (283 kg)
8H1400	M	625 lb (283 kg)
8H1300	L	625 lb (283 kg)
8H1200	XL	1,000 lb (454 kg)

Spacer

Part No.	Size	SWL
8H4700	Child	625 lb (283 kg)
8H4600	Junior	625 lb (283 kg)
8H4500	S	625 lb (283 kg)
8H4400	M	625 lb (283 kg)
8H4300	L	625 lb (283 kg)
8H4200	XL	625 lb (283 kg)

Mesh / Poly with Head Support

Part No.	Size	SWL
8H2730	Child	450 lb (204 kg)
8H2630	Junior	450 lb (204 kg)
8H2530	S	625 lb (283 kg)
8H2430	M	625 lb (283 kg)
8H2330	L	625 lb (283 kg)
8H2230	XL	1,000 lb (454 kg)

PolySlip with Head Support

Part No.	Size	SWL
8H1720	Child	450 lb (204 kg)
8H1620	Junior	450 lb (204 kg)
8H1520	S	625 lb (283 kg)
8H1420	M	625 lb (283 kg)
8H1320	L	625 lb (283 kg)
8H1220	XL	1,000 lb (454 kg)

Spacer with Head Support

Part No.	Size	SWL
8H4730	Child	625 lb (283 kg)
8H4630	Junior	625 lb (283 kg)
8H4530	S	625 lb (283 kg)
8H4430	M	625 lb (283 kg)
8H4330	L	625 lb (283 kg)
8H4230	XL	625 lb (283 kg)

Mesh / Poly with Hip Strap

Part No.	Size	SWL
8H2701	Child	450 lb (204 kg)
8H2601	Junior	450 lb (204 kg)
8H2501	S	625 lb (283 kg)
8H2401	M	625 lb (283 kg)
8H2301	L	625 lb (283 kg)
8H2201	XL	1,000 lb (454 kg)

PolySlip with Hip Strap

Part No.	Size	SWL
8H1701	Child	450 lb (204 kg)
8H1601	Junior	450 lb (204 kg)
8H1501	S	625 lb (283 kg)
8H1401	M	625 lb (283 kg)
8H1301	L	625 lb (283 kg)
8H1201	XL	1,000 lb (454 kg)

Spacer with Hip Strap

Part No.	Size	SWL
8H4701	Child	625 lb (283 kg)
8H4601	Junior	625 lb (283 kg)
8H4501	S	625 lb (283 kg)
8H4401	M	625 lb (283 kg)
8H4301	L	625 lb (283 kg)
8H4201	XL	625 lb (283 kg)

Mesh / Poly with Head Support and Hip Strap

Part No.	Size	SWL
8H2731	Child	450 lb (204 kg)
8H2631	Junior	450 lb (204 kg)
8H2531	S	625 lb (283 kg)
8H2431	M	625 lb (283 kg)
8H2331	L	625 lb (283 kg)
8H2231	XL	1,000 lb (454 kg)

PolySlip with Head Support and Hip Strap

Part No.	Size	SWL
8H1721	Child	450 lb (204 kg)
8H1621	Junior	450 lb (204 kg)
8H1521	S	625 lb (283 kg)
8H1421	M	625 lb (283 kg)
8H1321	L	625 lb (283 kg)
8H1221	XL	1,000 lb (454 kg)

Spacer with Head Support and Hip Strap

Part No.	Size	SWL
8H4731	Child	625 lb (283 kg)
8H4631	Junior	625 lb (283 kg)
8H4531	S	625 lb (283 kg)
8H4431	M	625 lb (283 kg)
8H4331	L	625 lb (283 kg)
8H4231	XL	625 lb (283 kg)

Hammock Sling



Multi-Purpose Seated Transfer Sling

- Provides support around the hips and sacral area and under the thighs up to the shoulders with optional head support
- Multiple loop and strap options can accommodate various seated and semi-reclined positions
- Divided leg supports that can be applied and removed in a seated or supine position
- Leg straps can be applied in various positions for different levels of support
- Built-in caregiver handles on each side add an additional level of control when moving and transferring user
- Small commode opening

Materials



Applications




Up from the floor
**Requires Head Support*



Higher up in bed
**Requires Head Support*



From bed to wheelchair

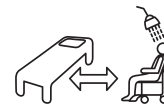

From bed to commode



From wheelchair to toilet



Transfer to tub



Positioning in chair

Hammock Sling

Part Numbers

Mesh / Poly with Head Support

Part No.	Size	SWL
8D2730	Child	450 lb (204 kg)
8D2630	Junior	450 lb (204 kg)
8D2530	S	625 lb (283 kg)
8D2430	M	625 lb (283 kg)
8D2330	L	625 lb (283 kg)
8D2230	XL	1,000 lb (454 kg)

PolySlip with Head Support

Part No.	Size	SWL
8D1720	Child	450 lb (204 kg)
8D1620	Junior	450 lb (204 kg)
8D1520	S	625 lb (283 kg)
8D1420	M	625 lb (283 kg)
8D1320	L	625 lb (283 kg)
8D1220	XL	1,000 lb (454 kg)

Spacer with Head Support

Part No.	Size	SWL
8D4730	Child	625 lb (283 kg)
8D4630	Junior	625 lb (283 kg)
8D4530	S	625 lb (283 kg)
8D4430	M	625 lb (283 kg)
8D4330	L	625 lb (283 kg)
8D4230	XL	625 lb (283 kg)

Quilted with Head Support

Part No.	Size	SWL
537105	Junior	800 lb (363 kg)
537110	S	800 lb (363 kg)
537120	M	800 lb (363 kg)
537130	L	800 lb (363 kg)
537140	XL	800 lb (363 kg)
537150	XXL	800 lb (363 kg)

Comfort Care Sling



Multi-Purpose Seated Transfer Sling

- Provides support from under the thighs up to the shoulders with optional head support
- Small commode opening is optional
- Available with multiple loop and strap options that accommodate various seated and semi-reclined positions

Materials



Applications




Up from the floor
*Requires Head Support



Higher up in bed
*Requires Head Support



From bed to wheelchair

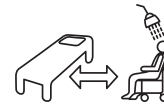

From bed to commode
*Requires Commode Opening



From wheelchair to toilet
*Requires Commode Opening



Transfer to tub



Positioning in chair

Comfort Care Sling

Part Numbers

Spacer

Part No.	Size	SWL
8E4700	Child	625 lb (283 kg)
8E4600	Junior	625 lb (283 kg)
8E4500	S	625 lb (283 kg)
8E4400	M	625 lb (283 kg)
8E4300	L	625 lb (283 kg)
8E4200	XL	625 lb (283 kg)

Spacer with Hip Strap

Part No.	Size	SWL
8E4701	Child	625 lb (283 kg)
8E4601	Junior	625 lb (283 kg)
8E4501	S	625 lb (283 kg)
8E4401	M	625 lb (283 kg)
8E4301	L	625 lb (283 kg)
8E4201	XL	625 lb (283 kg)

Spacer with Commode

Part No.	Size	SWL
8E4702	Child	625 lb (283 kg)
8E4602	Junior	625 lb (283 kg)
8E4502	S	625 lb (283 kg)
8E4402	M	625 lb (283 kg)
8E4302	L	625 lb (283 kg)
8E4202	XL	625 lb (283 kg)

Spacer with Head Support

Part No.	Size	SWL
8E4730	Child	625 lb (283 kg)
8E4630	Junior	625 lb (283 kg)
8E4530	S	625 lb (283 kg)
8E4430	M	625 lb (283 kg)
8E4330	L	625 lb (283 kg)
8E4230	XL	625 lb (283 kg)

Spacer with Head Support and Hip Strap

Part No.	Size	SWL
8E4731	Child	625 lb (283 kg)
8E4631	Junior	625 lb (283 kg)
8E4531	S	625 lb (283 kg)
8E4431	M	625 lb (283 kg)
8E4331	L	625 lb (283 kg)
8E4231	XL	625 lb (283 kg)

Spacer with Head Support and Commode

Part No.	Size	SWL
8E4732	Child	625 lb (283 kg)
8E4632	Junior	625 lb (283 kg)
8E4532	S	625 lb (283 kg)
8E4432	M	625 lb (283 kg)
8E4332	L	625 lb (283 kg)
8E4232	XL	625 lb (283 kg)

Hygiene Sling



Seated Transfer Sling for Toileting


- Provides support from under the thighs, behind the mid to low back, around the torso
- Includes buckle enclosure
- Divided leg supports that can be applied and removed in a seated position
- Large commode opening

Materials



Quilted

Applications

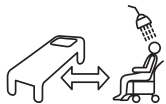

From bed to commode



From wheelchair to toilet



Transfer to tub

Part Numbers

Quilted with Buckle

Part No.	Size	SWL
537605	Junior	1,000 lb (454 kg)
537610	S	1,000 lb (454 kg)
537620	M	1,000 lb (454 kg)
537630	L	1,000 lb (454 kg)
537640	XL	1,000 lb (454 kg)
537650	XXL	1,000 lb (454 kg)


Dual Access Sling



Seated Transfer Sling for Toileting

- Provides support from under the thighs, behind the mid to low back, around the torso with a Velcro and belt enclosure
- Side retention straps are optional
- Divided, padded leg supports can be applied and removed in a seated position
- Large commode opening

Materials

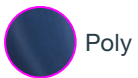

Poly

Applications



From bed to commode



From wheelchair to toilet



Transfer to tub

Part Numbers

Poly

Part No.	Size	SWL
8B0700	Child	450 lb (204 kg)
8B0600	Junior	450 lb (204 kg)
8B0500	S	625 lb (283 kg)
8B0400	M	625 lb (283 kg)
8B0300	L	625 lb (283 kg)
8B0200	XL	1,000 lb (454 kg)

Poly with Side Retention Straps

Part No.	Size	SWL
8C0700	Child	450 lb (204 kg)
8C0600	Junior	450 lb (204 kg)
8C0500	S	625 lb (283 kg)
8C0400	M	625 lb (283 kg)
8C0300	L	625 lb (283 kg)
8C0200	XL	1,000 lb (454 kg)

Band Sling



Limb Support Sling

- Designed to assist caregivers in supporting and positioning limbs
- Situations include wound care, foot care, hygiene care, sling application, bed exercises, and general limb elevation

Materials



Quilted



Disposable

Applications



Support for limbs

Part Numbers

Quilted

Part No.	Size	SWL
507720	Standard	450 lb (204 kg)
507740	Bariatric	450 lb (204 kg)

Disposable

Part No.	Size	SWL
507770-10	One Size	450 lb (204 kg)


Positioning Sling



Positioning Sling for Full Body Support

- Provides full body support for lateral supine transfers, repositioning and turning in bed
- Multiple loop and strap options can accommodate various user repositioning and transfer tasks
- Intended to be left in place (assessment required)

Materials

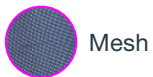

Mesh



Poly




Spacer



Disposable

Applications

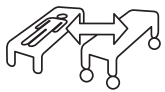

Supine transfer



Turning in bed



Higher up in bed

Part Numbers

Mesh

Part No.	Size	SWL
507810	36" x 70"	800 lb (363 kg)
507811	42" x 78"	800 lb (363 kg)
507814	56" x 78"	1,000 lb (454 kg)

Spacer with Elastic

Part No.	Size	SWL
9Y4901	42" x 78"	1,000 lb (454 kg)

Poly

Part No.	Size	SWL
507802	36" x 51"	800 lb (363 kg)
507807	36" x 70"	800 lb (363 kg)
507816	42" x 78"	800 lb (363 kg)
507818	56" x 78"	1,000 lb (454 kg)

Disposable

Part No.	Size	SWL
547820-10	42" x 66"	1,100 lb (499 kg)
547810S-10	44" x 69"	660 lb (299 kg)
547810-10	52" x 78"	1,000 lb (454 kg)
547810XXL-10	63" x 87"	1,000 lb (454 kg)


Tri-Turner Sling



Turning Sling

- Symmetrically designed to assist with turning in either direction in bed
- Velcro straps securely attach to bedframe to prevent sling migration
- Provides access for hygiene care, wound care, offloading, dressing

Materials

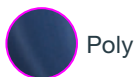

Poly



Spacer



Disposable

Applications



Turning in bed

Part Numbers

Poly

Part No.	Size	SWL
507782	Standard	450 lb (204 kg)
507783	Large	450 lb (204 kg)
507784	Bariatric	450 lb (204 kg)

Spacer

Part No.	Size	SWL
507786	Standard	625 lb (283 kg)
507787	Large	625 lb (283 kg)
507788	Bariatric	625 lb (283 kg)

Disposable

Part No.	Size	SWL
547102-10	Standard	450 lb (204 kg)
547103-10	Large	450 lb (204 kg)
547104-10	Bariatric	450 lb (204 kg)


Rehab Total Support System



Standing and Walking Support Sling

- Provides support around the torso with dual adjustable Velcro enclosures
- Designed for transfers, standing, gait training and fall prevention
- Padded leg harness is attached to the torso support via adjustable buckle enclosures

Materials

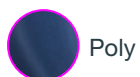

Poly

Applications



Support for gait training



Standing training



Rising – sitting down

Part Numbers

Poly

Part No.	Size	SWL
510421	Junior	800 lb (363 kg)
510422	S	800 lb (363 kg)
510423	M	800 lb (363 kg)
510424	L	800 lb (363 kg)
510425	XL	800 lb (363 kg)
510426	XXL	800 lb (363 kg)

Additional Leg Straps

Part No.	Size	SWL
510427	Junior	800 lb (363 kg)
510428	S	800 lb (363 kg)
510429	M	800 lb (363 kg)
510430	L	800 lb (363 kg)
510431	XL	800 lb (363 kg)
510432	XXL	800 lb (363 kg)


Full Standing Support Sling



Sit to Stand and Walking Support Sling

- Provides support around the torso with adjustable dual buckle enclosures
- Designed for transfers, sit to stand, standing, gait training, and fall prevention
- Thigh supports feature adjustable straps with buckle enclosure support

Materials

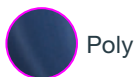

Poly



Disposable

Applications



Support for gait training



Standing training



Rising – sitting down

Part Numbers

Poly

Part No.	Size	SWL
507905	Junior	600 lb (272 kg)
507910	S	600 lb (272 kg)
507920	M	600 lb (272 kg)
507930	L	600 lb (272 kg)
507940	XL	600 lb (272 kg)
507950	XXL	600 lb (272 kg)

Disposable

Part No.	Size	SWL
547111-10	Junior	625 lb (283 kg)
547112-10	S	625 lb (283 kg)
547113-10	M	625 lb (283 kg)
547114-10	L	625 lb (283 kg)
547115-10	XL	625 lb (283 kg)
547116-10	XXL	625 lb (283 kg)


Stand Aid Sling



Sit to Stand Transfer Sling

- Designed to provide support for sit to stand transfers, toileting, and standing training
- Provides support from mid to low back, around the torso with velcro and buckle enclosure
- Optional non slip back available with velcro enclosure

Materials

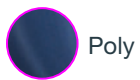

Poly


Applications



Standing training



From bed to wheelchair

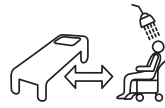

From bed to commode



From wheelchair to toilet



Rising – sitting down



Positioning in chair

Part Numbers

Poly

Part No.	Size	SWL
280070	Junior	600 lb (272 kg)
280071	S	600 lb (272 kg)
280072	M	600 lb (272 kg)
280073	L	600 lb (272 kg)
280074	XL	600 lb (272 kg)
280075	XXL	600 lb (272 kg)

Non-Slip

Part No.	Size	SWL
521041	Junior	600 lb (272 kg)
521042	S	600 lb (272 kg)
521043	M	600 lb (272 kg)
521044	L	600 lb (272 kg)
521045	XL	600 lb (272 kg)
521046	XXL	600 lb (272 kg)

Thorax Sling



Sit to Stand Transfer Sling

- Designed to provide support for sit to stand transfers, toileting, standing training and gait training
- Provides support from the mid to low back around the torso with buckle enclosure
- Optional seat support and arm support are available, fabric dependent

Materials



Disposable


Applications



Standing training



From bed to wheelchair

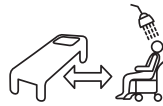

From bed to commode



From wheelchair to toilet



Rising – sitting down



Positioning in chair

Part Numbers

Disposable

Part No.	Size	SWL
45690004	S	440 lb (200 kg)
45690006	M	440 lb (200 kg)
45690007	L	440 lb (200 kg)
45690008	XL	440 lb (200 kg)

Situations Table

	Supine transfer	Turning in bed	Higher up in bed	Up from the floor	Support for gait training	Standing training	From bed to wheelchair	From bed to commode	From wheelchair to toilet	Transfer into tubs	Rising – sitting down	Positioning in chair	Support for limbs
Lifting Slings	Page												
Universal Sling	6-7		••				•	•	•			•	
Deluxe Hammock Sling	8-9		••				•	•	•			•	
Hammock Sling	10-11		••				•	•	•			•	
Comfort Care Sling	12-13		••				•	•	•			•	
Hygiene Sling	14						•	•	•				
Dual Access Sling	15						•	•	•				
Band Sling	16											•	
Positioning Sling	17	•											
Tri-Turner Sling	18	•											
Rehab Total Support System	19			•		•				•			
Full Standing Support Sling	20			•		•				•			
Stand Aid Sling	21					•				•		•	
Thorax Sling	22					•				•		•	

*commode opening optional on Comfort Care Slings **requires head support

Are you ready to choose from our
comprehensive line of slings?

To learn more, call 1-888-637-8155 or
visit www.handicareusa.com / handicare.ca

HANDICARE

10888 Metro Court, St. Louis, MO 63043
Toll Free: 866.891.6502 | Fax: 314.692.7858
customerservice.northamerica@handicare.com
www.handicareusa.com

81 Romina Drive, Concord, ON, L4K 4Z9
Toll Free: 877.304.5438 | Fax: 416.260.0256
customerservice.canada@handicare.com
www.handicare.ca

