**2-Post Pressure Fit System**  
(341500)

**3-Post Pressure Fit System**  
(341502)

**4-Post Pressure Fit System**  
(341503)

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SWL: 450 lb / 204 kg

# Contents

<b>Requirements and Cautions</b> .....	<b>3</b>
<b>Requirements</b> .....	<b>3</b>
<b>Cautions</b> .....	<b>3</b>
<b>Use and Care</b> .....	<b>5</b>
<b>Introduction</b> .....	<b>5</b>
<b>Overview of the Pressure Fit System Lift System</b> .....	<b>5</b>
<b>Determine the Space for the Post Pressure Fit System Installation</b> .....	<b>8</b>
<b>Assemble the Supporting Posts</b> .....	<b>9</b>
<b>Mount the Adjustable Track</b> .....	<b>15</b>
<b>Install the Traverse Track</b> .....	<b>17</b>
<b>Perform the Final Inspection Before Use</b> .....	<b>18</b>
<b>Mount the Portable Lift</b> .....	<b>18</b>
<b>Disassemble the Pressure Fit System</b> .....	<b>18</b>
<b>Contraindications</b> .....	<b>24</b>
<b>Troubleshooting</b> .....	<b>25</b>
<b>Troubleshooting Chart</b> .....	<b>25</b>
<b>General Inspection and Preventive Maintenance</b> .....	<b>27</b>
<b>Purchase and Service Information</b> .....	<b>29</b>
<b>Service Record History</b> .....	<b>30</b>
<b>Display Symbols</b> .....	<b>31</b>
<b>Warranty</b> .....	<b>31</b>
<b>About Handicare</b> .....	<b>31</b>

## Requirements and Cautions

Check all Requirements and Cautions before you start!

### Requirements

Note: Failure to meet the listed requirements could result in serious injury or death.

#### Ceiling Requirements

- Install the Pressure fit system against a rigid ceiling structure (i.e. drywall ceiling).
- Ceilings other than drywall e.g., wood) must be at least 12.7mm (1/2") thickness.
- Ceiling structure must have joists that are spanned a maximum of 24" on center (no more than 24" between joists).
- Pressure Fit systems cannot be used where ceiling structures are suspended by wire or hooks (e.g., T-bar ceilings or tile ceilings).
- Pressure Fit systems cannot be installed in rooms with plaster ceilings or textured ceilings where over compression from Pressure Fit system is possible.

The ceiling structure must be:

- Fixed
- Capable of sustaining upward pressure from the Pressure Fit system without bending or compressing the ceiling structure material

#### Floor Requirements

- Ensure the floor area is clean and free of dirt and debris.
- The floor should meet the requirements of building code standards.
- The floor must not have a slope of more than .5 degrees. See Figure 1 to determine the allowable slope of the floor for an installation.
- When installing the Pressure Fit system on carpets, take extra care with carpet with long pile (i.e., shag) or thick carpet padding to ensure the Pressure Gauge remains in the safe zone (green on both sides) throughout the use of the system (see Figure 19).

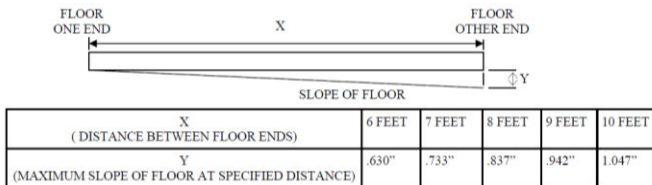
Figure 1: Allowable Floor Slope

### Cautions

Failure to meet the listed cautions could result in serious injury or death.

#### Before Installation

- Pressure Fit System parts should not be loose, broken, or bent. *If any part is missing or damaged, do not install.*
  - Contact your local authorized dealer immediately.

- Do not install the Pressure Fit System to suspended ceiling.
- Do not install the Pressure Fit System on a plaster ceiling.
- Do not install the Pressure Fit System on a sloped floor greater than .5 degrees.
- Ensure the floor area is clean and free of dirt and debris.
- Never expose the Pressure Fit System directly to water. Warranty does not cover any misuse or abuse of the 2-Post Mount Pressure Fit System.
- The Pressure Fit System must be assembled prior to use. Should you have any questions during assembly contact your local authorized dealer.
- Check any accessories used with the Pressure Fit System, including lifts and slings, to ensure that they are in good working order. Check for signs of wear or fraying prior to use. Report any unusual wear or damage immediately to your local authorized dealer.
- Never install the Bottom Foot Plate on the top portion of post section or Top Foot Plate on the bottom portion of post section. See “Assemble the Supporting Posts” (page 9).
- Ensure that a clear space is maintained around the lift and track. Move all furniture and other obstacles out of the way before performing a transfer.
- Ensure the rubber feet are cleaned and dry prior to each use as described in “General Inspection and Maintenance—Each Use” (page 28).

## After Installation

- Ensure the Wall Mount track is levelled. Use a master level to verify the straightness.
- Ensure the Wall Mount does not shake after installation.
- Ensure the Plate Bolts are secured properly.
- Ensure the Wall Mount Pivot Bracket is secured properly.
- Ensure that the pressure gauge is in the safe zone during set up and before each use.
- Ensure the Post is level, both visually and with the built in level.
  - *If you are unsure, use a master level to verify.*
- Ensure the track is sitting properly on the post pins.
- Ensure the trolley slides smoothly on the track.
- Ensure the safety lock button and level are locked.
- Ensure the end stop for the swivel trolley is working properly.
- Never unlock the Post when the Track is attached.
- Never exceed the maximum load rating for this piece of equipment.
  - Do not use a P-600 lift.
  - Refer to the “Specifications” section of this manual.
- The installation of the Pressure Fit System, lift, accessories, and sling are certified to a maximum load.
  - Do not exceed the maximum rated load of any of the components.
- The Pressure Fit System has been designed to lift vertically at its maximum load and at its maximum height.
  - Do not attempt to lift an individual at an angle to the track.
- The Pressure Fit System and associated lift and slings are intended only for lifting and transferring a person.
- Handicare will not be responsible for any damage caused by the misuse, neglect or purposeful destruction of the lift and/or its associated components.
- Under no circumstance should the track, lift and slings, or entire system be controlled by a person who has not been properly trained in the use and care of this equipment. Failure to adhere to this warning may result in serious injury to the operator, and/or the individual being lifted/transferred.
- In places where more than one operator is responsible for using the Pressure Fit System, associated lift and slings, all operators must be trained in its proper use.
- The Pressure Fit System and associated lift and slings are not toys. Do not use them for unsafe practices. Do not allow children to play with this equipment or any of its components.
- To maintain optimum function, inspect and maintain the Pressure Fit System, on a regular basis. See “General Inspection and Preventive Maintenance” (page 27).
- The manufacturer’s warranty is void if persons unauthorized by Handicare perform work on the Pressure Fit System.

# Use and Care

## Introduction

To ensure safe operation of your Pressure Fit System lift, before using this equipment, carefully read this entire manual, especially the sections “Requirements” (page 3) and “Cautions” (page 3). The Pressure Fit System is designed to be used in conjunction with Handicare/Prism Medical lifts, track, accessories, and slings; Refer to the user guides supplied with these parts before using them and refer to them while reviewing this manual.

Be sure you understand the contents of this manual completely before using this equipment.

**Failure to comply with warnings in this manual or proper lifting instructions may result in injury to the operator or the individual being lifted or transferred, as well as damage to the lift and related parts.**

If you have any questions after reviewing this manual, contact your local Handicare dealer or your local Handicare business development manager.

Store this manual with the documents included with the lift system, sling, and all related parts. Contents of this manual are subject to change without prior written notice.

## Overview of the Pressure Fit System Lift System

The Pressure Fit System is a lifting aid used by health care professionals and caregivers in homes, institutions, and long-term care facilities to lift, position, and transfer clients or a disabled family member.

The Pressure Fit System is one part of a ceiling lift system that lifts an individual directly from above, making it possible to transfer and reposition mobility impaired individuals safely, with dignity and in comfort, while minimizing strain and risk to the caregiver. The ceiling lift also does not take up valuable floor space as most traditional methods do.

The Pressure Fit System is designed to work with a Handicare portable lift system such as the P-440 or P-300. However, make sure when using the portable lift units with this system, the weight should not go beyond the capacity of the Pressure Fit System or of the Lift capacity. For inquiries about portable units, contact your local Handicare dealer.

The Pressure Fit System lift is one of the four major parts that make up a ceiling lift system:

- **Lift** - Motorized unit that raises and lowers an individual directly from above
- **Track** - Custom tailored and securely mounted to a ceiling structure, wall or floor, provides the path for the lift to traverse
- **Trolley** - Mounted inside the track, the wheeled trolley allows the lift to move along the track; the lift attaches to the trolley via the lift's carabiner and the eyelet of the trolley
- **Sling** - Specially-designed fabric accessory that attaches to the lift's carry bar to hold and support an individual for positioning or transfer

Refer to any user guides supplied with the accessories and reference them while reviewing this lift guide.

# Component List

## 2-Post Pressure Fit System Components

Description	Qty
Bottom Foot	2
Top Foot	2
Supporting Post	2
Adjustable Track	1
Owner's Manual	1
Warranty Card	1

## 3-Post Pressure Fit System Components

Description	Qty
Bottom Foot	3
Top Foot	3
Supporting Post	3
Adjustable Track	2
Swivel Trolley	1
Lift Trolley	1
Owner's Manual	1
Warranty Card	1

## 4-Post Pressure Fit System Components

Description	Qty
Bottom Foot	2
Top Foot	2
Supporting Post	1
Adjustable Track	1
Swivel Trolley	1
Lift Trolley	1
Owner's Manual	1
Warranty Card	1

## Specifications

- High-strength lightweight aluminum
- Portable or semi-permanent installation
- Eliminates the mounting of a connection system in the ceiling
- Pressure gauge indicator to help ensure proper installation
- Quick attachment with secure fixings
- Addresses a number of different room and bath tubs situations
- Weight capacity: 450 lb (204 kg)
- Adjustable in length
- Built-in easy-slide trolley
- Exerting pressure between floor and ceiling is 80 to 100 lb (36,35 kg to 45.45 kg)
- Specially designed and engineered rubber molded feet for ultimate grip
- Leveling indicators on posts
- Expected product lifespan: 7 years

## Minimum and Maximum Dimensions

Figure 2, Figure 3, and Figure 4 show the minimum and maximum dimensions for setting up 2, 4, and 4-post pressure fit systems.

**Do not use the system outside of this range of placement—serious injury may occur to the operator of the lift and/or the individual being transferred, and/or the Post Pressure Fit System otherwise.**

### 2-Post Pressure Fit System



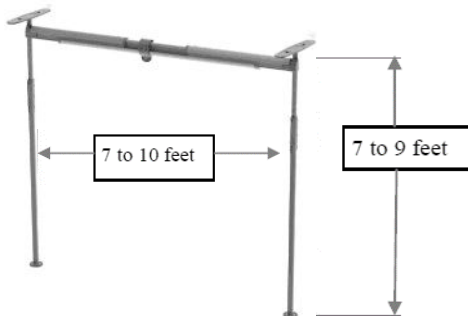
Figure 2: Minimum and Maximum Dimensions—2-Post Pressure Fit System

### 3-Post Pressure Fit System

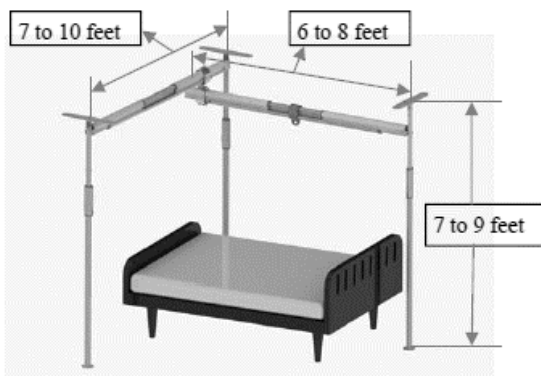**Figure 3: Minimum and Maximum Dimensions—3-Post Pressure Fit System**

### 4-Post Pressure Fit System

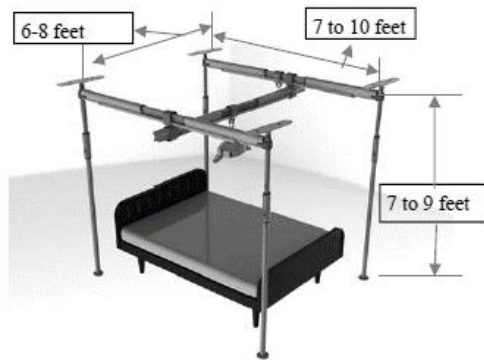**Figure 4: Minimum and Maximum Dimensions—4-Post Pressure Fit System**

## Determine the Space for the Post Pressure Fit System Installation

To determine the space for the pressure fit system:


1. Examine the room where you plan to install the Pressure Fit System.
  - Make sure the room meets all the specifications outlined in “Ceiling Requirements” (page 3) and “Floor Requirements” (page 3).
  - Use the measurements in “Minimum and Maximum Dimensions” (page 7) to determine the best spot to place the pressure fit system.
2. Install the pressure fit system 36”–40” away from the wall or the bed’s headboard (Figure 5).

**Figure 5: Recommended Placement Distance from Wall or Headboard (2-Post System Shown)**

- Allow 36” (1000 mm) of open area for transfer working space (Figure 6).

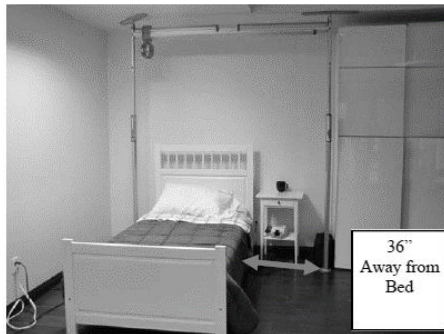
**Figure 6: Recommended Open Area for Transfer Working Space (2-Post System Shown)**

## Assemble the Supporting Posts

- ⚠ Before using a 2, 3, or 4-Post Pressure Fit System, visually check the top plate, bottom plate, supporting posts, and adjustable track to ensure there are no missing parts or unusual wear and tear.
- ⚠ Clean and dry the top plate and bottom plate with a wet cloth before installation as described in “General Inspection and Maintenance—Each Use” (page 28).
- ⚠ Failure to comply with these cautions could result in serious injury to the operator, the individual being lifted, and/or damage to the pressure fit system and/or portable lift unit.

To assemble the supporting posts,


1. Place the Bottom foot of one supporting post on the floor.
2. Insert the bottom plate through the opening located at the bottom of the Post.
  - *Do not install the top foot at the bottom of the post.*

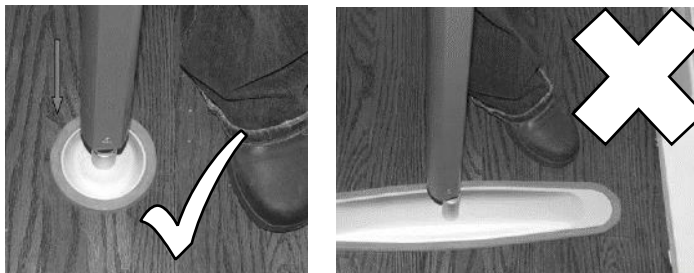**Figure 7: Post with Bottom Foot Installed Correctly (Left) and with Top Foot Installed Incorrectly (Right)**


3. Tilt the supporting post to the side to facilitate the installation of the top foot.

4. Insert the top plate through the top opening of the supporting post.
- Never install bottom foot at the top section of the post.

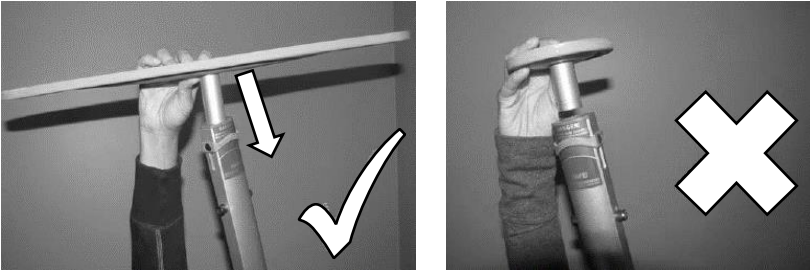
**Figure 8: Post with Top Foot Installed Correctly (Left) and with Bottom Foot Installed Incorrectly (Right)**

5. Press the red button down to release the safety lock.

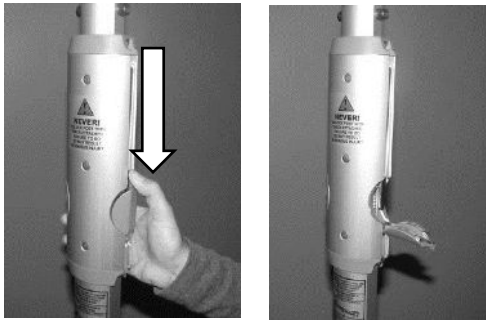
**Figure 9: Release the Safety Lock (Left) and Safety Lock Released (Right)**

6. Grab the top section of the post with two hands and push down.

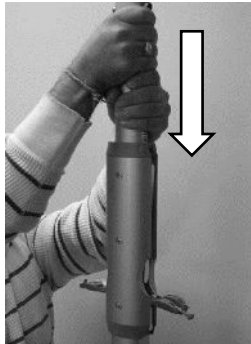
**Figure 10: Push Down on the Top Section of Post with Two Hands**

- The lever moves out (Figure 11).

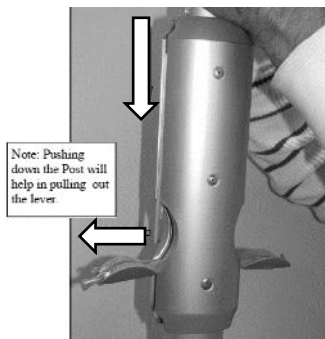**Figure 11: Lever Out**

7. Press in and hold the ratchet while pulling out the lever.

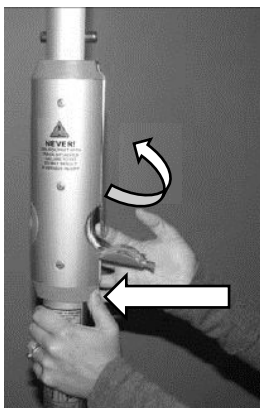**Figure 12: Hold the Ratchet While Pulling the Lever**


8. Holding the ratchet, push the lever up to engage the magnet.

**Figure 13: Engage the Magnet**

9. Push the top section of the post up towards the ceiling.

- When the top foot meets the ceiling, continue pushing until you hear one or two more clicks.

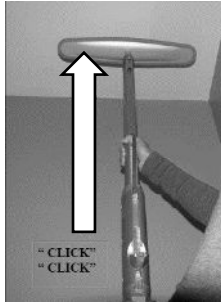
**Figure 14: Push the Top Section of the Post Up**

10. Swing open the post level.
  - Ensure the level is open fully.


**Figure 15: Open the Post Level Fully**

11. Level the post:
  - a. Grasp the bottom section of the post.
  - b. Lift the post slightly.
  - c. Adjust the post's position until the bubble is centered and the post is perpendicular to both the floor and ceiling.
    - Use the bubble as a guideline, then verify the alignment with a master level.

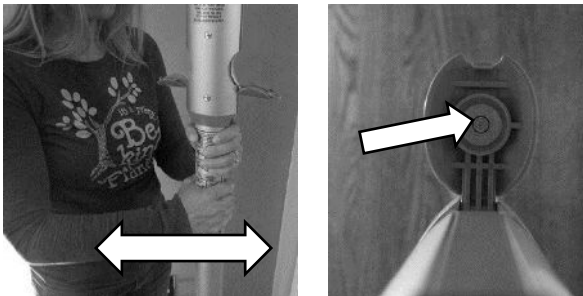
**Figure 16: Level the Post**

12. Push the lever down until it clicks into place.

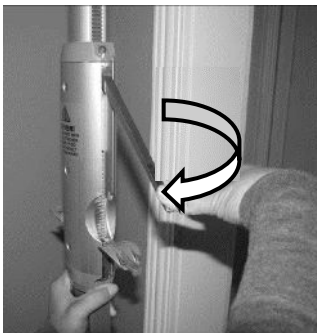**Figure 17: Push the Lever Down**

13. Push the lever in to secure it, until you hear a click.
- *The lever must rest in a completely vertical position!*

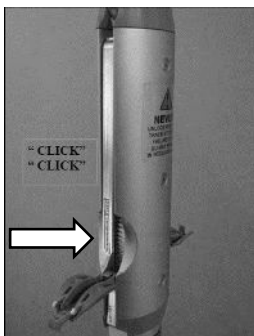
**Figure 18: Push the Lever in Until It Clicks**

14. Ensure the pressure gauge is in the safe zone.
- *Green must be visible both above and below the gauge!*

**Figure 19: Pressure Gauge in the Safe Zone**

15. Swing the safety lock back to its original position.

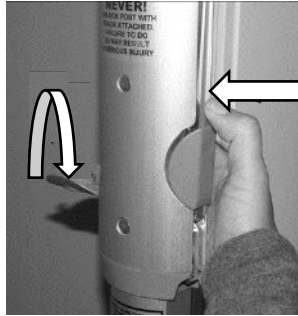**Figure 20: Safety Lock in its Original Position**

16. Ensure the safety lock and level are firmly engaged.

**Figure 21: Safety Lock and Level Engaged**

17. Repeat steps 1–16 to install for each supporting post.

## Mount the Adjustable Track

Mount adjustable track sections to connect the supporting posts, as shown in Figure 22.

**Figure 22: Adjustable Track Connecting Supporting Posts (2-Post System Shown)**


**Before mounting the adjustable track, ensure the lifting height is sufficient to lift the patient.**

**Ask for assistance if you require help installing the track. Failure to comply with this caution could result in serious injury to the operator, the individual being lifted and/or damage to the Pressure Fit and/or portable lift unit.**


To mount the adjustable track to the installed supporting posts:

1. Get a step ladder.
  - A step ladder is required to perform this assembly.
2. Hook one end of the adjustable track to the post pin.

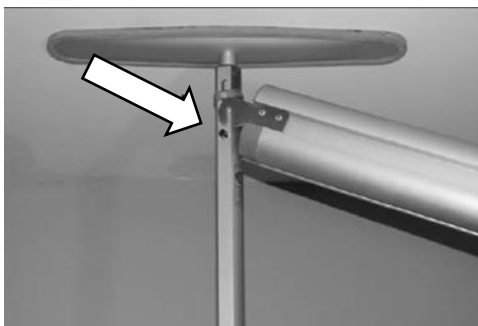
**Figure 23: Adjustable Track Hooked to the Post Pin**

3. Extend the track and hook to the other post.

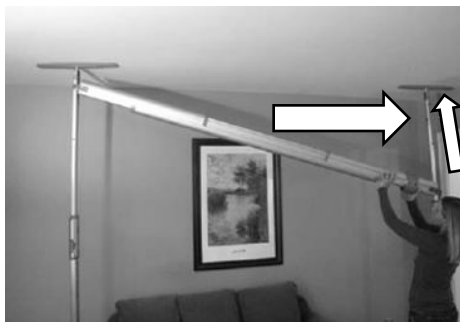
**Figure 24: Adjustable Track Extended to the Other Post**

- Ensure the track connector is sitting properly on the spring pin—Don't leave the track connector resting above the spring pin (Figure 25).

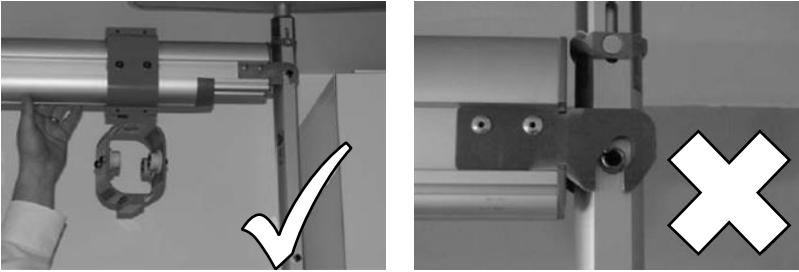**Figure 25: Proper Spring Pin Placement**

4. Attach a track end cap to each end of the adjustable track.

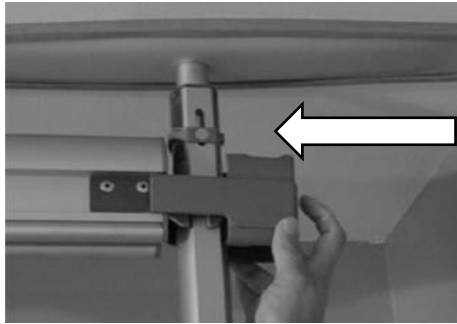**Figure 26: Track End Cap**

5. Repeat steps 1–4 for each adjustable track to install.

## Install the Traverse Track

If your pressure fit system includes a traverse track, install it after you set up the supporting posts and adjustable track.


To install the traverse track:

1. Insert the traverse track into the trolley until the pin hits the trolley.

**Figure 27: Traverse Track in the Trolley (Pin Against the Trolley)**


2. Holding the track in place with one hand, with the other hand reach a finger through the track slot and pull down on the pin ring.

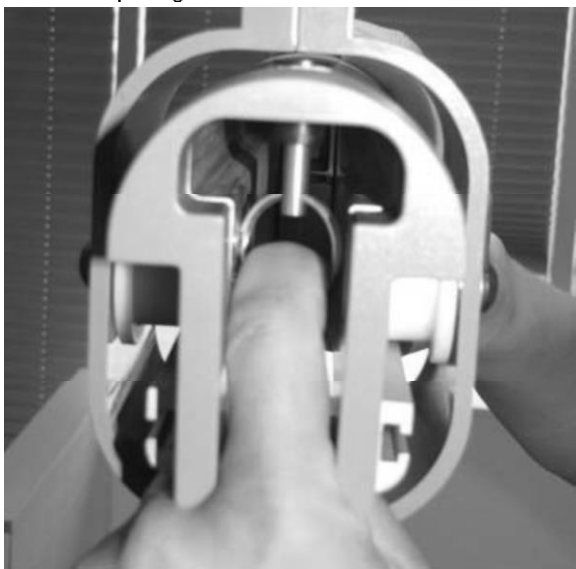**Figure 28: Pull Down on the Pin Ring**

3. Slide the traverse track through the trolley until it's out the other side.

4. Release the pin ring.
  - The pin pops up, preventing the traverse track from sliding back through the trolley.

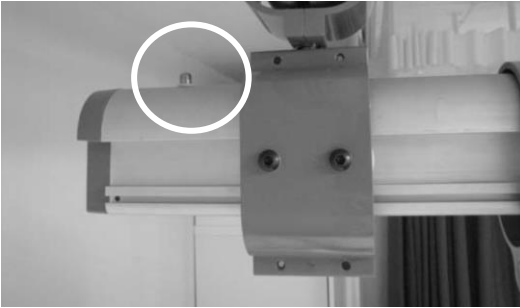**Figure 29: Pin Popped Up to Prevent the Traverse Track from Sliding Out**

5. Repeat steps 1–4 for the opposite end of the traverse track.
6. Take hold of the traverse track and slide it from one end to the other.
  - While moving the Traverse Track there will be some resistance.
  - Push lightly to move it from end to end of the adjustable track.
  - Make sure the traverse track goes all the way to both ends of the adjustable track.
7. Bring the track to the middle and make sure the end of the traverse track does not make contact with the wall.

## Perform the Final Inspection Before Use

1. Ensure the Pressure Gauge is in the Safe Zone.
  - Green must be visible on both sides.
  - Confirm this before each use, as it may depress over time.
2. Ensure the safety lock and level are firmly engaged in housing.
3. Ensure the end covers are installed properly.

## Mount the Portable Lift

Follow the instructions included with your portable lift for:

- Installing the lift
- Attaching the carry bar
- Charging the lift
- Operating the lift

## Disassemble the Pressure Fit System

Failure to comply with these cautions could result in serious injury to the operator, the individual being lifted, and/or damage to the pressure fit system and/or portable lift unit.


To disassemble a 2, 3, or 4-Post Pressure Fit system:

1. Remove the traverse track (page 19).
2. Remove the adjustable track (page 19).
3. Disassemble the supporting posts (page 21).

## Remove the Traverse Track

If your pressure fit system includes a traverse track, begin by removing it.

1. Holding the track in place with one hand, with the other hand reach a finger through the track slot and pull down on the pin ring.

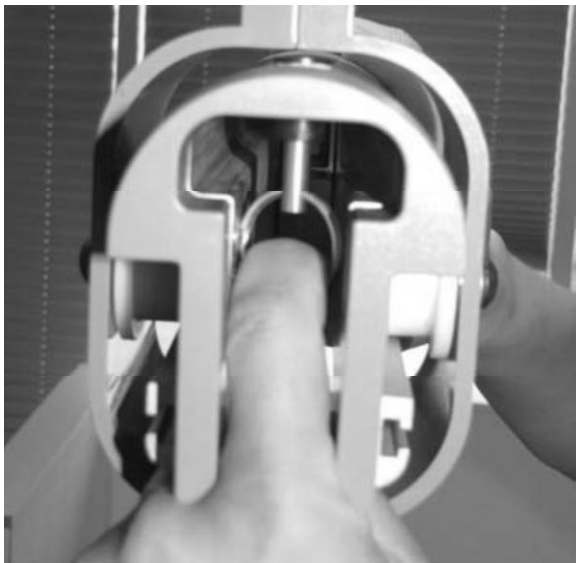**Figure 30: Pull Down on the Pin Ring**

2. Slide the traverse track through the trolley until it's out the other side.
3. Release the pin ring.
  - The pin pops up.
4. Repeat steps 1–3 for the opposite end of the trolley.
5. Remove the traverse track from the trolley.

**Figure 31: Remove the Traverse Track from the Trolley**

## Remove the Adjustable Track

1. Get a step ladder.
  - A step ladder is required to perform this assembly.
2. Remove the end caps from both posts.

**Figure 32: Remove the End Caps from Both Posts**

3. On one end, lift the track up to unhook it from the post.

**Figure 33: Lift Up to Unhook the Track**

4. Squeeze the track and unhook it from the post at the opposite end.

**Figure 34: Squeeze and Unhook the Track**

## Disassemble the Supporting Posts


**Never unlock a supporting post with the track connected.**

**Failure to read and follow these instructions carefully may result in serious injury.**

To disassemble the supporting posts:

1. Press the red button to release the safety lock.

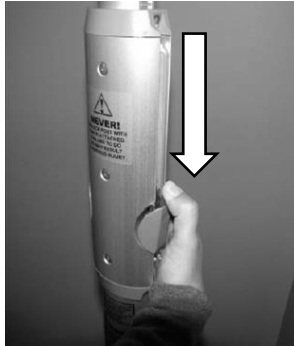**Figure 35: Release the Safety Lock**

2. Press the lever in all the way to release the pressure from the ratchet.


**Figure 36: Release Pressure from the Ratchet**

3. While pressing in the lever, push and hold the ratchet.


**Figure 37: Push and Hold the Ratchet**

4. While holding the ratchet, pull out the lever.

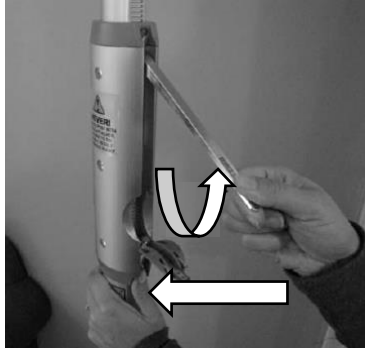
**Figure 38: Pull Out the Lever**

5. Push the lever up to engage with magnet.

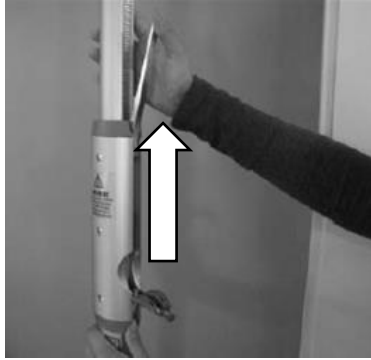**Figure 39: Push the Lever to Engage the Magnet**

6. Push in and hold the ratchet while pushing the top section of post down.

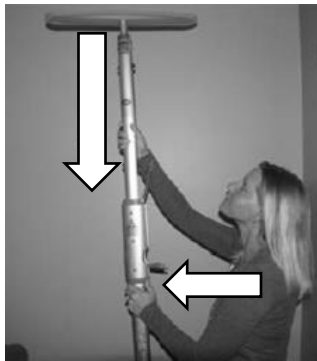
**Figure 40: Hold the Ratchet while Pushing Down the Post's Top Section**

7. Make sure the bottom pin touches the plastic cover.

**Figure 41: Bottom Pin Touches the Plastic Cover**

8. Return the lever to the vertical position.

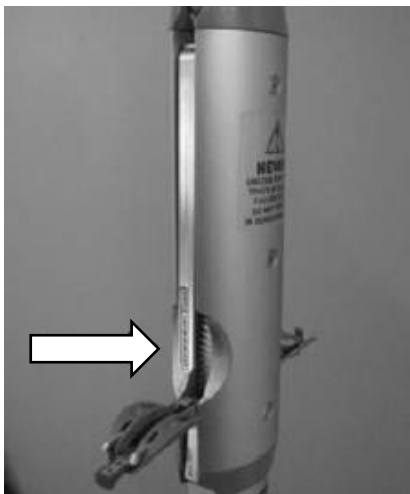
**Figure 42: Lever at Vertical Position**

9. Return the level and safety lock to their original positions.

**Figure 43: Level and Safety Lock at Original Positions**

10. Remove the top foot and bottom foot from the supporting post.

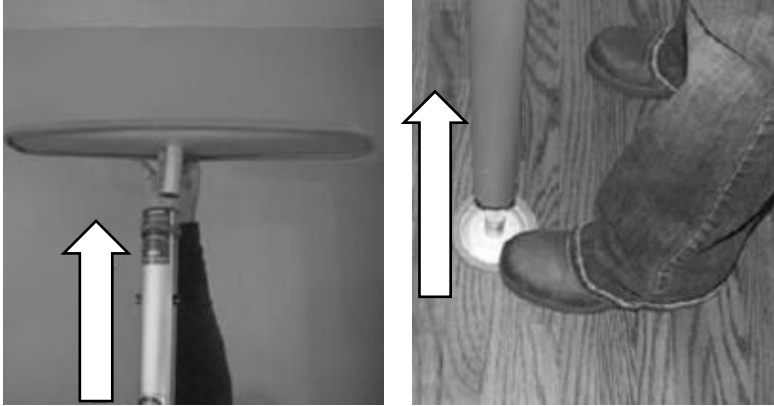**Figure 44: Remove the Supporting Post's Top Foot and Bottom Foot**

11. Repeat steps 1–10 for each supporting post.

## Contraindications

There are no known “contraindications” associated with the use of Pressure Fit System lifts and their accessories, provided they are used according to the manufacturer’s recommendations and guidelines.

## Troubleshooting

If you encounter problems using the Pressure Fit System, review the Troubleshooting chart.

### Troubleshooting Chart

Use the Fault column to locate the fault you've encountered, then complete the recommended solution in the adjacent column on the row.

If you cannot find the fault or the recommended solution does not correct the problem, immediately contact your local Handicare authorized dealer for service.

Fault	Possible Reason	Recommended Solution
Visually, the system is not aligned straight	<ul style="list-style-type: none"> <li>• Level is not working</li> <li>• Floor is not leveled</li> <li>• Level is damaged</li> <li>• Post was knocked with impact</li> </ul>	<ul style="list-style-type: none"> <li>• Set up the post following Post Assembly instructions. Check visually and with master level to ensure the post is aligned straight. <b>Note: Never unlock the post with the track connected.</b></li> <li>• Check the floor for flatness using a master level.</li> <li>• Visually check the level is fine.</li> <li>• Dismantle the post following "Disassemble the Supporting Posts" (page 21), then assemble the post following "Assemble the Supporting Posts" (page 9). <b>Note: Never unlock the post with the track connected</b></li> <li>• <i>If the problem persists</i>, contact your local authorized dealer immediately</li> </ul>
Feet are slipping after installation	<ul style="list-style-type: none"> <li>• Feet are not cleaned</li> <li>• Floor is not cleaned</li> </ul>	<ul style="list-style-type: none"> <li>• Clean the rubber feet as described in "General Inspection and Maintenance—Each Use" (page 28).</li> <li>• Ensure the floor area is dry and free of debris.</li> <li>• <i>If the problem persists</i>, contact your local authorized dealer immediately</li> </ul>
Lever is not engaged with magnet	<ul style="list-style-type: none"> <li>• Post Assembly instructions not followed</li> <li>• Clutch bearing function fails</li> <li>• Lever not cleaned</li> </ul>	<ul style="list-style-type: none"> <li>• Press in the ratchet and hold it. Grab the lever and push up to the magnet label.</li> <li>• Ensure the lever is cleaned and free of grease or dirt.</li> <li>• <i>If the problem persists</i>, contact your local authorized Handicare dealer immediately.</li> </ul>
Pressure gauge is not in safe zone	<ul style="list-style-type: none"> <li>• Post assembly instructions not followed.</li> <li>• Barrel nut is loose or not secured tightly</li> <li>• Top plate Spring broken or bent</li> </ul>	<ul style="list-style-type: none"> <li>• Set up the post following the post following "Assemble the Supporting Posts" (page 9).</li> <li>• Tighten the barrel nut using a flat screw driver.</li> <li>• Contact your local authorized Handicare dealer immediately.</li> </ul>

Fault	Possible Reason	Recommended Solution
The Adjustable Track does not extend smoothly or doesn't extend up to required limit	<ul style="list-style-type: none"> <li>• Physical damage to track</li> <li>• Track slots are not cleaned</li> </ul>	<ul style="list-style-type: none"> <li>• Check for physical damage to the track. <i>If so</i>, contact your local authorized Handicare dealer immediately.</li> <li>• Clean the track slots with a cloth and wipe out the dirt or debris, then try again to extend the tracks.</li> <li>• <i>If the problem persists</i>, contact your local authorized dealer immediately.</li> </ul>
Trolley does not move smoothly along the adjustable track	<ul style="list-style-type: none"> <li>• Physical damage to track or trolley wheels</li> <li>• Track slots or trolley wheels are not cleaned</li> </ul>	<ul style="list-style-type: none"> <li>• Visually make sure the track and wheels are functioning and free of obstacles.</li> <li>• Ensure that the track is cleaned with cloth and free of dirt.</li> <li>• <i>If the problem persists</i>, contact local authorized Handicare dealer immediately.</li> </ul>
The adjustable track is not locking on the Post Pins	<ul style="list-style-type: none"> <li>• One or more track connector brackets are damaged or broken</li> <li>• Track lock pin is broken or damaged</li> </ul>	<ul style="list-style-type: none"> <li>• Do not use the system.</li> <li>• Contact your local authorized dealer immediately.</li> </ul>
Misalignment of adjustable track on the post pins. (for Example; track is not sitting straight on the post assembly)	<ul style="list-style-type: none"> <li>• Track connector hooks are not sitting right on the post pins</li> </ul>	<ul style="list-style-type: none"> <li>• Follow "Mount the Adjustable Track" (page 15).</li> <li>• Check visually to ensure the track connector hooks are sitting at the same height.</li> <li>• <i>If the problem persists</i>, contact your local authorized Handicare dealer immediately.</li> </ul>
Post cannot retract	<ul style="list-style-type: none"> <li>• Ratchet is not held during disassembly</li> <li>• Lever arm is not engaged with magnet</li> </ul>	<ul style="list-style-type: none"> <li>• Dismantle the post following "Disassemble the Supporting Posts" (page 21).</li> <li>• Assemble the post following "Assemble the Supporting Posts" (page 9).</li> </ul>
Safety lock not working	<ul style="list-style-type: none"> <li>• Red button is not pressed in enough</li> <li>• Physical damage to safety lock cover</li> <li>• Lock latch spring broken</li> </ul>	<ul style="list-style-type: none"> <li>• Press the red button all the way in so that the safety lock can be engaged.</li> <li>• Contact your local authorized Handicare dealer immediately.</li> </ul>

Fault	Possible Reason	Recommended Solution
Difficult to insert the top and bottom foot into the post assembly.	<ul style="list-style-type: none"> <li>• Top post spacer damaged</li> <li>• Bottom post plug damaged</li> <li>• Bottom plate foot post damaged or bent</li> <li>• Top plate foot post damaged or bent</li> </ul>	<ul style="list-style-type: none"> <li>• Contact your local authorized dealer immediately to continue the proper functioning of Pressure Fit System.</li> </ul>
Track connector brackets are loose.	<ul style="list-style-type: none"> <li>• Rivets are broken or loose</li> </ul>	<ul style="list-style-type: none"> <li>• Do not use the system.</li> <li>• Contact your local authorized dealer immediately.</li> </ul>

## General Inspection and Preventive Maintenance

Periodic general inspection and maintenance should be performed by a person who is properly qualified and trained with the use and care of The Pressure Fit System.

Any defects and damage to the Pressure Fit System that have led to corrective actions should be noted and dated by the inspector. The defects and corrective actions report should be submitted in written form to the authorized Handicare dealer.

**Do not perform any inspection or maintenance while the lift is in use with a patient.**

**Failure to follow inspection and maintenance instructions below as instructed may result in serious injury or death.**

### Upon Receipt of the Pressure Fit System

- Ensure all the received components are according to the component list.
- Ensure the user manual is present otherwise do not install the pressure fit system.
- Ensure all the components are in working condition and not damaged.
- Ensure service record history forms are included in the package to record any completed service and repairs.
- Complete the “Purchase and Service Information” as soon as the equipment is installed.

### Periodic Maintenance

- Ensure all the components of pressure fit system are in working condition.
  - *If any component is damaged*, contact your authorized Handicare dealer for a replacement before using the pressure fit system.
- Inspect all the joints and components for wear and fatigue.
  - *If the parts are damaged*, contact dealer for replacement of parts.
- Clean the rubber feet as described in “General Inspection and Maintenance—Each Use” (page 28).
- Check the rubber feet for damage.
  - *If there is damage*, contact your authorized Handicare dealer for replacement.
- Check the track and trolley for dirt and debris and clean the track and trolley with a cloth every use.
- Ensure the labels are legible and in good condition.
  - *If they are not*, contact dealer for replacement.
- Record the service record history form for all completed service and repairs.
- Sign and date the service record each time the pressure fit system is used.

## General Inspection and Maintenance—Each Use

*To be completed by the user:*


Prior to each use, visually inspect the Pressure Fit System and the associated lift, accessories, and slings.

- Refer to the lift, accessory, and sling user guides for specific details regarding their inspection.

**Should any of the items below fail the inspection do not use the Pressure Fit System. Contact your authorized Handicare dealer for service.**

- Inspect the portable lift as detailed in its owner's manual.
- Inspect the sling as detailed in its owner's manual.
- Visually check the following:
  - The pressure fit system parts are not loose, broken or bent.
  - The ceiling area is rigid and without cracks, dust, grease, or any slippery substance.
  - The bases are on a stable, level surface and the side support assemblies are secure.
  - The pressure gauge is in the green "safe" zone (Figure 19).
  - The trolley moves easily along the track.
  - The adjustment track is in the correct location for transferring.
- Clean the bottom and top rubber feet with a wet cloth soaked with water to remove dirt and debris.
  - After cleaning them, make sure the rubber feet are clean and dry, with no residue of dirt or dust left on them, and ensure the floor area is clean and free of dirt and debris.

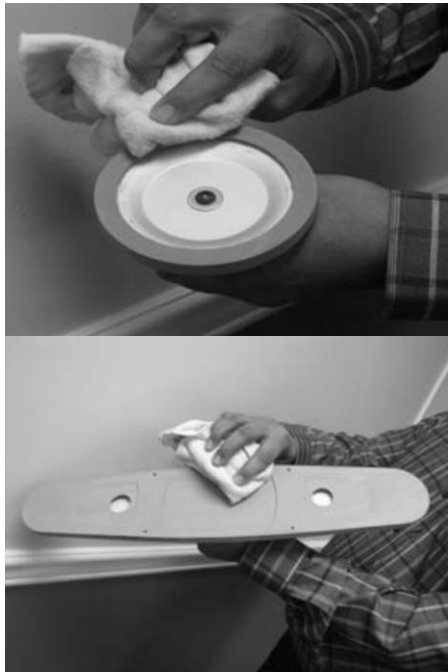**Figure 45: Wipe Off the Bottom (Left) and Top (Right) Rubber Feet**

## Purchase and Service Information

As soon as this equipment is installed, complete the Purchase Information and Service Information sections.

- Use the service record history to record any completed service and repairs.
- Ensure that the service record is signed and dated each time it is used.
- Be sure to have this piece of equipment serviced on a regular basis as described in "General Inspection and Preventive Maintenance" (page 27).

### PURCHASE INFORMATION

Product Name: \_\_\_\_\_ Serial#: \_\_\_\_\_  
\_\_\_\_\_

Date of Purchase: \_\_\_\_\_ Date Installed: \_\_\_\_\_  
\_\_\_\_\_

Purchased From: \_\_\_\_\_  
\_\_\_\_\_

Address: \_\_\_\_\_  
\_\_\_\_\_

City: \_\_\_\_\_ Post Code: \_\_\_\_\_

Telephone No: \_\_\_\_\_

Comments:

# Service Record History

Complete this section after each service, repair inspection and/or maintenance. *Photocopy additional pages as required.*

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic Inspection <input type="checkbox"/> Monthly Inspection <input type="checkbox"/> 6 Month Inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly Inspection <input type="checkbox"/> Other: _____			
Completed By: _____		_____	
Printed Name		Signature	
Company: _____			
Remarks & Action Taken:			

## Display Symbols

Symbol	Definition	Symbol	Definition
	Caution risk of danger or Attention, consult accompanying Documents		General Warning Sign
	Manufacturer as defined in EU directive 93/42/EEC		Refer to Instructions, Manual, Booklet
	Date of Manufacturing		Model/Catalogue Number
	Product Serial Number		

## Warranty

This Warranty does not affect or in any way limit your Statutory Rights.

Handicare guarantees all equipment, which includes Ceiling Lifts, Floor Lifts, Slings, Service Parts, and Track and accessories, supplied as new, against failure within the period of 1 year from date of installation or 18 months from date of manufacturing, whichever is shorter, by virtue of defects in material or workmanship.

Handicare guarantees all refurbished equipment supplied against failure within a period of three months from date of installation or six months from date of purchase whichever is shorter.

This guarantee does not apply to failure attributable to normal wear and tear, damage by natural forces, user neglect or misuse or to deliberate destruction, or to batteries more than 90 days after original purchase.

This guarantee shall be void if the equipment is not serviced by Handicare or its authorized service agents in accordance with the manufacturer's recommendations or if any unauthorized person carries out works on the equipment.

The liability of Handicare under the terms of this guarantee shall be limited to the replacement of defective parts and in no event shall Handicare incur liability for any consequential or unforeseeable losses.

Always make sure you have the correct version of the manual. The most recent version is available on our website, [www.handicare.com](http://www.handicare.com).

For questions about the manufacture or operation of this equipment, contact your local Handicare dealer or your local Handicare business development manager.

### About Handicare

Handicare offers solutions and support to increase the independence of the physically challenged, mobility impaired or elderly people and to enable them to live an active life—on their terms—and to facilitate for their care providers and family. The offering encompasses a comprehensive range of curved and straight stairlifts, transfer, lifting and repositioning aids, vehicle adaptations and medical equipment. Handicare is a global company with sales in more than 20 countries and is a market leader in this field. The head office is in Stockholm, Sweden and manufacturing is located at six sites distributed across North America, Asia and Europe.

### Always make sure you have the correct version of the manual

The most recent version of all manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

For questions about the product and its use, contact your local Handicare dealer or your local Handicare business development manager.

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**Handicare USA, Inc.**