DisposableBelt

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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REF

6054

SWL: 100 kg/220 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Danske	9-10
Suomi	11-12
Deutsch	13-14
Netherlands	15-16
Français	17-18
Italiano	19-20
Español	21-22
<hr/>	
Product information	23

The DisposableBelt is intended for use as an accessory for extra support during various types of transfers and must not be used as an assistive device for lifting. The DisposableBelt is made of a very durable non-woven material. It must be discarded when soiled or when no longer required.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the user manual

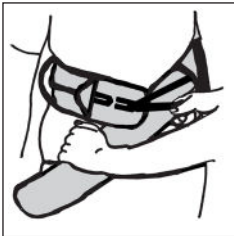
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Placement

Apply the belt around the user, below the waist.

Select the appropriate size loop. Fasten the belt and tighten it.

Removing

Release the velcro and then remove the belt.

Using of DisposableBelt

1

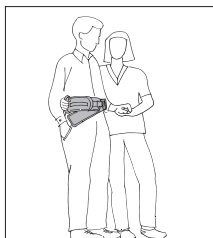
2

Caregivers can work in pairs or alone, depending on the user's needs, since DisposableBelt have several vertical and horizontal handles. Cross-grips at the back provide support and enable a good working stance (1 and 2).

It is important to lean the user forward to get a natural pattern of movement, working in towards the body with DisposableBelt.

3

4

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the user by the hand at the front (3).

DisposableBelt provide good support during transfers between wheelchair and bed (4).

Material

Polypropylene

Care of the product

Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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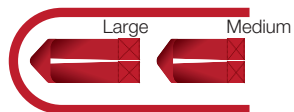
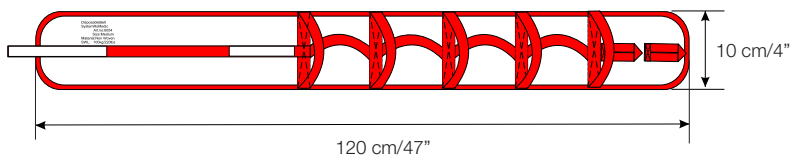
Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com