# EasySlide

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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1400 x 900 mm/ 55.1" x 35.4"  
1420

2000 x 900 mm/ 78.7" x 35.4"  
1470

**REF**

1420

1470

**SWL: 180 kg/400 lbs**

## Table of contents

<b>English</b> .....	3-4
<b>Svenska</b> .....	5-6
<b>Norsk</b> .....	7-8
<b>Danske</b> .....	9-10
<b>Suomi</b> .....	11-12
<b>Deutsch</b> .....	13-14
<b>Netherlands</b> .....	15-16
<b>Français</b> .....	17-18
<b>Italiano</b> .....	19-20
<b>Español</b> .....	21-22
<hr/>	
<b>Accessories</b> .....	23

EasySlide is used for turning in bed. Since this model of EasySlide has clasps that allow it to be secured to the bed, the bed's backrest can be raised without the user sliding down. The product can be bedded into the bed.


## Check Safety

---

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Friction test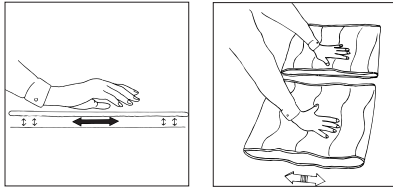

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

## Always read the user manual

---

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).


## Placement of EasySlide

The EasySlide is placed with the locking clasps facing upward and towards the head of the bed. Place a sheet over the EasySlide and fold it in under the mattress on both sides of the bed for transfers to/from the bed and for repositioning.

## Using EasySlide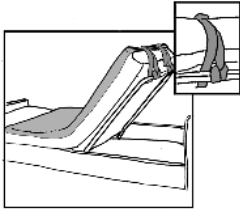

Tighten the straps before the bed backrest is raised. Loosen the straps when the backrest is lowered.

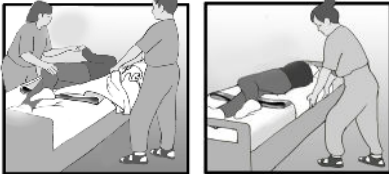Turn the user by pulling on the bed sheet that covers the EasySlide. When turning is complete, tuck the sheet in on both sides.

## Note

Be aware that there is a risk of the user sliding out of bed. Always raise the bed side rails when the user is unattended.

## Material

Polyester, microfibre, polyamide

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Vida útil estimada

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## Accessories

Art. no	Product	Description
1451	Protective cover	For EasySlide 1420
1481	Protective cover	For EasySlide 1470
1599	LockingStrap	Spare part

1430	Bundle	EasySlide 1420 + Protective cover 1451
1480	Bundle	EasySlide 1470 + Protective cover 1481

## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

### ***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)