





Making everyday life easier

QuickTips


Eva™

EVA450- 450lb/205kg weight capacity

EVA600- 600lb/273kg weight capacity

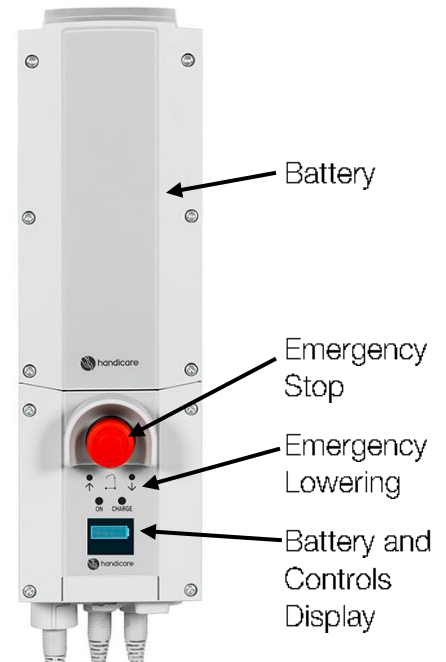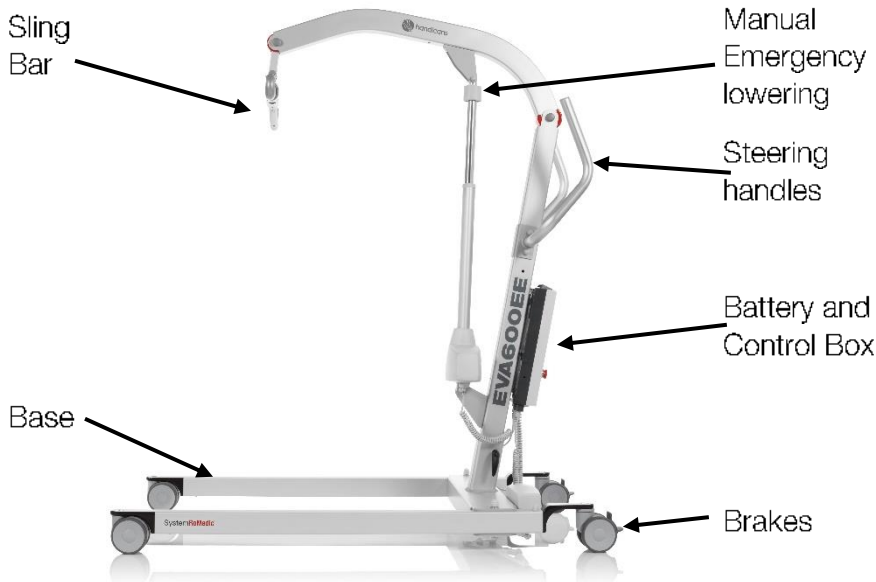

Hand Control

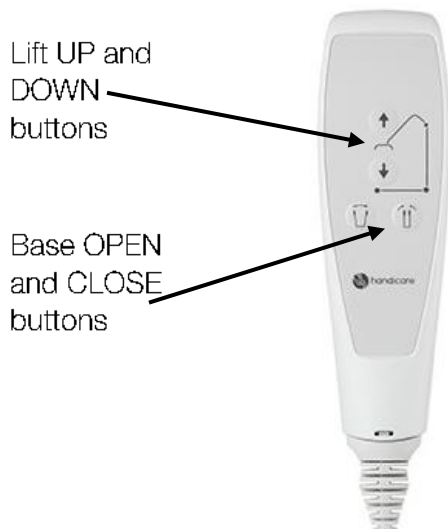

Troubleshooting

If the lift is not working as expected, follow these troubleshooting tips:

1. Ensure lift is unplugged to operate.
2. Ensure red emergency stop is disengaged (twist to disengage).
3. Ensure battery is properly secured and "clicked-in"
4. Check battery charge.
5. Secure hand control plug beneath control box, check and secure all plugs below control box.