# ReTurnBelt

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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ReturnBelt

6031 6035  
6032 6036  
6033 6037  
6034 6038

ReturnBelt  
Disposable

6233  
6234  
6235  
6236

ReturnBelt  
Wipeable

6133  
6134  
6135  
6136

**SWL: 205 kg/450 lbs**

## Table of contents

<b>English</b> .....	3-5
<b>Svenska</b> .....	6-8
<b>Norsk</b> .....	9-11
<b>Dansk</b> .....	12-14
<b>Soumi</b> .....	15-17
<b>Deutsch</b> .....	18-20
<b>Netherlands</b> .....	21-23
<b>Français</b> .....	24-26
<b>Italiano</b> .....	27-29
<b>Español</b> .....	30-32
<hr/>	
<b>Product information</b> .....	33

ReTurnBelt is used for raising from a seated to a standing position, and for support in the standing position in combination with the use of ReTurn transfer platform. ReTurnBelt has a more rigid lower section for easy application and an elastic upper section for a better and more comfortable fit. ReTurnBelt has several gripping alternatives and a locking strap for extra support and safety in connection with the use of the ReTurn platform.


## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.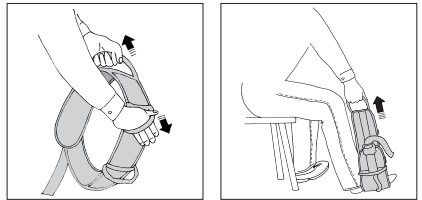

## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).


## Placement of ReTurnBelt

Fit the ReTurnBelt to the user with the elastic section facing upward and the rigid, lower section downward, as shown in the illustration on the product label.

Lean the user forward or to one side and guide the lower section of the belt down towards the 'sit bones' (lowest bony prominences or ischial tuberosities).


Secure the belt with the Velcro straps. The entire length of the front Velcro strap must be secured to the soft Velcro strap.

The locking strap can be temporarily folded under the edge of the belt, so that it is accessible when the user has risen to a standing position. The locking strap is then looped around the ReTurn for added security while the user is standing.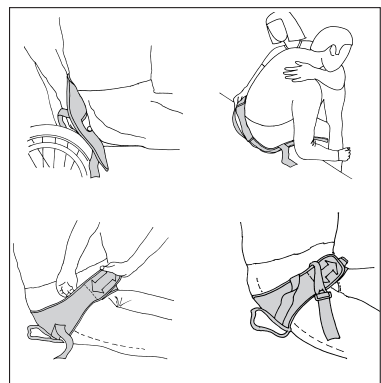

## Using the ReTurnBelt


### Two caregivers with ReTurnBelt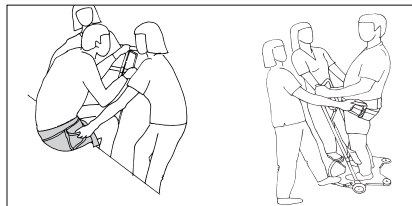

Brake the wheels on the ReTurn. The user then leans forward and grasps the handles on the ReTurn. The caregivers then grasp the handles on the ReTurnBelt.

ReTurnBelt allows different gripping alternatives, and helps and encourages the user to lean forward. One caregiver helps the user to lean forward and provides support during raising.

### One caregiver with ReTurnBelt

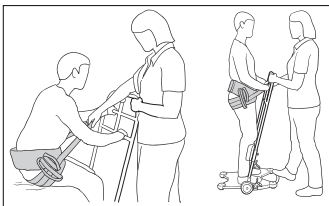
Fasten the clasp without tensioning the strap. A click is heard when the clasp is secured.

Brake the wheels on ReTurn.

The caregiver encourages the user to lean forward while providing support during raising by grasping either a handle or the strap.

The user leans forward and grasps the handles on ReTurn.

### When it is necessary to attach ReTurnBelt to ReTurn7500
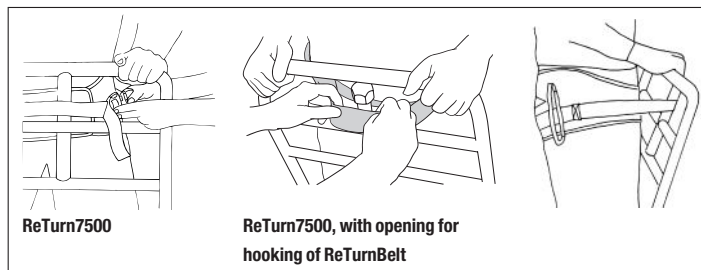
#### ReTurn7500:

Do not secure the clasp on the ReTurnBelt before rising. When the user is in a standing position facing the ReTurn, the strap can be placed around one of the vertical handles. Secure the clasp on the ReTurnBelt. A click is heard when the clasp is secured.


#### ReTurn7500, with opening on the upper vertical handle:

Fasten the clasp on the ReTurnBelt without tensioning the strap. A click is heard when the clasp is secured.

When the user is in a standing position facing the ReTurn, the strap is passed through the opening in the upper vertical handle and placed against the middle support bar.

Now, transfer the user with ReTurn. Plan carefully to ensure that the transfer is as short as possible.

### Sitting down

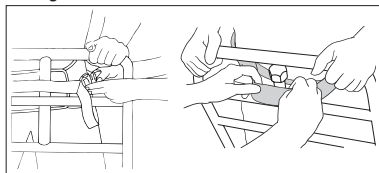Brake the wheels on the ReTurn. Ensure that the user is standing in a secure position facing the ReTurn, so that the locking strap is loosened. Undo the clasp and remove the strap from the ReTurn.

Fold it under the edge of the belt, so that it does not catch on anything. Help the user into a seated position with the help of the ReTurnBelt.

## Using ReTurnBelt without ReTurn

The ReTurnBelt can also be used to advantage even without ReTurn, for example, for gait training. Make sure you use the right size and secure the both the Velcro straps and the clasp.

## Material

---

Polyamid, nylon

Disposable (non-woven): Polypropylene, hydrophobic

Wipeable: Nylon + TPU, possible to use disinfectant.

## Care of the product polamid/nylon

---

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

If disinfection is decided, 70% ethanol, 45% isopropanol or similar should be used.

Fasten the Velcro straps and clasp before laundering. Clean the Velcro straps to remove dust and dirt, so that they retain their adhesiveness.

## Care of the product, disposable

---

Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

## Care of the product, wipeable

---

Read the product label.

Use ethanol (alcohol) to clean the cover. Concentration must not exceed one part ethanol to three parts water. Similar surface disinfectants may be used.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

### ***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)