




QuickTips

RoWalker™

400lb Weight Capacity

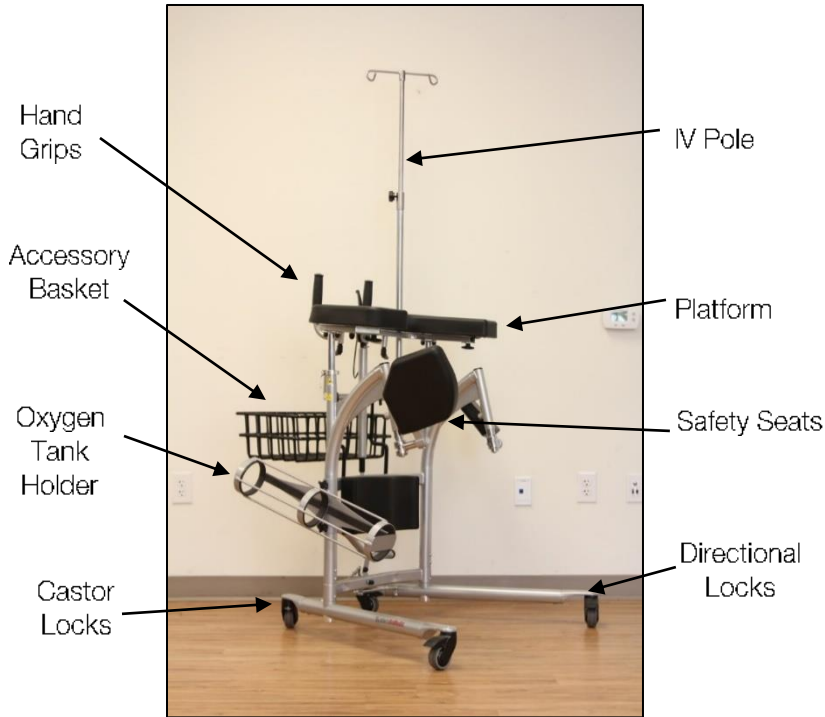

Release the hydraulic latch to allow platform adjustment. Close latch once platform is in desired position



Squeeze grip handle to raise or lower platform to desired height



Loosen lever under hand grips to adjust for patient needs (i.e. contractures)



Twist knob below platform to widen or narrow platform width as desired



Apply directional locks on both sides to keep device moving in straight path, apply directional lock to one side to correct a gait drift.



Safety seats may be opened and closed two ways. Lift and rotate laterally or flip the seats up vertically. Use preferred method