SupportStraps

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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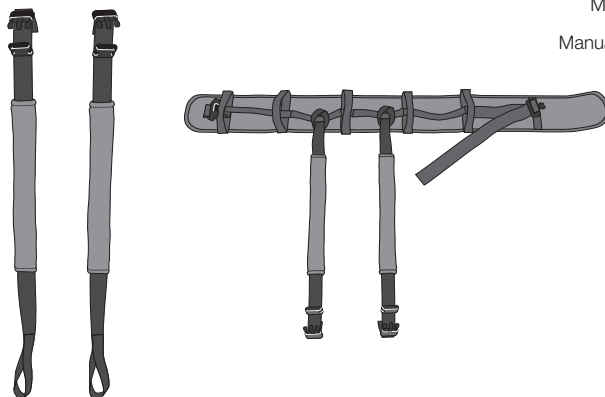
REF

6061

SWL: 150 kg/330 lbs
(2x 75 kg/163 lbs)

Table of contents

English	3-5
Svenska	6-8
Norsk	9-11
Dansk	12-14
Suomi	15-17
Deutsch	18-20
Nederlands	21-23
Français	24-26
Italiano	27-29
Español	30-32
Product information	33

SupportStraps have been developed as an accessory for use with EasyBelt, FlexiBelt, ReadyBelt, ReTurnBelt and FlexiMove. There are three different application areas for SupportStraps:

- Leg harness for EasyBelt/FlexiBelt/ReadyBelt
- Torso/side support when using QuickMove together with EasyBelt/FlexiBelt/ReadyBelt or FlexiMove
- Side support when using ReTurn together with ReTurnBelt.

SupportStraps consist of two separate straps, each with a loop on one end and a plastic clasp on the other end. They have a removable, padded case that relieves pressure when the strap is used as a leg harness.

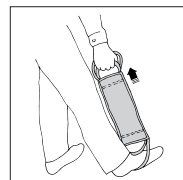
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

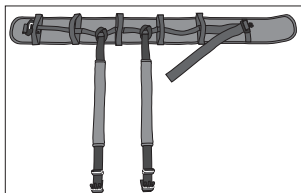
The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Before use

- Always read and follow the instructions in the user manual for the product with which SupportStraps are to be used.
- Plan the lift and decide how many caregivers will be needed to assist to ensure safety.
- Always perform a trial fitting.
- Do not leave the user unattended during a lifting situation.



Use of SupportStraps as leg harness for EasyBelt/FlexiBelt/ReadyBelt

Read and follow the instruction in the user manual for the belt.

SupportStraps are secured to the appropriate handle at the back of the belt, so that the straps are behind the user's hips.

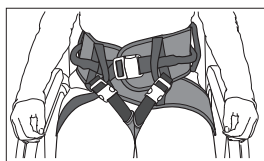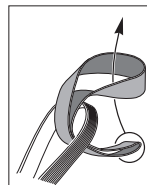
Secure each SupportStrap to the belt by drawing the strap through the hole and then through itself to form a loop.

Pull the straps between the user's legs and then secure the clasps to the appropriate handles on the front of the belt.

The straps now function as a leg harness and prevent the belt from sliding upward during transfer/gait training.

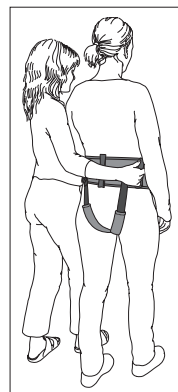Adjust the straps and the belt, so that the leg harness fits comfortably. The handles may be better placed if a different size is used.

Use as side support with ReTurn7500/7400

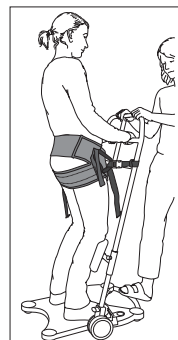
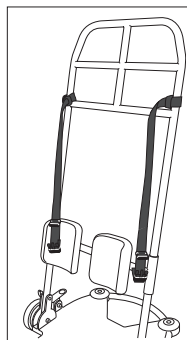
Follow the user manuals for ReTurn and ReTurnBelt.

Remove the padded cases from the SupportStraps.

Secure the SupportStraps to ReTurn. Loop the loops around the outer, vertical posts on the ladder.

Secure the SupportStraps to the appropriate handles on each side of the ReTurnBelt when the user has risen to a standing position.

Adjust the length of the straps. They should always be extended, with sufficient tension to ensure optimal support.

Use as torso/side support with QuickMove

Read and follow the instruction in the user manual for QuickMove.

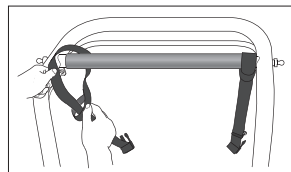
Remove the padded cases from the SupportStraps.

EasyBelt/FlexiBelt/ReadyBelt:

Fasten the SupportStraps around the handles on the QuickMove (recommended), alternatively on the appropriate handles on the sides/towards the back of the belt.

When the user has risen up in the QuickMove and is sitting on the seat, the straps are secured to the appropriate handles on the sides/towards the back of the belt, alternatively on the handles of the QuickMove.

Adjust the length of the straps. They should always be extended, with sufficient tension to ensure optimal support.


With EasyBelt/FlexiBelt


FlexiMove:

Secure the SupportStraps around the handles on QuickMove. Then, fasten one strap around the handle on FlexiMove. When the user has risen and is sitting on the seat, the FlexiMove is drawn around the user's back and the other strap is secured to the side of the FlexiMove.

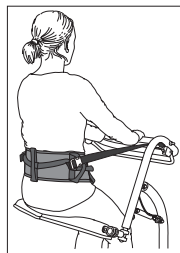Adjust the length of the straps. They should always be extended, with sufficient tension to ensure optimal support.

With EasyBelt/FlexiBelt

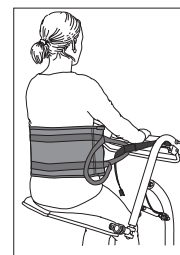With FlexiMove

Material

Polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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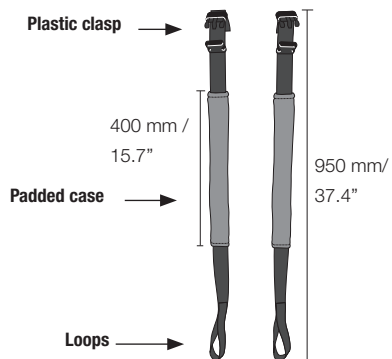
Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com