

SPAN PressureGuard
Easy Air[®] LR



Microclimate Management
with Alternating Pressure &
Lateral Rotation

Industry-best package
of performance and
multi-modality
treatment flexibility.

Proprietary design
delivers powerful,
proven impact on
excess moisture
and maceration.

Minimizes risk
associated with
transfers, edge-of-
bed/entrapment
issues, and power loss.

science, safety, simplicity


 savaria[®]

SPAN PressureGuard Easy Air[®] LR*


Packed with the design innovations that have made Span a recognized pioneer in high-performance microclimate management and pressure management systems.

Treatment Flexibility


Select one of three therapy modes:

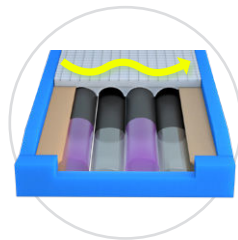

Alternating Pressure
Gently changes loading across the surface (15-minute cycle).

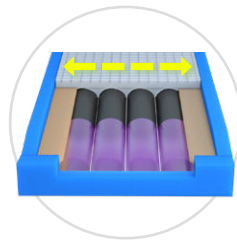

"Float" Mode
Provides active, powered flotation therapy.

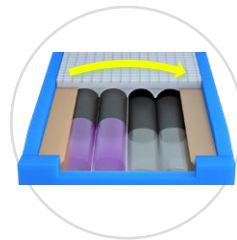


Lateral Rotation
A versatile modality indicated for a variety of needs (15-minute cycle).

PressureGuard Design

Integrated air system/engineered foam shell with "Safety Edge" bolster system delivers superior edge-of-bed safety, stability and comfort.

Indications

- Patients with excessive skin moisture due to sweating, decreased mobility, inability or refusal to be repositioned.
- Patients with macerated skin due to any of the above.
- Patients with increased skin or body temperature due to infection, sepsis, or other conditions.
- Patients with a combined score ≤ 5 on Braden Moisture, Activity and Mobility scales.
- Treatment of Stage 1-4 Pressure Injuries.
- Prevention of Pressure injuries for high-risk patients.
- Some patients with stabilized flaps/grafts (see Span algorithm for parameters).

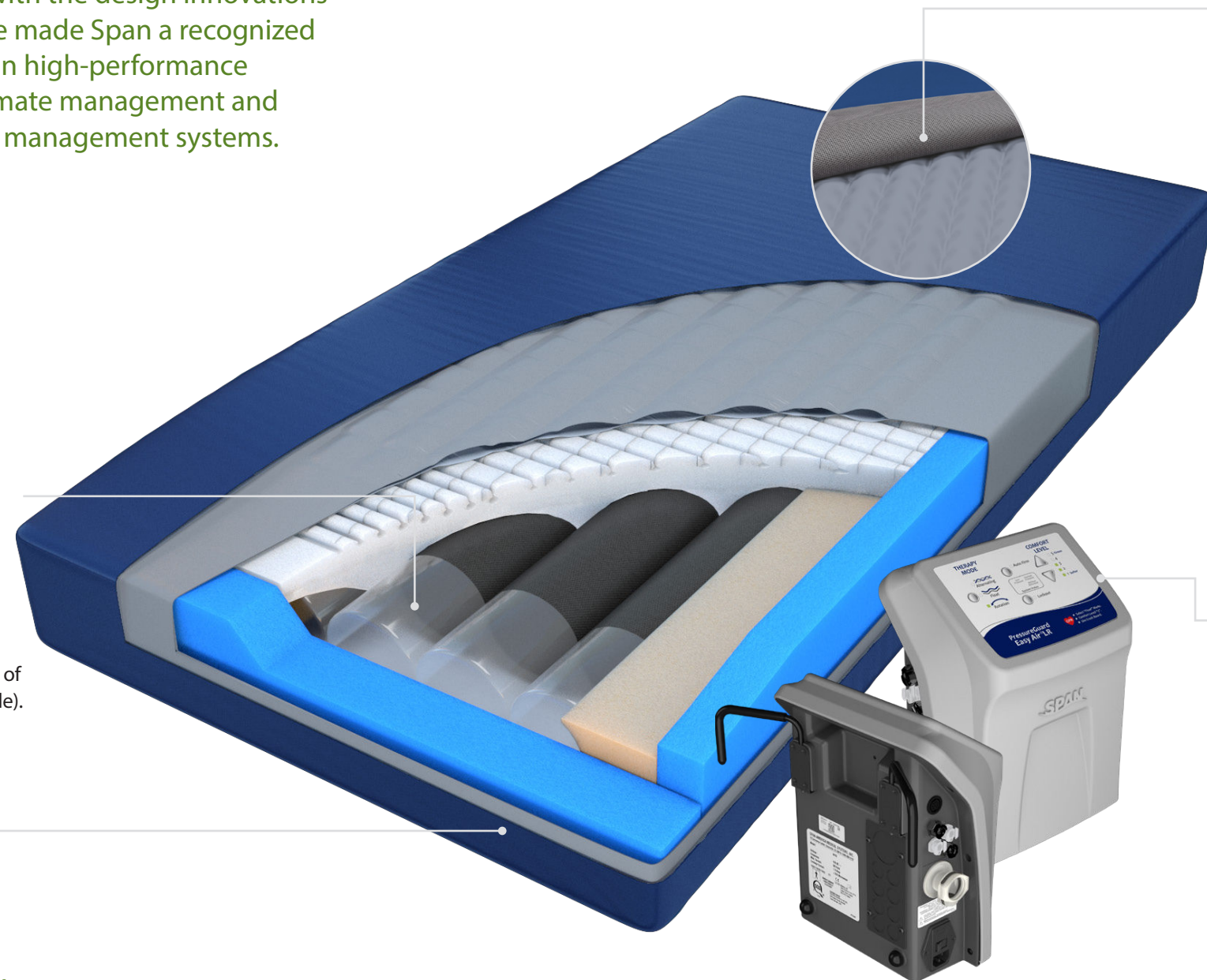


Proprietary Air Diffusion Matrix™

Three-dimensional liner helps ensure uninterrupted air flow in both cover and coverlet.

Control Unit Features

- Does not deflate during power interruption and transport.
- Timed (30-minute) Auto Firm mode for added stability during transfers and ADL.
- Simple comfort adjustment, including ultra-soft "Max Immersion" setting for users under 120 lbs.
- Lockout function eliminates accidental or unintentional changes.
- Normal/Low Pressure indicator.
- Streamlined, lightweight control unit.

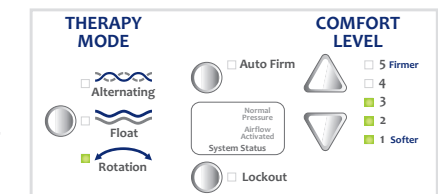


- Efficient: Uses less than 15 watts.
- Generates less heat, noise than traditional roll-up surfaces.
- Recessed, shielded connectors for protection from impact, damage.
- Detachable power cord for easy replacement.

Outer Coverlet

- Highly vapor-permeable
- Superior air flow
- Wipes clean in place with standard hospital-grade cleaners
- Zippered, machine-laundryable (air dry)

Inner Air Delivery Cover

- Dedicated air supply
- Directs constant stream of air
- Sweeps away moisture vapor before it can re-form as liquid
- Reduces maceration

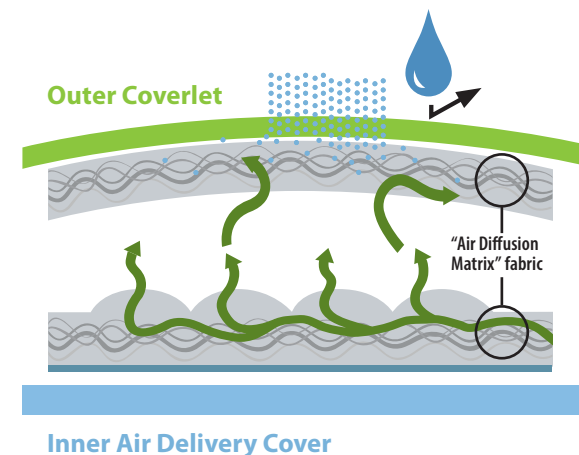

*Easy Air is also available without Lateral Rotation.

WEIGHT RANGE

500 lbs all models (350 lbs effective limit in Lateral Rotation mode). 39", 42" and 53" widths accommodate wider, non-bariatric beds for users up to 500 lbs. Bariatric (Easy Air XL) models also available for users up to 1000 lbs.

FLAMMABILITY

Conforms to NFPA 101 (Life Safety Code); Cal. TB 117; ASTM E1590; and 16 CFR Parts 1632 and 1633.

COMPLETE SYSTEM

Includes Mattress, Control Unit, and Coverlet

To order coverlet only, add "CLT-" to part number (example: CLT-L8035LR). Mattress Weight approx. 22 lbs. Two-year warranty (all components).

CONTROL UNIT

Dimensions	12" x 9.5" x 7"
Weight	6 lbs.
Leakage Current	< 250 micro-amps
Voltage	120 AC
Max Current	1.0 Amp
Frequency	60 Hertz
Conforms to:	ANSI/AAMI ES60601-1:2005 + C1:2009 + A2:2010 + A1:2012 CAN/CSA-C22.2 NO. 60601-1:2014; EN 60601-1:2006 + A1:2013; ANSI/AAMI HA60601-1-11:2015; CAN/CSA-C22.2 NO. 60601-1-11:2015; IEC 6060101-2:2014 (4th ed.); EN 60601-1-2:2014 (4th ed.)

ITEM CODE

DESCRIPTION

L7535LR-29	Easy Air LR System 75"L x 35"W x 7"H
L8035LR-29	Easy Air LR System 80"L x 35"W x 7"H
L8435LR-29	Easy Air LR System 84"L x 35"W x 7"H
L8039LR-29	Easy Air LR System 80"L x 39"W x 7"H
L8439LR-29	Easy Air LR System 84"L x 39"W x 7"H
92726	Easy Air LR System 80"L x 42"W x 7"H
42137	Easy Air LR System 84"L x 42"W x 7"H
L8053LR-29	Easy Air LR System 80"L x 53"W x 7"H
L8453LR-29	Easy Air LR System 84"L x 53"W x 7"H
L85VLR29	Easy Air LR for Recessed Deck Frame
8210	Easy Air LR Control Unit
L7535-29	Easy Air System 75"Lx 35"W x7"H
L7835-29	Easy Air System 78"Lx 35" W x7"H
L8035-29	Easy Air System 80"Lx 35" W x7"H
L8435-29	Easy Air System 84"Lx 35"W x7"H
[8039-29	Easy Air System 80"Lx 39" W x 7"H
6470029	Easy Air System 80"Lx 42"W x 7"H
81300	Easy Air System 80"Lx 42" W x 7"H
8010	Easy Air Control Unit



Get more information

800.888.6752 (Span)

savariapatientcare.com



Span-America

| 800.888.6752

| PO Box 5231 Greenville SC 29606

Span-America is a division of Savaria Corporation, a global leader in accessibility. Learn more about Savaria at savaria.com.

PL Span Easy Air LR CO# 2937 Rev.0