

# Instructions of Use

## High Easy Sling

### STANDARD SERIES

Reference: **SU-HYS-IOU**

Product Code/s: **M921515 - Poly Clips,**  
**M921513 - Mesh Loops, M921514 - Poly Loops**

Revision: **14.04.2021**


Copyright & Design Rights: **Silvalea.**



### WARNING

Do NOT use clip sling attachments on a loop type spreader bar and Do NOT use loop sling attachments on a stud type spreader bar.


#### CLIPS

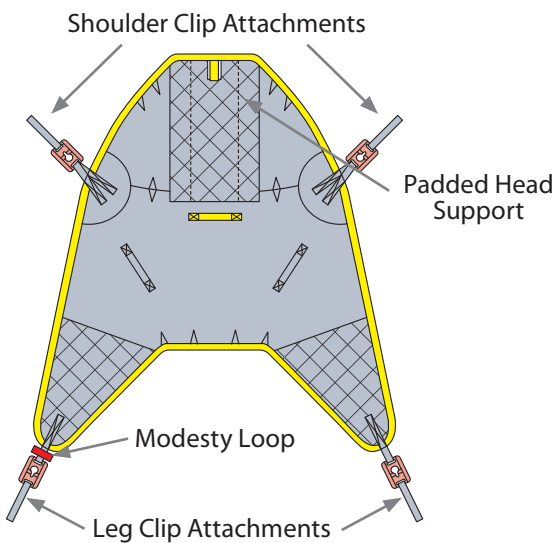


#### Clip Sling Attachments - Stud Type Spreader Bars:

Please note there are many variations of the stud type spreader bar but they should all be used exclusively by slings with:

**CLIP SLING ATTACHMENTS**

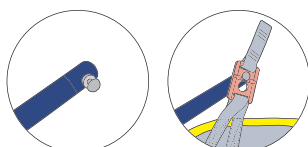


#### LOOPS

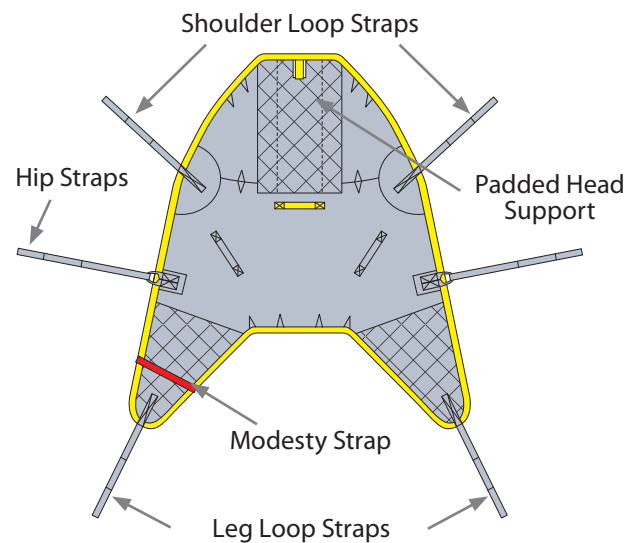


#### Loop Sling Attachments - Loop Type Spreader Bars:

Please note there are many variations of the loop type spreader bar but they should all be used exclusively by slings with:


**LOOP SLING ATTACHMENTS**

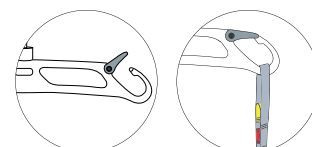


## Sling Application in Chair:

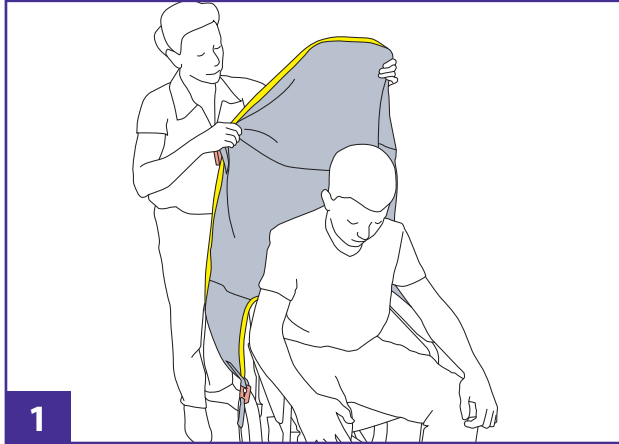

With sling label facing outwards glide sling down back of patient.

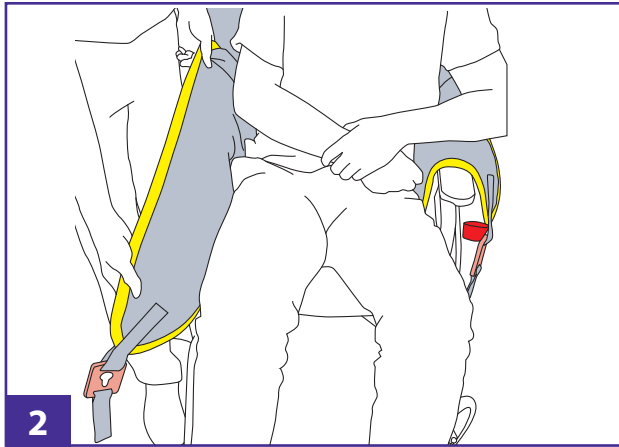


Take the leg sections of the sling and place them down each side of the patient.

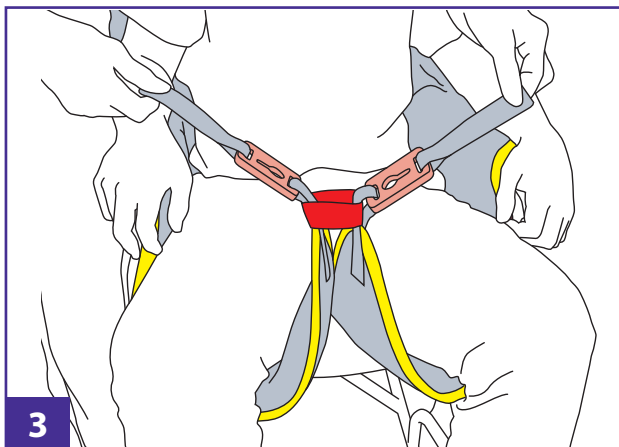


Glide each leg section under each thigh. Ensure that the leg sections are not twisted. Connect leg straps together by taking the right leg strap and thread through the modesty loop.


## Sling Application on Bed:

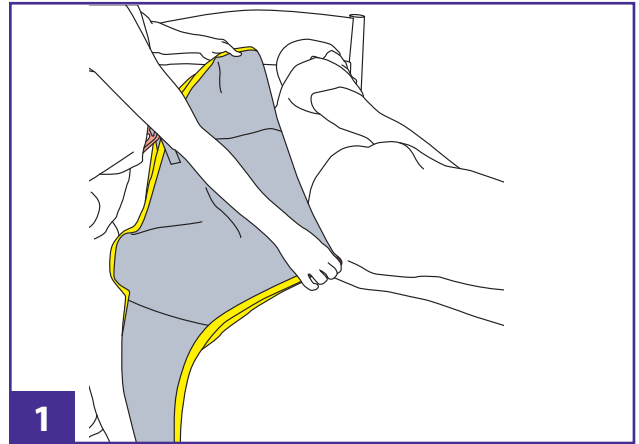

Fold the sling in half with the label inside the sling. Turn patient on their side and place folded edge of the sling against spine area

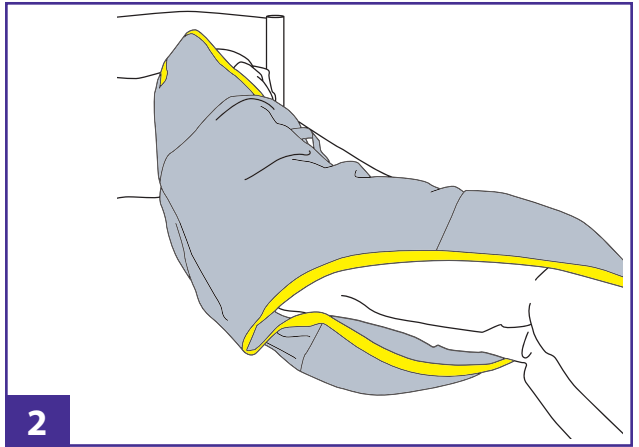


Open sling out and lay half over the patient then roll up the other half of the sling underneath against the spine area.

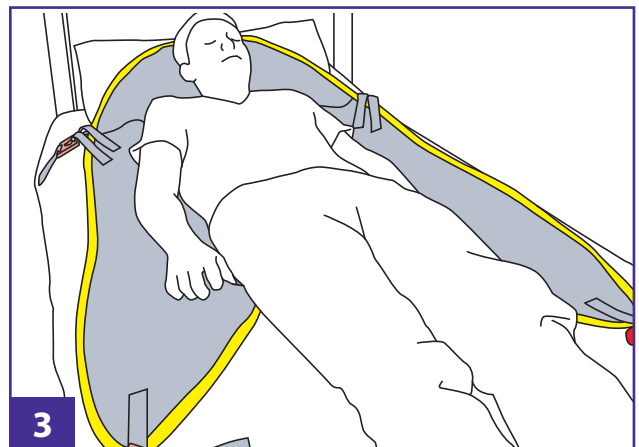


Roll patient onto the opposite side and pull through the underside of the sling. Lay patient flat on their back and position central to the sling.

## Attaching a Clip Sling: (stud spreader bar)

(stud spreader bar)

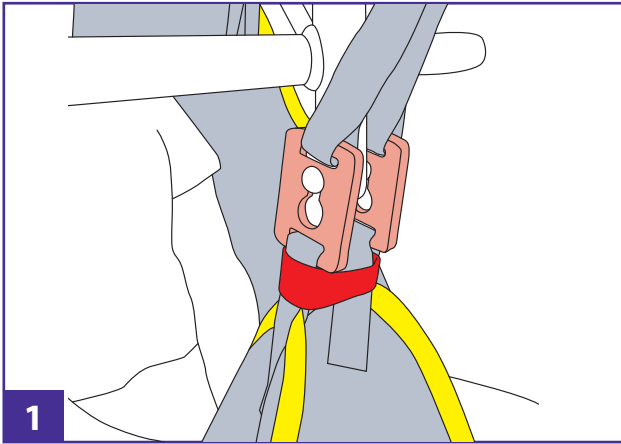

With the leg straps threaded through the modesty strap, attach clips to the lower stud on the spreader bar. When securing the clips there should be a click sound to confirm the clip is located correctly.

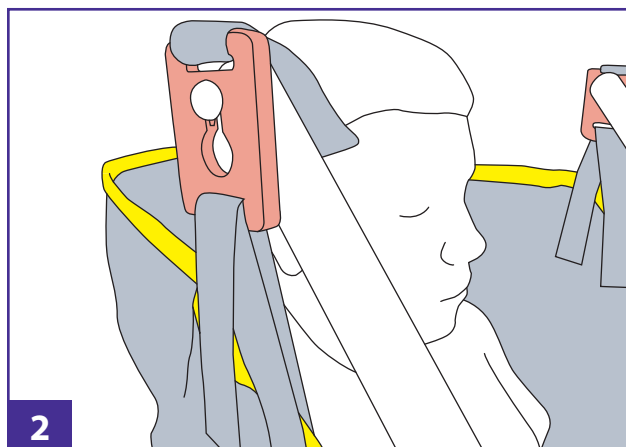


Fit the shoulder tape clips to the spreader bar. As before when securing the clips there should be a click sound to confirm the clip is located correctly. Once you are happy that the sling is fitted correctly you can begin the transfer.


**NOTE: Once the sling takes the weight of the patient you may need to re-adjust the position of the sling for the patients comfort. Lower the patient back onto the chair/bed to make any adjustments before continuing the transfer.**

## Attaching a Loop Sling: (loop spreader bar)

(loop spreader bar)

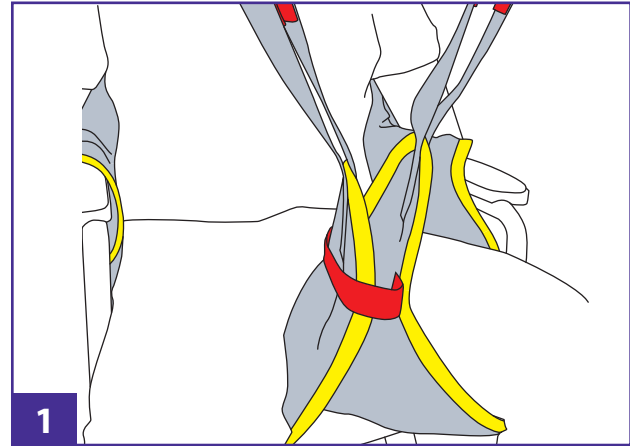

Ensure the leg sections are brought together by threading the leg paddle through the modesty strap. This gives a closed leg position.

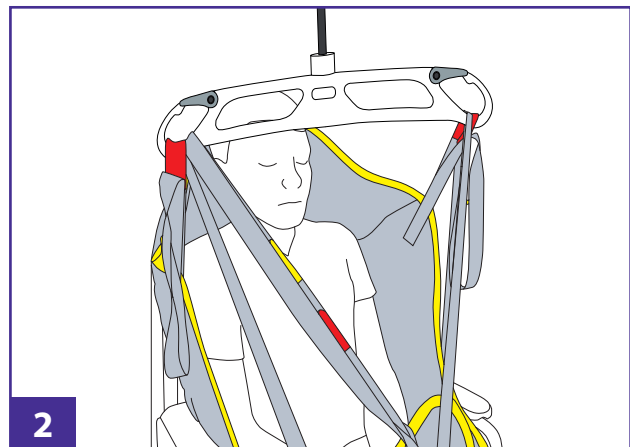

Once the sling has been fitted correctly to your patient, attach all loop straps (shoulder/hip/leg) to the spreader bar of the lift. Color coding on the pick up loops are provided for ease of reference and position. You can now begin the transfer.

**NOTE: Once the sling takes the weight of the patient you may need to re-adjust the position of the sling for the patients comfort. Lower the patient back onto the chair/bed to make any adjustments before continuing the transfer.**



## General Instructions:

**The High Easy Sling with loops & hip tapes is designed for a general purpose transfer from a recumbent or seated position.**

**The patient must be assessed by a competent person who is fully trained in the suitability, application and fitting of the sling.**

Carers should always be trained in the use of lifts and slings. Correct sizing of a sling is important, if in any doubt consult your Moving and Handling Advisor.

## Service Label & Sling Checks:

**All products should be checked each time they are used for any signs of deterioration.**

### *Please Check:*

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the Lift
- safe working load of both the lift and sling /product (if applicable)
- label is legible

**Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.**

The **Service Label** is a visual indicator of when the 6-monthly or annual lift inspection has been completed.

## Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- DO NOT IRON

## Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

## Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.



**SWL 272kg/600lb/42st**

For sizing guides and images of this product please visit [www.silvaleaslings.com](http://www.silvaleaslings.com)  
and for further assistance, contact your Silvalea distributor



**Silvalea Ltd** silvalea.com  
Silverhills Buildings. Decoy Ind. Estate, Newton Abbot, Devon, TQ12 5LZ. UK



**Distributed by Span-America**  
**Span-America; Span Medical Products Canada:** spanamerica.com



**Distributed by Savaria:** savaria.com / patientcarelifts.com



ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013. Malta

