

Instructions of Use

In-Situ Deluxe Leg

IN-SITU RANGE

Reference: **IS-DLS-IOU**

Product Code/s: **M921511**

Revision: **09.04.2021**

Copyright & Design Rights: **Silvalea.**


Silva-Superfine® PLUS is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark.



WARNING



Do **NOT** use clip sling attachments on a loop type spreader bar and Do **NOT** use loop sling attachments on a stud type spreader bar.

LOOPS

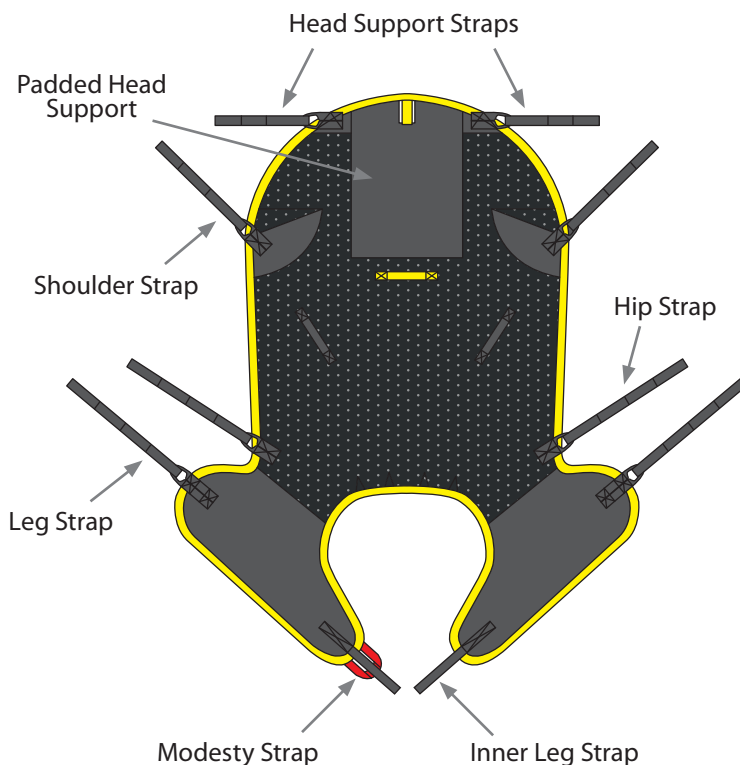


Loop Sling Attachments - Loop Type Spreader Bars:


Please note there are many variations of the loop type spreader bar but they should all be used exclusively by slings with: **LOOP SLING ATTACHMENTS**

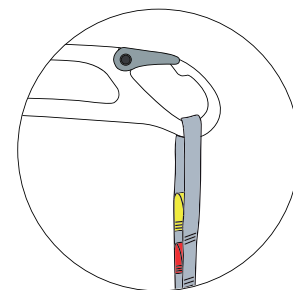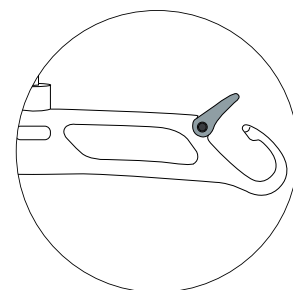


Sling Application:

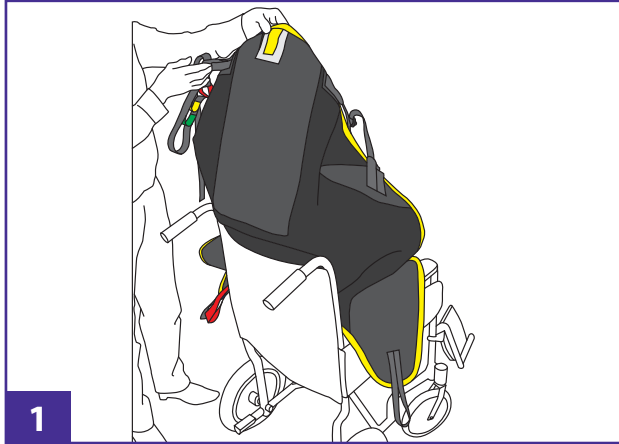


1 With sling label facing outwards glide sling down back of patient.

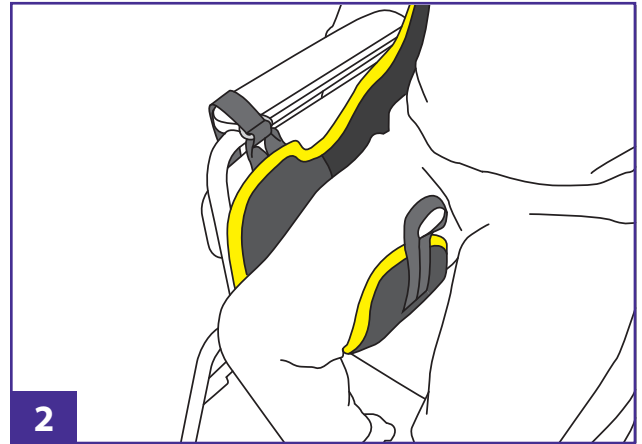


2 Place each leg section down the side and under each thigh. Check it is not twisted.

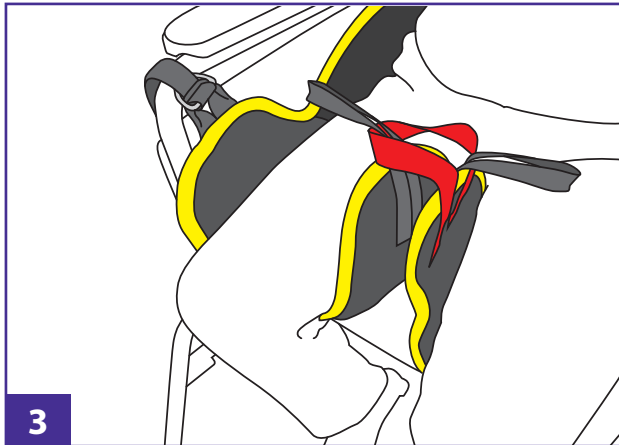

3 Thread the shorter inner leg strap through the colored modesty strap to prevent patient's legs abducting.

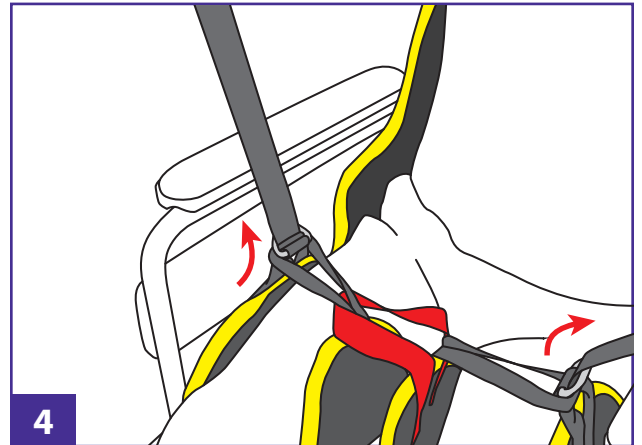


4 Thread the longer leg straps through the shorter inner leg straps as shown.

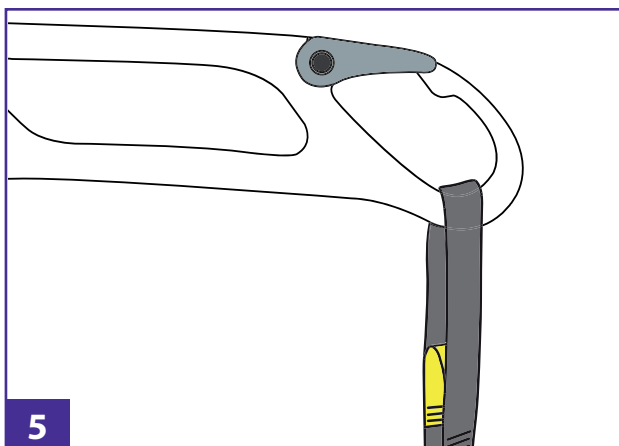

5 Once you are happy that the sling has been correctly fitted. Attach the two shoulder straps onto the spreader bar of the lift and then the two leg straps. Color coding on the pick up loops are provided for ease of reference and position.


Attaching the Head Support:

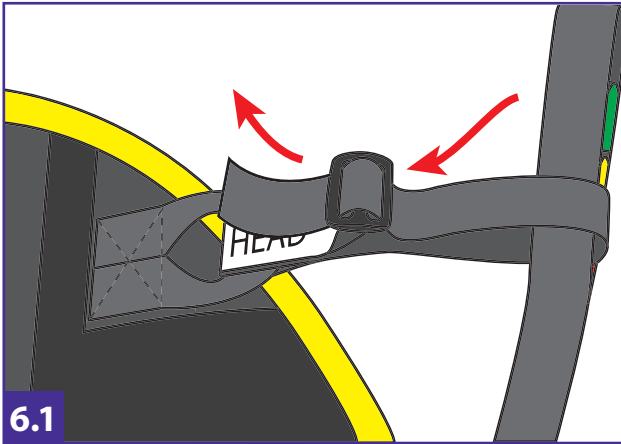

Either 6.1, feed the head support strap through the shoulder strap of the sling.


Then loop the head support strap back through the plastic slider correctly. Adjust the head support to suit patient's needs by adjusting the amount of strap passed through the slider.

Attaching the Head Support:

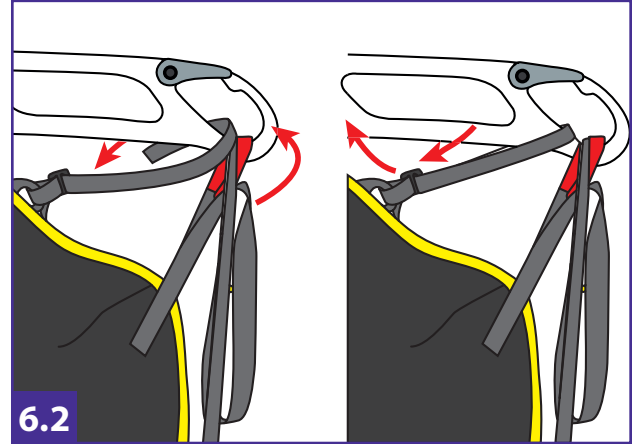


Or 6.2, attach head support strap directly to the lift spreader bar.

Then loop the head support strap back through the plastic slider correctly. Adjust the head support to suit patient's needs by adjusting the amount of strap passed through the slider.

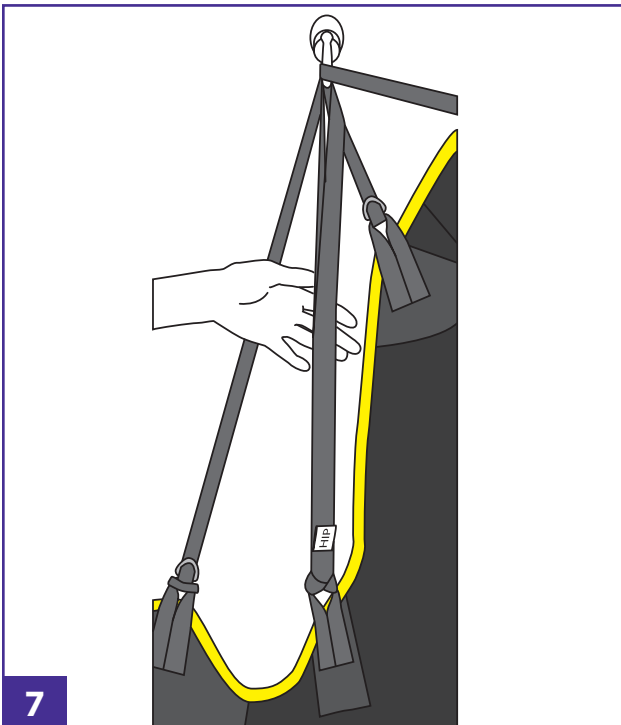

When attaching hip tapes, the ideal pick up should enable the pelvic area and the legs to pick-up simultaneously.

Adjustment can be made by using the color coded connections to enable the best pick-up.

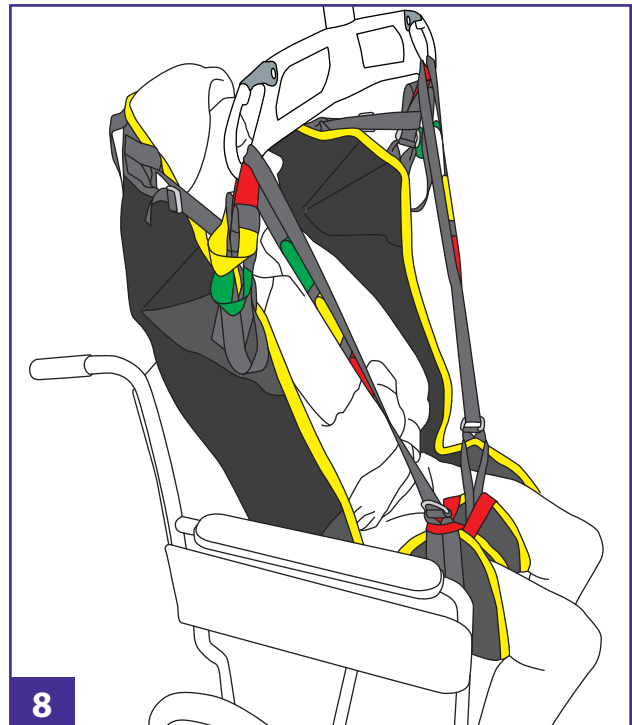

Ensure that all straps are not twisted and are attached securely before commencing the transfer.



General Instructions:

The In-Situ Deluxe Sling is designed for a general purpose transfer from a recumbent or seated position. The sling can remain in position under the patient subject to a risk assessment.

The patient must be assessed by a competent person who is fully trained in the suitability, application and fitting of the sling.

Carers should always be trained in the use of lifts and slings. Correct sizing of a sling is important, if in any doubt consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products should be checked each time they are used for any signs of deterioration.

Please Check:

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the Lift
- safe working load of both the lift and sling /product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.

The **Service Label** is a visual indicator of when the 6-monthly or annual lift inspection has been completed.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- DO NOT IRON

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.



SWL 272kg/600lb/42st

For sizing guides and images of this product please visit www.silvaleaslings.com
and for further assistance, contact your Silvalea distributor



Silvalea Ltd silvalea.com
Silverhills Buildings. Decoy Ind. Estate, Newton Abbot, Devon, TQ12 5LZ. UK



Distributed by Span-America
Span-America; Span Medical Products Canada: spanamerica.com



Distributed by Savaria: savaria.com / patientcarelifts.com



ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013. Malta

