



# QuickMove

Sit-to-Stand Aid for Patient Transfers

# Safe and Secure Sit-to-Stand Aid

QuickMove is a non-powered, lightweight sit-to-stand aid for patient transfers, early gait training, activity tolerance testing, bedside exercise and functional assessment. It supports and activates the user when rising up to a standing position and during transfers, either standing or sitting, from bed to chair, from chair to chair or to and from the toilet. QuickMove has been developed for users with impaired balance and strength in their legs, but the user must have some upper strength and be able to bear weight and participate in the transfer.

## How it works



QuickMove has an adjustable base width that makes it easy to get close enough and in the right position for the user.



The adjustable buttock cushions are angled to make room for the user and caregiver.



The user places his/her feet on the stable anti-slip foot plate, and gets support from the adjustable lower leg.



The user chooses a comfortable horizontal or vertical grip and rises using his/her own strength. When the user has risen using his/her own strength, the caregiver or the user folds down the cushions behind the buttocks, to provide stable support.



The user then has the option to remain standing or to be seated during the transfer. The support of the cushions behind the buttocks and the knee brace facilitates a stable and safe transfer.



When the user is rising again, the caregiver can, with a push on the handle, angle the cushions to assist achieving a standing position.

QuickMove is versatile enough to be used in either acute care or long-term care situations, including post-operative care when balance is temporarily impaired. It's all about moving users where they need to go with peace of mind while encouraging mobility.



## Accessories for Extra Support and Security

BackSupport provides extra stability and security for users during transfers with QuickMove, and can be used as torso / side support.

SupportStraps, together with EasyBelt and FlexiBelt, also can be used as torso / side support for QuickMove.

### BackSupport Accessory



### SupportStraps Accessory



# QuickMove Specifications

## Material

Powder coated steel

## Dimensions (L x W x H)

Length: 38.6 in (980 mm)

Width: 24.2 in (605 mm)

Height: 44.8 in (1120 mm)

Base width: 20.4–36.0 in (510–900 mm)

## Castor Size

3.0 in (75 mm)

## Weight

77 lb (35 kg)

## Safe Working Load

375 lb (170 kg)

## Part Numbers

**400801334:** QuickMove

**70200037:** BackSupport, accessory for QuickMove

**6061:** SupportStraps

CE



Are you ready to get started  
with QuickMove?

Call 1-888-637-8155 or visit [www.handicareusa.com](http://www.handicareusa.com)