QuickMove

SystemRoMedic™

CE

User manual – English

Bruksanvisning – Svenska

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
REF

400801334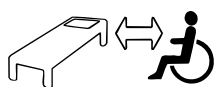

SWL: 170 kg/375 lbs

Table of contents

| | |
|--------------------------|-------|
| English | 4-6 |
| Svenska | 7-9 |
| Norsk | 10-12 |
| Dansk | 13-15 |
| Suomi | 16-18 |
| Deutsch | 19-21 |
| Netherlands | 22-24 |
| Français | 24-27 |
| Italiano | 28-30 |
| Español | 31-33 |

| | |
|------------------------------------|----|
| Technical information | 34 |
|------------------------------------|----|

QuickMove is an active transfer and transport aid, designed for users with poor balance and reduced lower limb strength. Users will need a degree of arm strength and standing function in order to actively participate in the transfer.

QuickMove is an ideal aid for users who can raise themselves to an upright, or semi-upright position to be transferred.

QuickMove can also be used as a turning device to transfer the user from bed to chair, wheelchair, toilet or similar.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Description of parts

1. Seat support
2. Depth adjustable kneepad
3. Angle adjustment of the seat
4. Manual adjustment of seat depth
5. Height adjustable kneepad
6. Footpedal for leg spread adjustment
7. 2 x rear braked castors
8. Detachable footplate

Functions

Split seat pads: both swing away and flip-up.

Knee pad: depth and height adjustable and tiltable.

Foot pedal: for adjusting leg spread in and out.

Angle adjustment crossbar: adjusts seat and handgrip angle up or down to optimise seating comfort and positioning of user.

Seat depth: two positions, manually adjustable.

Footplate: covered with non-slip surface, detachable.


Special requirements and conditions

- QuickMove is not intended for long transfers.
- QuickMove can only be used on flat floor.

To transfer with QuickMove, the user must be able to:

- Support weight while standing
- Stand up and maintain balance (with support)
- Grasp handles
- Understand instructions

Testing standing function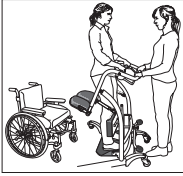

QuickMove is ideal for testing a user's standing function in cases where there is uncertainty as to whether the user is able to rise from a seated to a standing position.

Training standing function


QuickMove can be used to train the user's standing, sitting and sit-to-stand function. To measure improvement, use a stopwatch or clock with a second hand to time the user.

Farther back in the chair

If the user slides forward in the wheelchair, the QuickMove can be used to help the user to slide farther back in the chair. Remove/raise the wheelchair foot plates, and then run the QuickMove in towards the wheelchair. The user places his/her feet on the foot plate, grasps the raising ladder, leans forward, and then lifts from the seat. The caregiver provides counter pressure. Slide the user farther back in the chair when he/she has lifted slightly from the seat.

Using QuickMove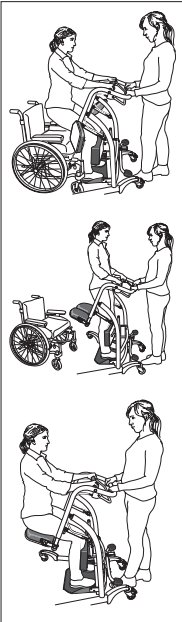

Place the QuickMove as close to the user as possible and ask the user to place his or her feet on the footplate. Adjust both the height and depth of the leg supports, as well as the seat depth, if necessary.

The user can then grasp the soft handlebar and rise to a standing position.


When the user has risen to a standing position, the caregiver lowers the seat behind the user.

The user will then be able to sit during transfer. For optimal sitting comfort, the caregiver can regulate the angle of the seat using the adjustment bar on the handlebar. This function also facilitates standing, since the sitting height is higher and the user's legs are straighter.

Accessories


BackSupport, art.nr. 70200037

BackSupport is a back support that provides torso/lateral support for QuickMove. The length is adjusted according to the height of user. The BackSupport is placed behind the user's back, and then fastened to the QuickMove.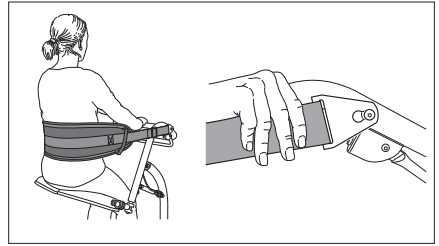

SupportStraps, art.nr. 6061

SupportStraps can together with EasyBelt/FlexiBelt/ReadyBelt or FlexiMove be used as torso/side support to QuickMove.

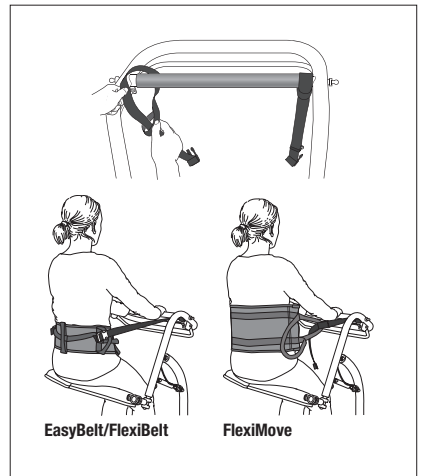Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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

Vida útil estimada

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Technical information

| | | |
|--|----------------|--------------|
| A Length | 900-1030 mm | 35.4 – 40.5" |
| B Width, inner. Min. leg spread. Castor to castor. | 455 mm | 17.9" |
| C Width, inner. Max. leg spread. Castor to castor. | 830 mm | 32.6" |
| D Width, outer. Min. leg spread. Castor to castor. | 615 mm | 24.2" |
| E Width, outer. Max. leg spread. Castor to castor. | 1050 mm | 41.3" |
| F Width between legs. Min. leg spread. | 480 mm | 18.8" |
| G Width between legs. Max. leg spread. | 620 mm | 24.4" |
| H Height adjustment: Floor to pushbar | 1065 – 1120 mm | 41.9 – 44" |
| I Height: Floor top of castor frame | 140 mm | 5.5" |
| J Height: Floor top of leg | 45 mm | 1.7" |
| K Seat height: Min. | 730 mm | 28.7" |
| L Seat height: Max. | 850 mm | 33.4" |
| Seat width: | 480 mm | 18.8" |
| Seat depth: | 265 mm | 10.4" |
| Castor Ø | Ø80 mm | Ø3" |
| Weight: | 35 kg | 77 lbs |
| Max. weight capacity: | 170 kg | 375 lbs |

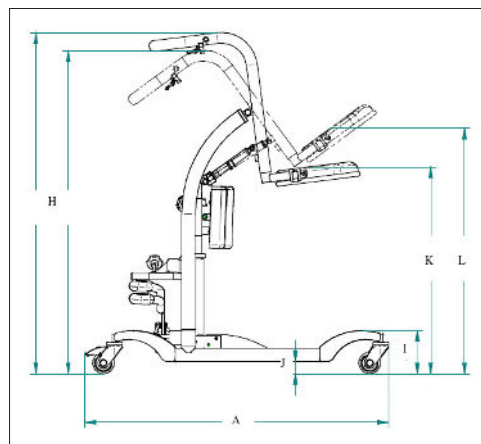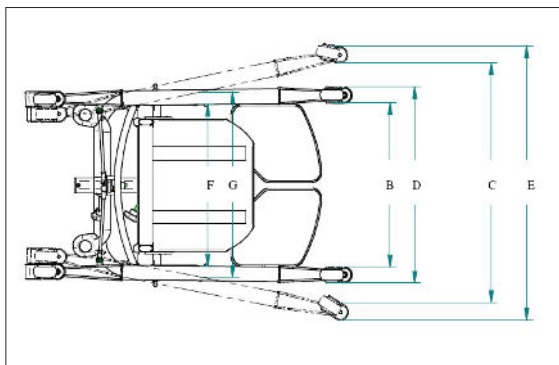Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com