



Instructions of Use

In-Situ High Easy Sling

REPLACEMENT SLING RANGE

Reference: **INSHE(C)-IOU**

Product Code: **M921530**


Revision: **07.11.2023**

Copyright & Design Rights: **Silvalea.**




WARNING


Do **NOT** use clip sling attachments on a loop type spreader bar and Do **NOT** use loop sling attachments on a stud type spreader bar.

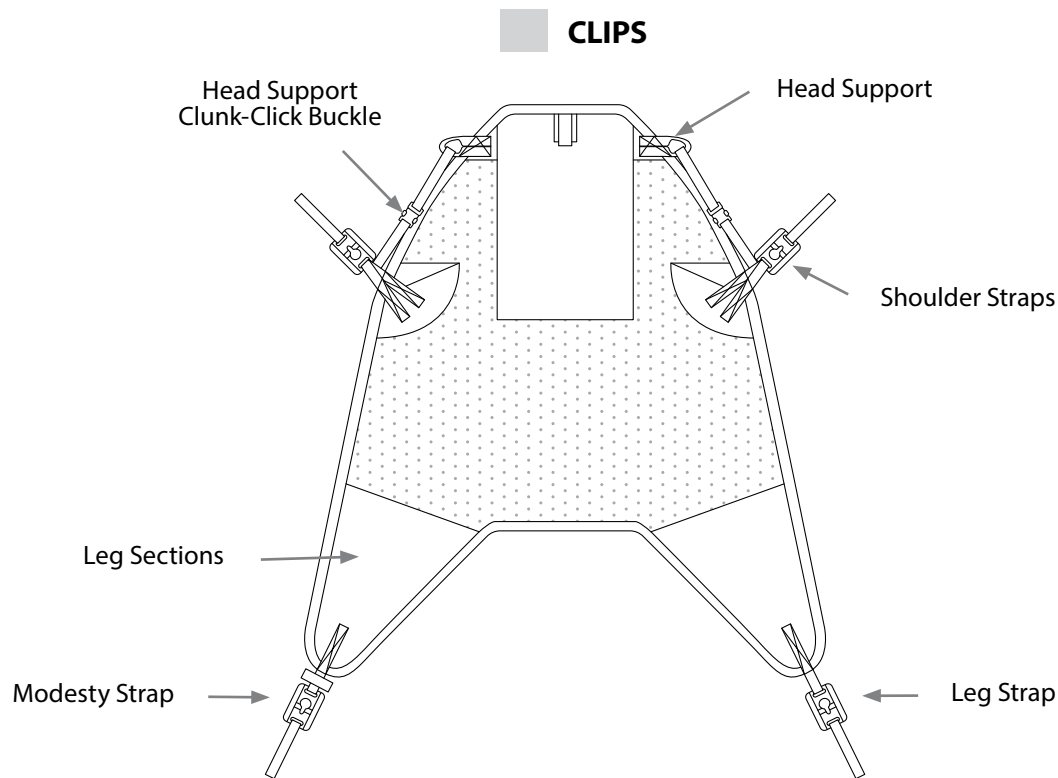


WARNING: Only use with suitable CLIP lift:

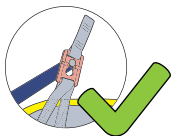


Sling Application in Chair:

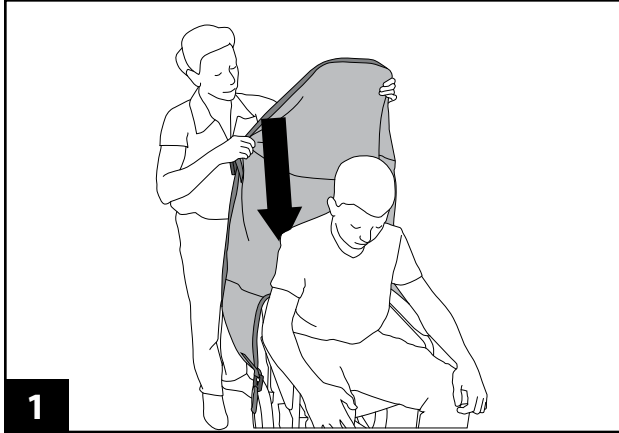


With sling label facing outwards glide sling down back of patient.

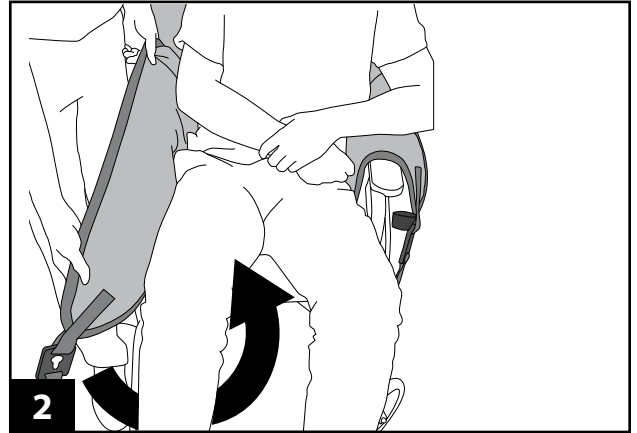


Take the leg sections of the sling and place them down each side of the patient.

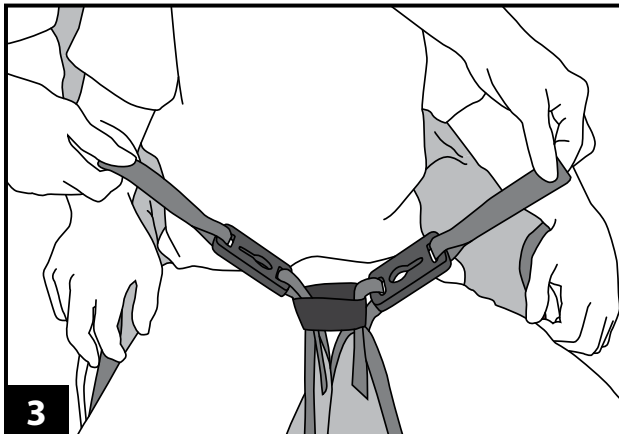


Glide each leg section under each thigh. Ensure that the leg sections are not twisted. Connect leg straps together by taking the right leg strap and thread through the modesty loop.

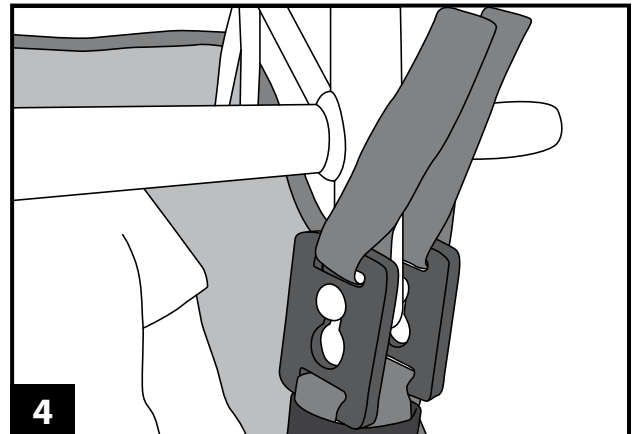

With the leg straps threaded through the modesty strap, attach clips to the lower stud on the spreader bar. When securing the clips there should be a "click" sound to confirm the clip is located correctly.

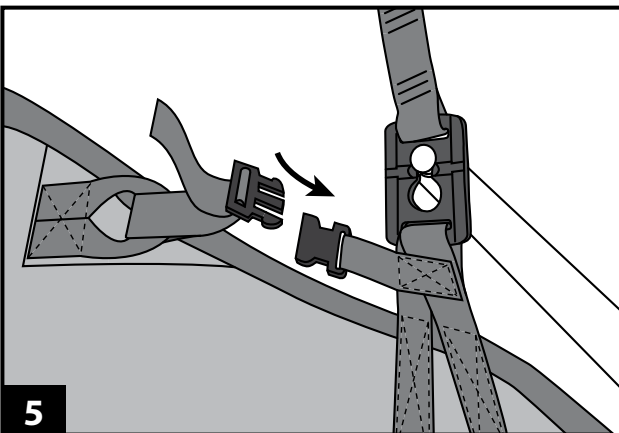


Once the shoulder straps are securely fitted, connect the buckle on the head support.

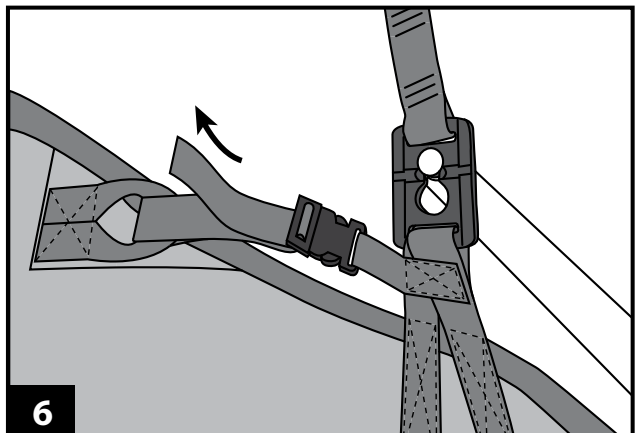


Adjust as required.


Sling Application on Bed:

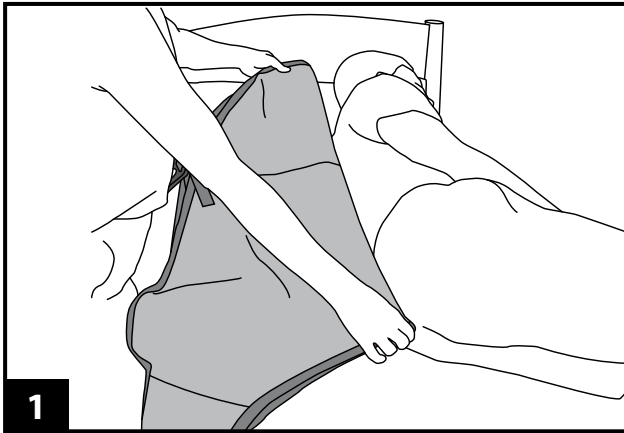


Fold the sling in half with the label inside the sling. Turn patient on their side and place folded edge of the sling against spine area

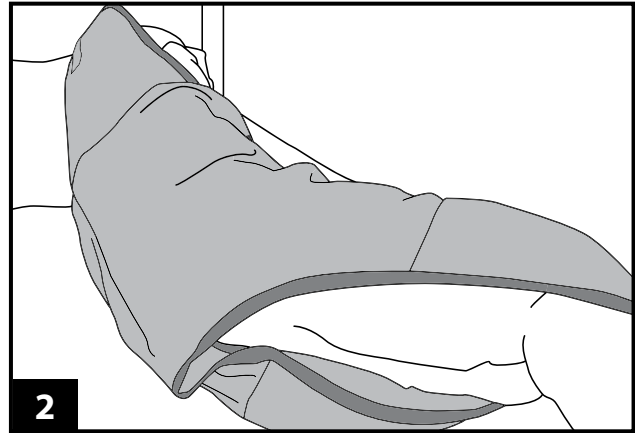


Open sling out and lay half over the patient then roll up the other half of the sling underneath against the spine area.

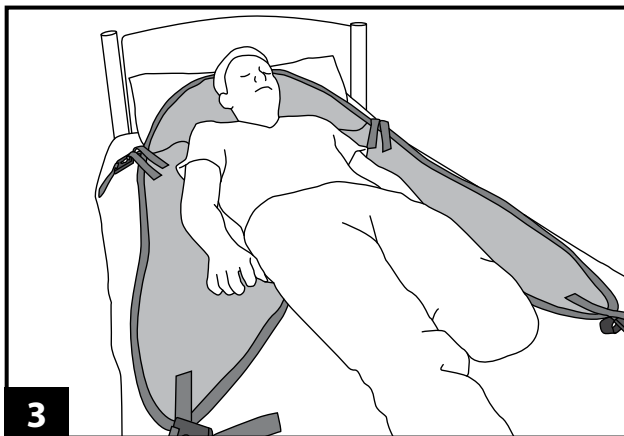


Roll patient onto the opposite side and pull through the underside of the sling. Lay patient flat on their back and position central to the sling.

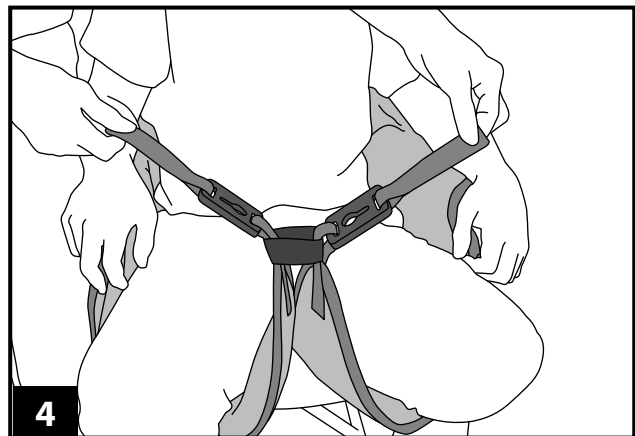

Glide each leg section under each thigh. Ensure that the leg sections are not twisted. Connect leg straps together by taking the right leg strap and thread through the modesty loop.

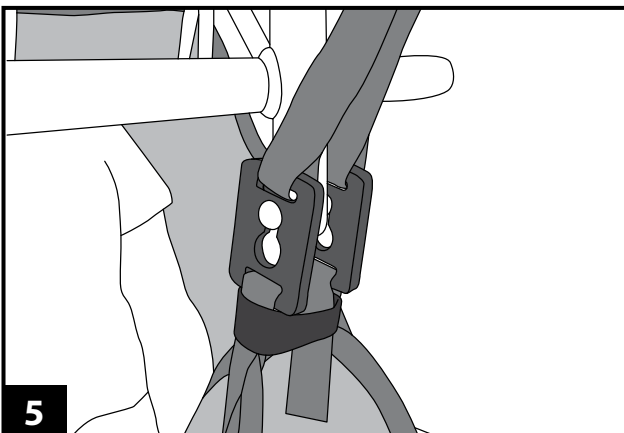

With the leg straps threaded through the modesty strap, attach clips to the lower stud on the spreader bar. When securing the clips there should be a "click" sound to confirm the clip is located correctly.

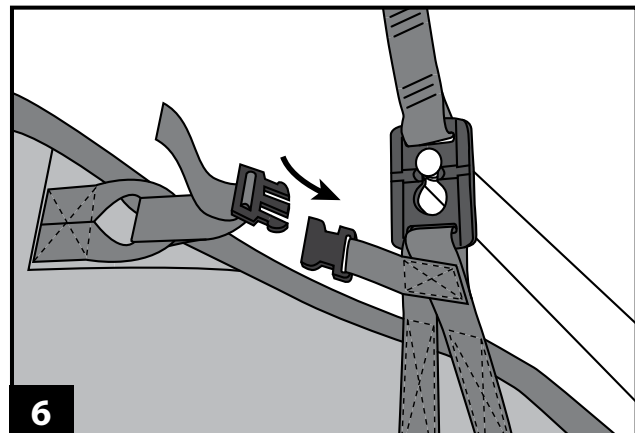

Once the shoulder straps are securely fitted, connect the buckle on the head support. Adjust as required.



General Instructions: WARNING

Do not attempt to transfer the patient until all appropriate sling attachments are secured to the lift spreader bar and you are satisfied that the sling is the correct style, size and fit for the patient.

Correct sizing of a sling is important as a sling that is too large could cause the patient to slip through the aperture.

While every attempt is made to instruct a safe move you must conform to the current moving and handling procedures/regulations. If in any doubt, consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products shall be checked each time they are used for any signs of deterioration.

The service label is a visual guide to the product's check. Please conduct a thorough sling check in line with current safety requirements in your country. If in doubt consult your distributor for current sling checking protocols.

A risk assessment must be carried out to ensure that the correct size and style of sling has been met before use.

Pre-use sling check list:

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the lift
- safe working load of both the lift and sling/product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label. Please note: temperatures advised are MAXIMUM washing temperatures.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products shall be laundered using non- biological detergents.
- DO NOT IRON

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label. Please note: temperatures advised are MAXIMUM drying temperatures.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.



SWL 272kg/600lbs/42st

For sizing guides and images of this product please visit www.silvaleaslings.com and for further assistance, contact your Silvalea distributor



Silvalea Ltd a division of **Savaria**

Silverhills Buildings, Decoy Ind. Estate,
Newton Abbot, Devon, TQ12 5LZ. UK



MADE IN UK

Distributed in North America by Savaria (savaria.com) and the following Savaria divisions:



handicare

handicare.ca | handicareusa.com



spanamerica.com | Span Medical Products Canada



ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta