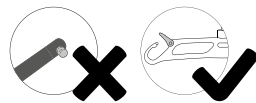
# Instructions of Use

## Specialised Sling Range

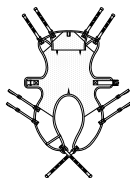
- |  |  |  |   |
|--|--|--|---|
|  Mesh      |  Parasilk                     |  <i>Silva-Superfine® with Coolmax®</i>              |  <i>Silva-Superfine® PLUS</i>                |
|  Polyester |  <i>Silva Quick -2-Apply®</i> |  <i>Silva-Superfine® with Coolmax® Double layer</i> |  <i>Silva-Superfine® PLUS Anti-microbial</i> |

*\*Not all fabrics available with all styles*


### TINKHAM SLING

 TINK/TINKHS


### STRETCHER SLING

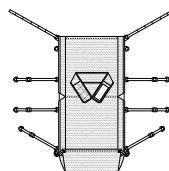 033


### FLOTATION SLING

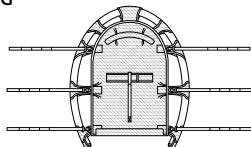 SFLOAT


### SILVA BALANCE

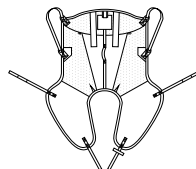 SILB


### CONTINENTAL EASY ACCESS

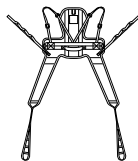 CCM1


### BIRTHING SLING

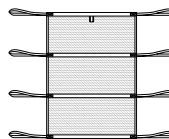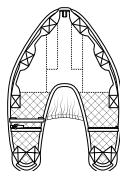 BIRTH1

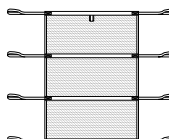
### MANUAL TRANSFER SLING




 062







### BIRTHING SLING

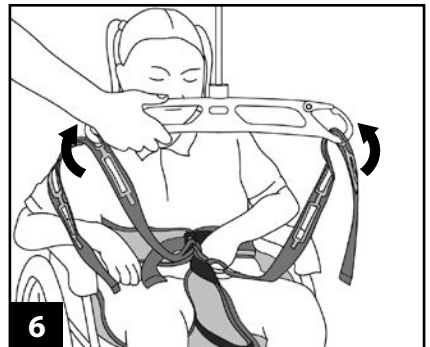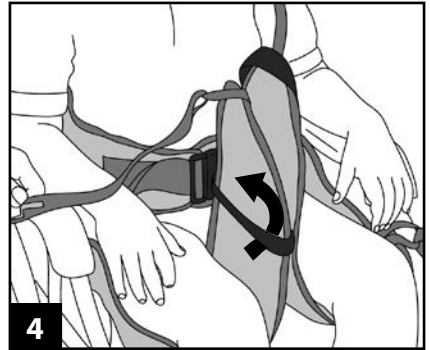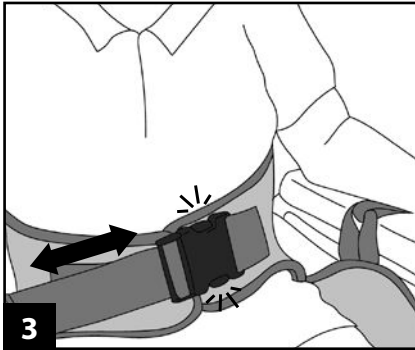 BIRTH2












# Tinkham Sling | TINK/TINKHS

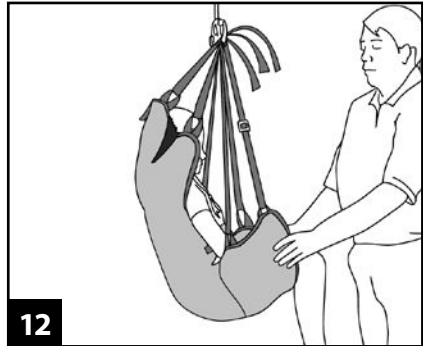
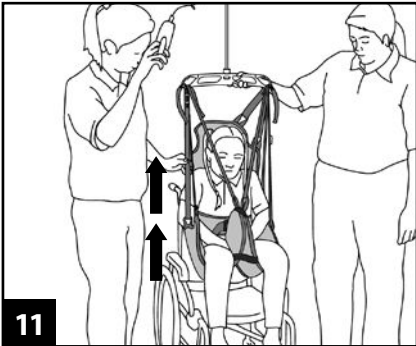
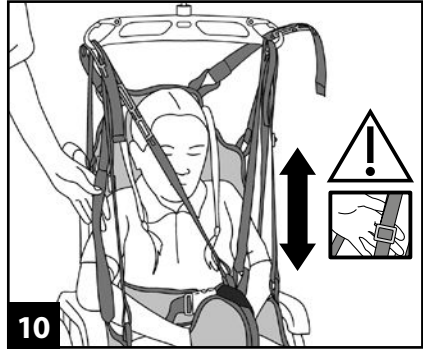
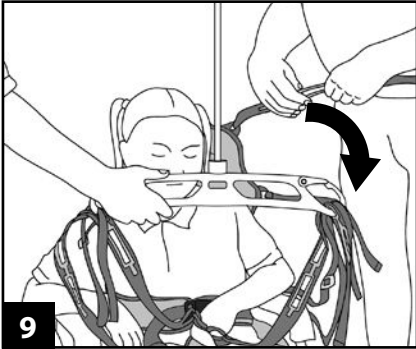
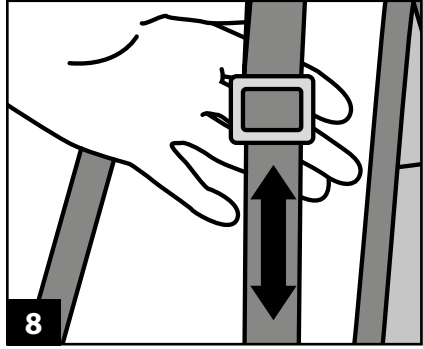
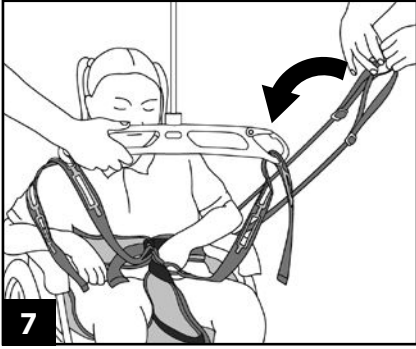
# Tinkham Sling | TINK/TINKHS

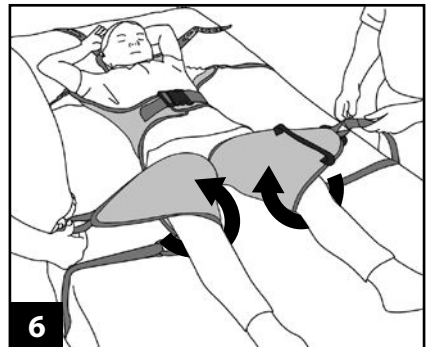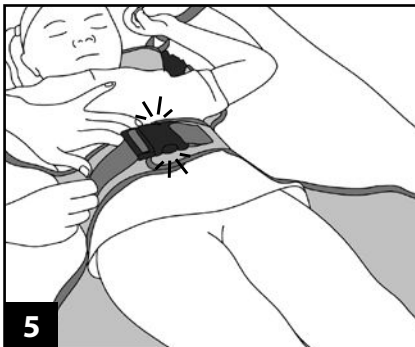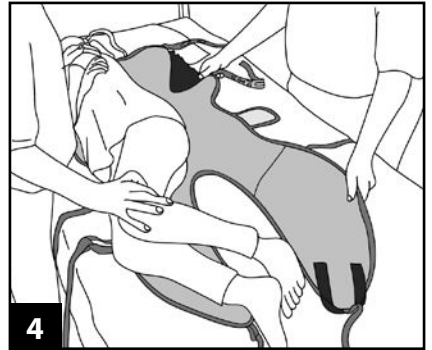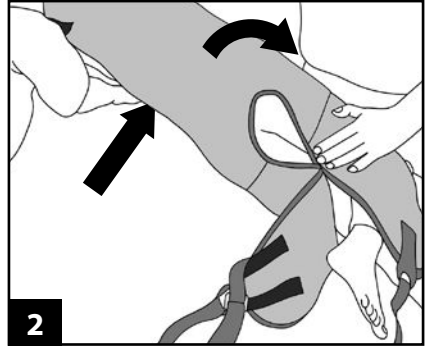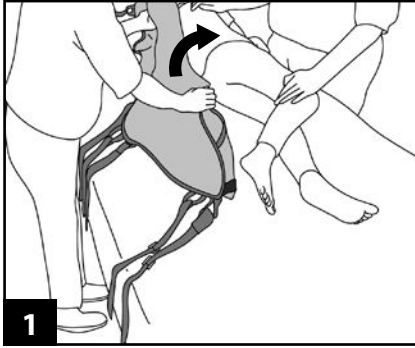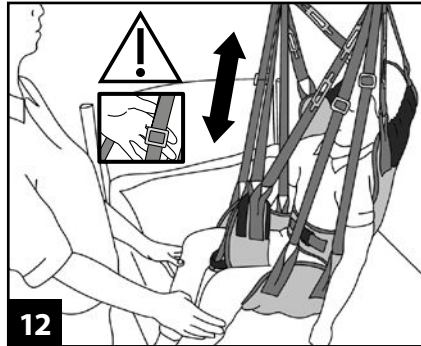
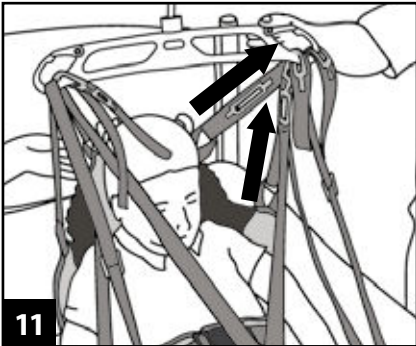
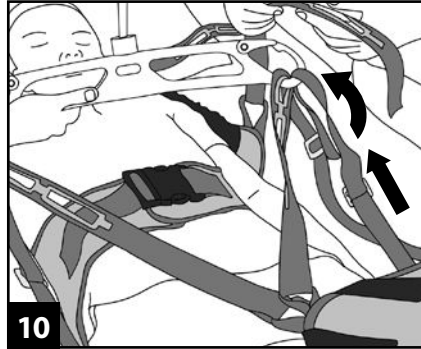
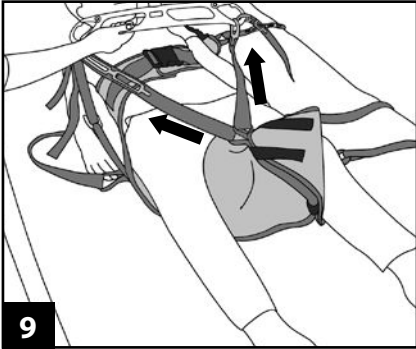
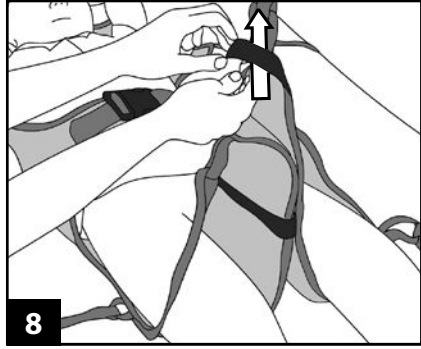
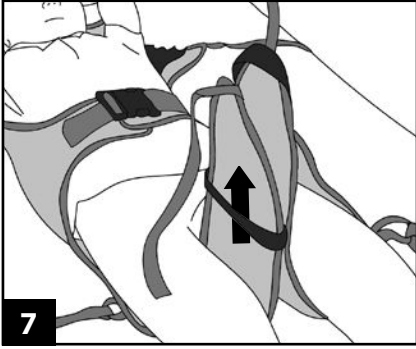
# Stretcher Sling | 033

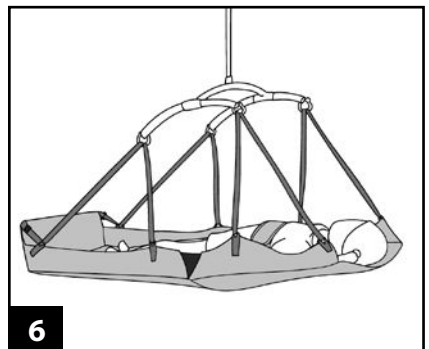
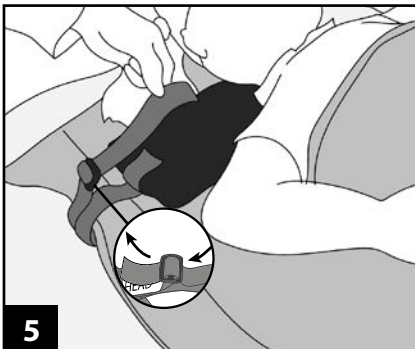
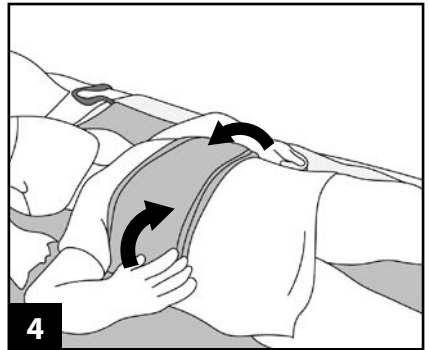
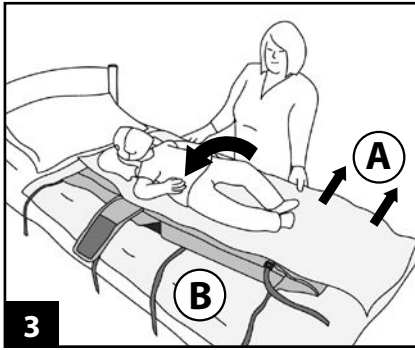
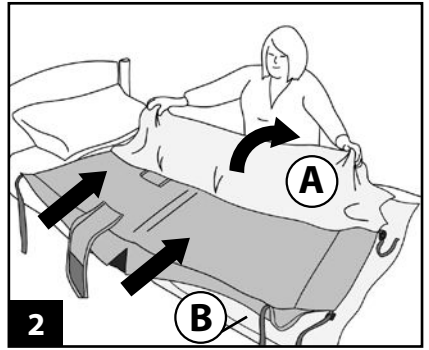
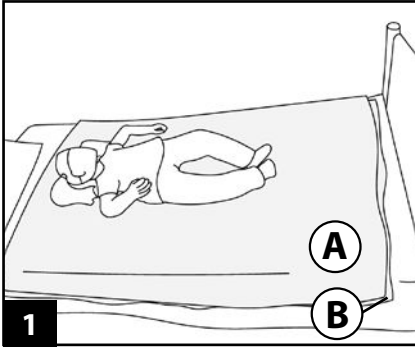
# Flotation Sling | SFLOAT

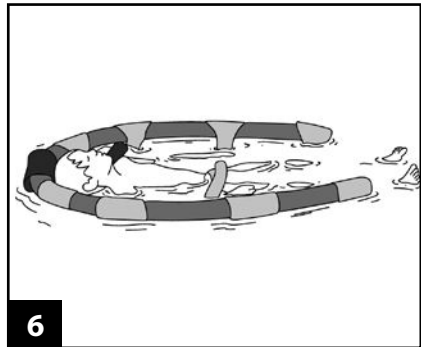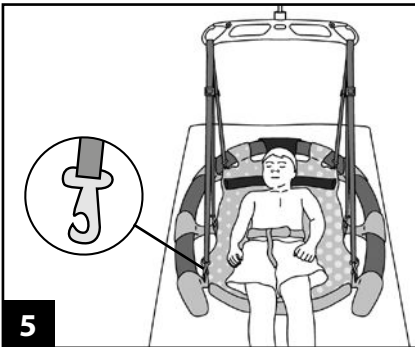
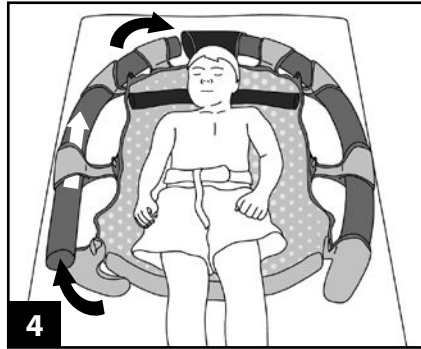
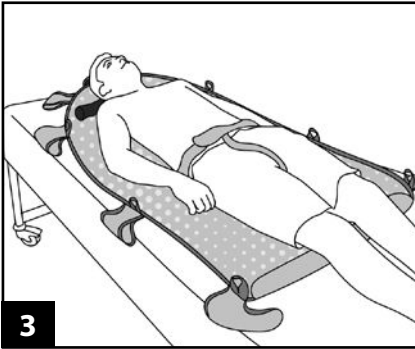
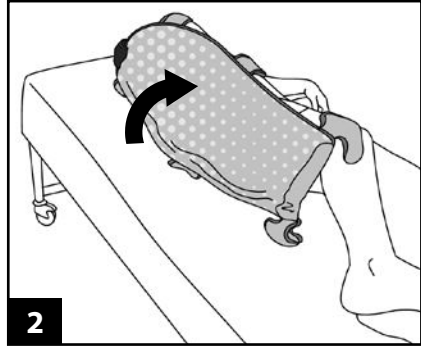
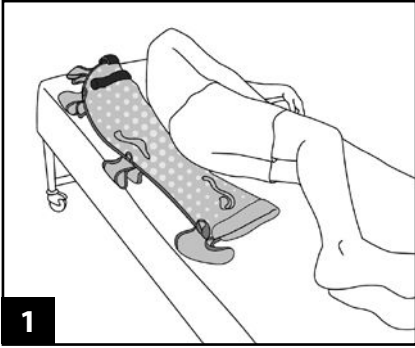
Silva Balance | SILB




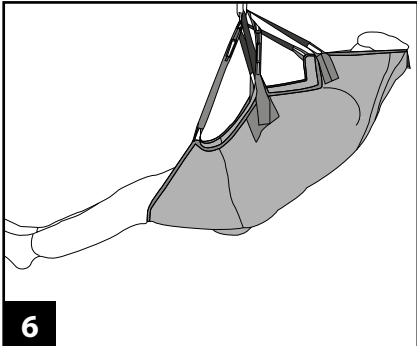
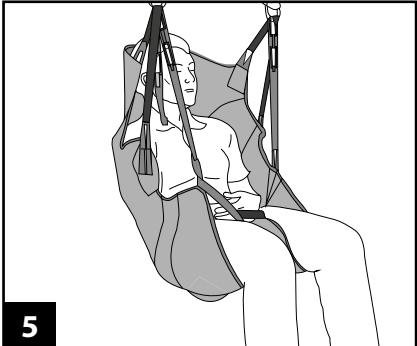
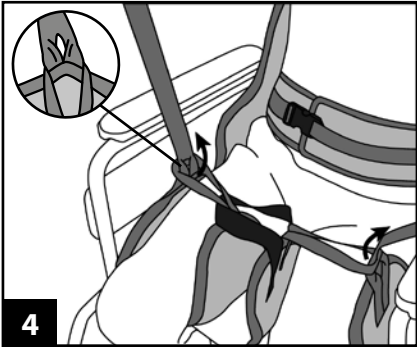
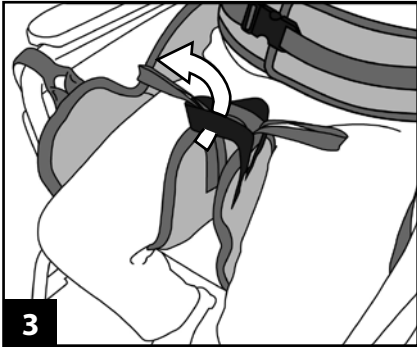
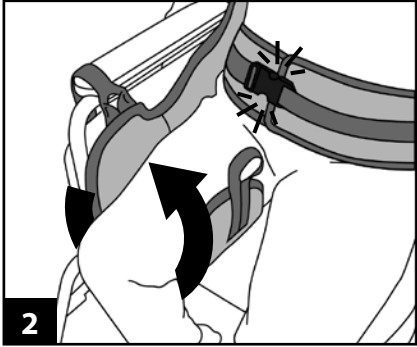
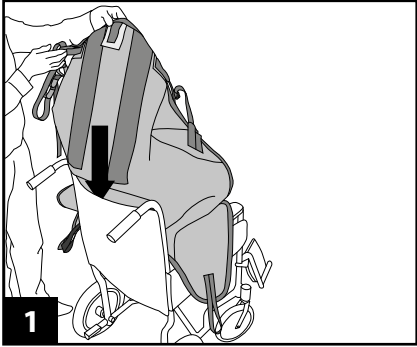
# Continental Easy Access Sling | CCM1







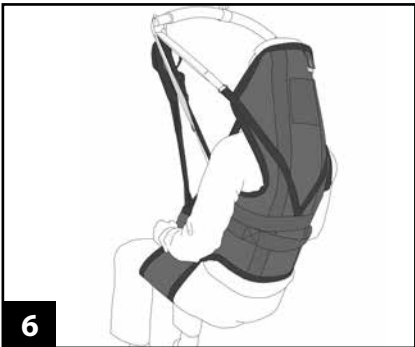
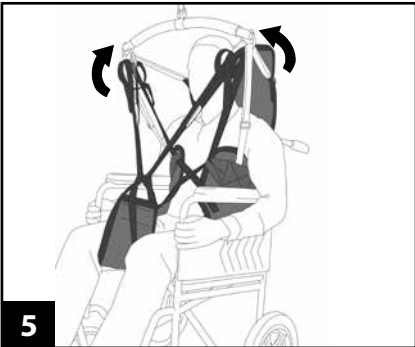
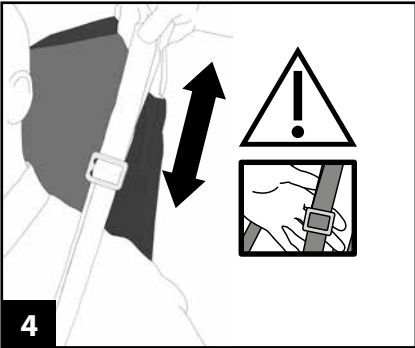
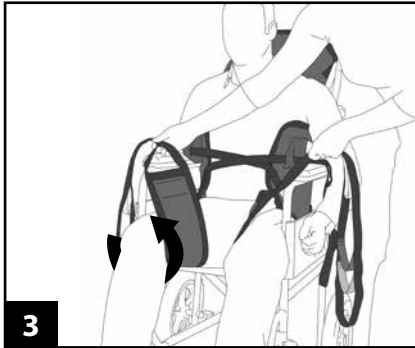
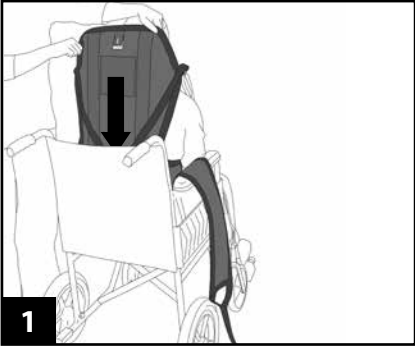
# Birthing Sling | BIRTH1

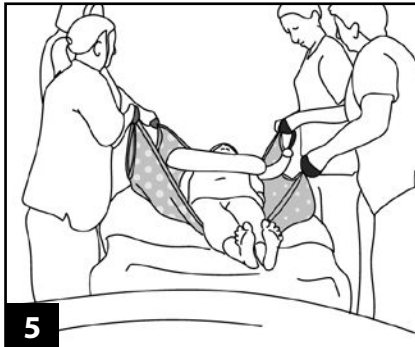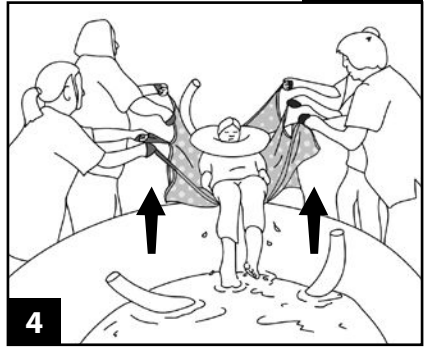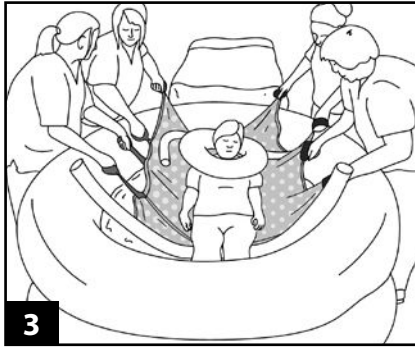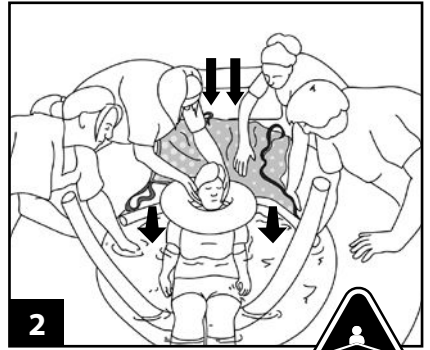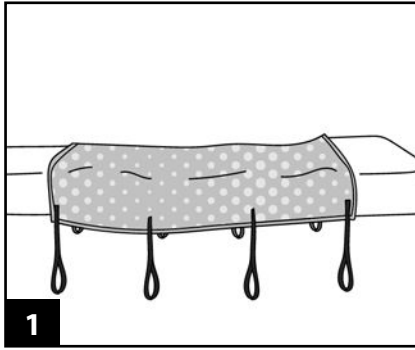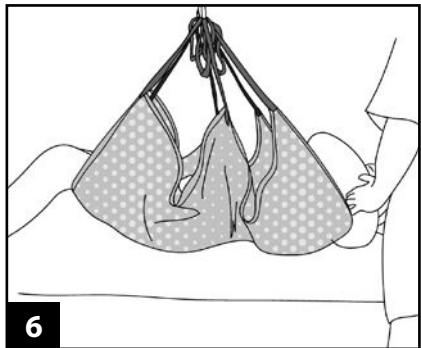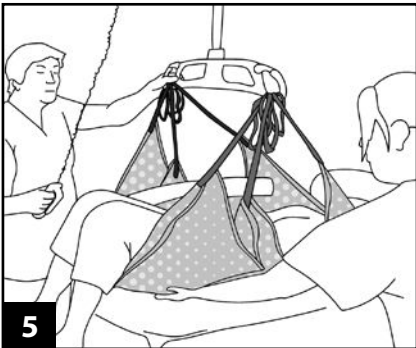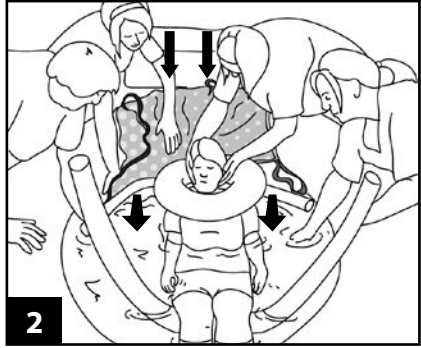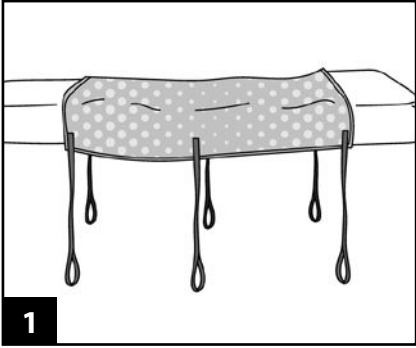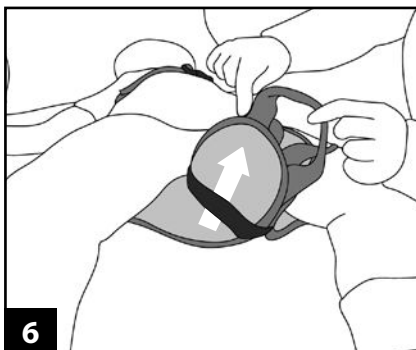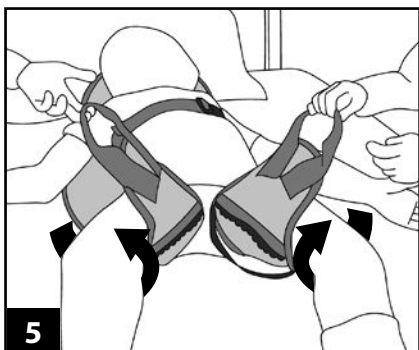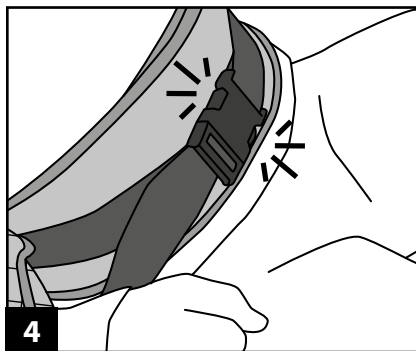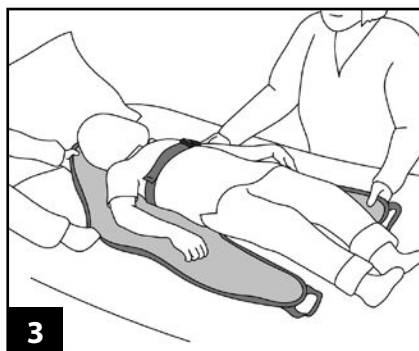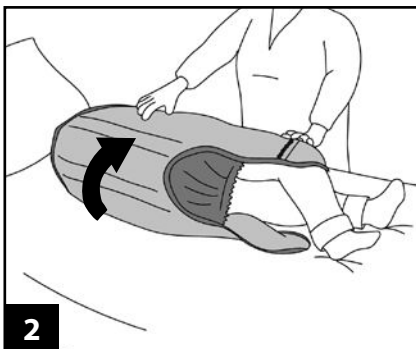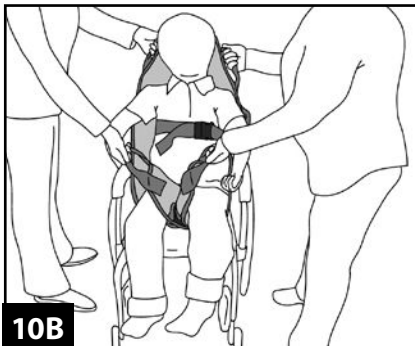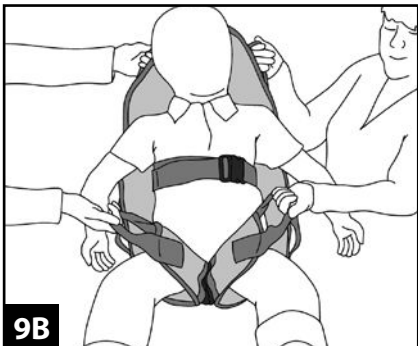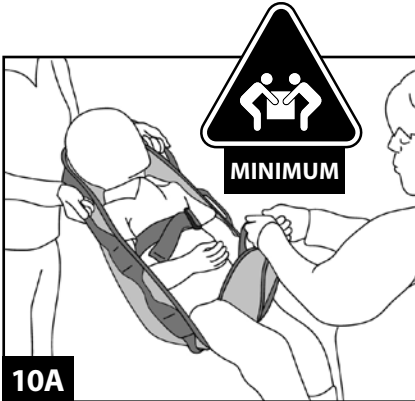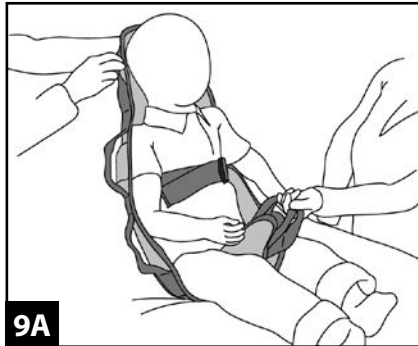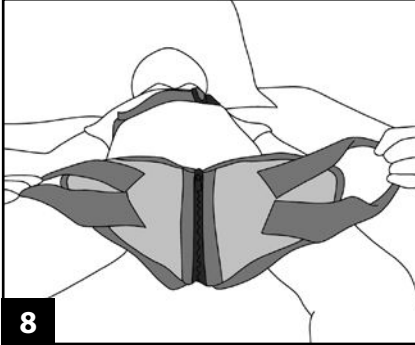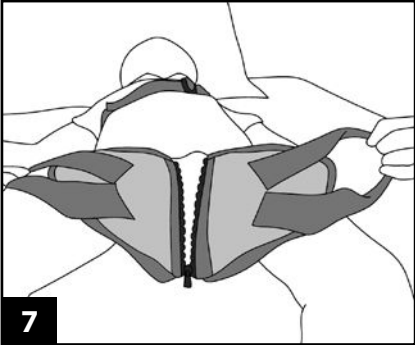For sizing guides and further information of this product please visit  
[www.silvalea.com](http://www.silvalea.com)

Or for further help and advice call  
**+44 (0)1626 331 655**

## Product Care Instructions

### Important:

This sling is designed for a general purpose transfer from a recumbent or seated position. If it is an In-Situ sling, the sling can remain in position under the patient subject to a risk assessment. Transfer accessories are intended to assist patient transfer.

The patient must be assessed by a competent person who is fully trained in the suitability, application and fitting of the sling. Carers should always be trained in the use of hoists and slings.

Correct sizing of a sling is important, if in any doubt consult your Moving and Handling Advisor.

### Service Label & Sling Checks:

**All products should be checked each time they are used for any signs of deterioration.**

#### Please Check:

- braiding/tapes for deterioration and fraying
- material for holes/snagging
- compatibility with the hoist
- safe working load of both the hoist and sling /product (if applicable)
- label is legible

**Remove from use immediately if there are any signs of deterioration to the sling or tapes or the label is illegible. The Service Label** is a visual indicator of when the 6-monthly or annual hoist inspection has been completed. This is in line with LOLER (Lifting Operations and Lifting Equipment Regulations).

### Warranty:

Silvalea offers a 12 month manufacturers warranty on all products unless otherwise stated. Scan QR Code or visit our website downloads page for more information.

**If any serious accident has occurred in relation to any of the devices provided by Silvalea, please report immediately using the contact details provided and to the Competent Authority of the Member State in which the user/patient is established. (incidentteam@silvalea.com)**

### Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- Metal fixings are sometimes used in our products. Care should be taken to ensure that all products containing metal attachments are washed in a pillowcase or wash bag to prevent damage to the product or to washing equipment. Silvalea accept no liability for damage to laundry equipment.
- DO NOT IRON
- Always remove the head support bones prior to washing to prevent damage to the sling & bones
- Silva-Superfine with COOLMAX® mesh slings are provided with a wash bag. The wash bag MUST be used during laundering.

### Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the user.

### Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.

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Revision: 11/01/2022

Silverhills Buildings, Decoy Ind. Estate,  
Newton Abbot, Devon, TQ12 5LZ, UK

ADVENA Ltd. Tower Business Centre, 2nd  
Floor, Tower Street, Swatar, BKR 4013. Malta

**Silvalea Ltd** a division of  **savaria**. Copyright 2022 © All rights reserved

Copyright & Design Rights: Silvalea. Silva-Quick-2-Applied® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Superfine® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Safety Slot® is registered trademark of D-ansermed Ltd. All rights reserved. COOLMAX® is a registered trademark of the LYCRA Company.

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**Silvalea Ltd** silvalea.com