REF  
8040

**SWL: 200 kg/440 lbs**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>4</b>
<b>Norsk</b> .....	<b>5</b>
<b>Dansk</b> .....	<b>6</b>
<b>Suomi</b> .....	<b>7</b>
<b>Deutsch</b> .....	<b>8</b>
<b>Nederlands</b> .....	<b>9</b>
<b>Français</b> .....	<b>10</b>
<b>Italiano</b> .....	<b>11</b>
<b>Español</b> .....	<b>12</b>
<b>Product information</b> .....	<b>13</b>

AntiSlip provides friction under the user's feet during different transfer situations, for example, when repositioning the user higher up in bed or when raising from a seated to a standing position. AntiSlip is made of a material with closed cells, which makes it hygienic.


## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly.

### Friction test

High friction - Ensure that the surface is rough and prevents the transfer aid from moving. Compare with a new product.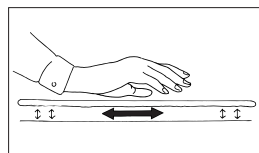


## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Placement of AntiSlip

AntiSlip is placed under the user's feet or in the bed to provide more friction, thus enabling a better grip.

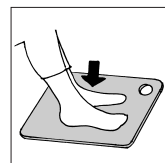
## Using AntiSlip

Provides friction under the feet in bed during pelvic lifting and for repositioning higher up in bed. Can be combined to advantage with a friction-reducing device under the shoulders.

AntiSlip can be used under the feet to provide a good grip during raising from a seated to a standing position or for transfers between different locations such as bed to wheelchair/chair. Use to advantage with EasyBelt/FlexiBelt.

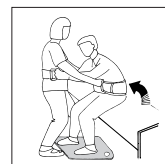## Material

Polyethylen

## Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

**Note:** Do not subject the product to hard pressure or impact. This may leave indentations in the material.

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

### ***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)