EasyGlide

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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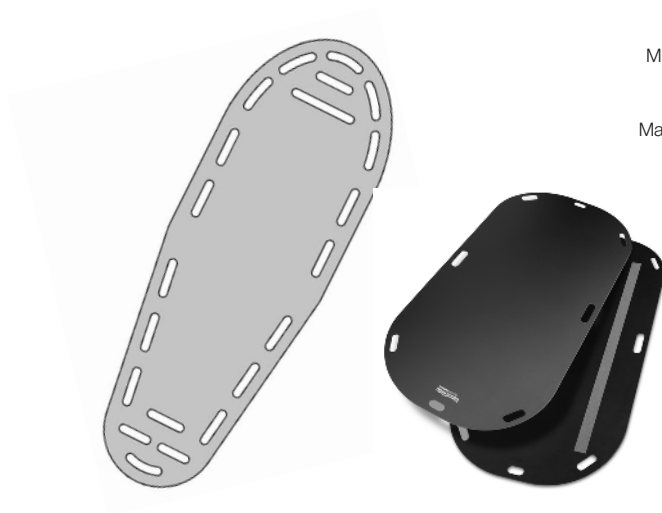
REF

5010

5000

SWL: 150 - 250 kg/330 - 550 lbs

Table of contents

English	4-5
Svenska	6-7
Norsk	8-9
Danske	10-11
Soumi	12-13
Deutsch	14-15
Nederlands	16-17
Français	18-19
Italiano	20-21
Español	22-23

EasyGlide, extra long is a sliding board which is used for supine transfers of users from one bed or stretcher to another. EasyGlide, extra long is used when the gap is max. 15 cm/5,9". For safety reasons, EasyGlide, extra long should have a firm underlying surface on both sides when transferring.

The rounded corners and the smooth, low-friction top surface provide for easy sliding transfers and make the sliding board very easy to position and remove. The double anti-slip tape on the underside prevents the board from sliding against the underlying surface during transfer.

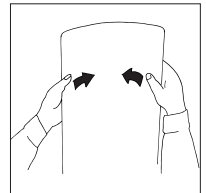
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that material is not worn or faded. If there are signs of wear, the product must be discarded.

Stability test

Carefully bend the assistive device to check strength and durability. The board should be both elastic and rigid. Discard the EasyGlide when its strength and durability are diminished.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

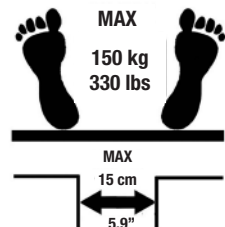
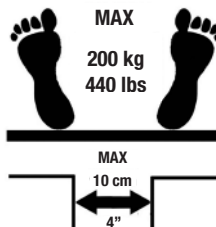
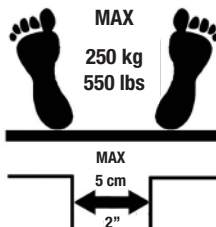
Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.


Important information

For the sake of safety, both underlying surfaces should be firm when transferring between bed and gurney.

Do not leave the user unattended during a transfer situation.

Transfer from bed to bed

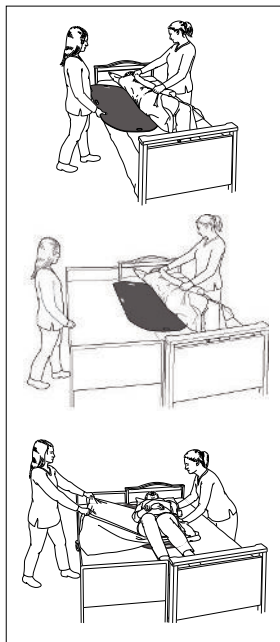EasyGlide is placed under the hips and shoulders.

The user is turned on one side. A draw sheet can be used to facilitate the transfer. Guide the EasyGlide in under the user, between the bed sheet and the draw sheet. Press EasyGlide down into the mattress somewhat and push in.

Ensure that the weight of the hips and shoulders is centred on EasyGlide.

Transfer on transfer slide. Place the transfer slide between the two underlying surfaces. By using a draw sheet, the caregiver's reach is extended, allowing a better working stance.

Make sure the transfer slide is between the two underlying surfaces and that the gap between the beds does not exceed 15 cm/5,9". Lean the user towards the transfer slide and pull the draw sheet to transfer the user.

To remove, lean the user to one side and pull EasyGlide out.

Material

Polyethen

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

CE Medical Device Class I

The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com