REF  
6080

**SWL: 150 kg/ 330 lbs**

## Table of contents

<b>English</b> .....	4-5
<b>Svenska</b> .....	6-7
<b>Norsk</b> .....	8-9
<b>Dansk</b> .....	10-11
<b>Suomi</b> .....	12-13
<b>Deutsch</b> .....	14-15
<b>Nederlands</b> .....	16-17
<b>Français</b> .....	18-19
<b>Italiano</b> .....	20-21
<b>Español</b> .....	22-23

SitWalk, 140 x 12cm/55 x 5", is used for repositioning farther back in the chair/wheelchair when the user has slid forward. It is equipped with several handles to ensure a good working stance for the caregiver.

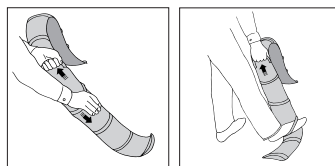
## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

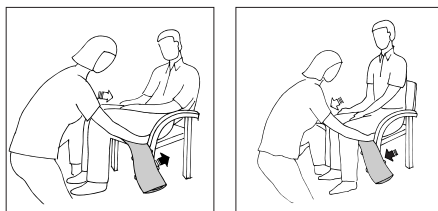
The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Placement and removal of SitWalk

Place SitWalk under the thighs and close to the seat, with the handles facing out from the body. Pull on the handles alternately from the right and left side, so that the strap slides against the thighs and inward towards the buttocks.

Raising the user's feet from the floor will relieve some of the weight of the thighs from the seat, thereby making repositioning easier.

Remove SitWalk by pulling alternately on the handles, so that it slides farther out on the thighs until it can be fully removed without moving the user.


## Using SitWalk

Caregivers can work in pairs or alone. The user should sit up straight and/or lean forward somewhat.

Lean the user to one side and pull by shifting weight. Work with straight arms. Then, lean the user in the opposite direction, continuing until the user is sitting fully against the back rest.

If you are working alone, be sure to move from side to side, so that you always work with straight arms and shift your weight.

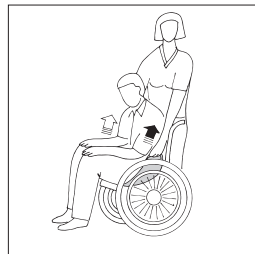A helpful hint is to place a FootStool under the feet, so that the thighs are lifted somewhat, which reduces friction.

### Material

Polyester

### Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Vida útil estimada

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)