


# QUICK START GUIDE - P-440/P-600 Ceiling Lift

## 1 Charge the Lift


Before initial use, the lift unit must be charged for 5 hours.

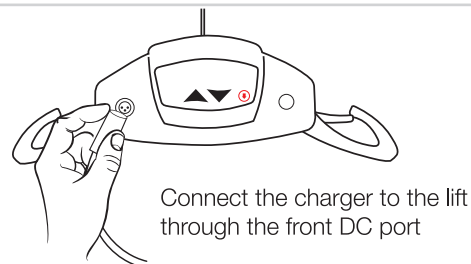


Connect the charger to the lift through the front DC port

## 2 Get Lift Ready


Make sure the Emergency On/Off switch on the back of the lift is positioned to the "ON" position.

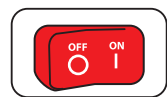

The pivoting hooks on the lift should also be in the DOWN position when preparing to lift.

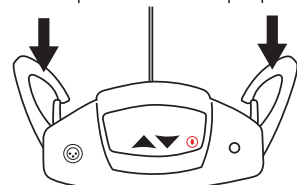


Connect the hand control to the right port for operation.

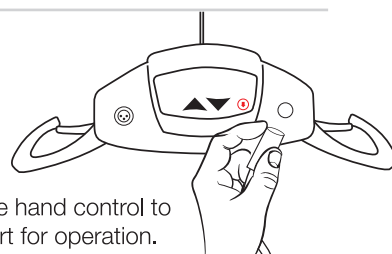

## 3 Turn On the Lift

Press any button on the hand control to turn the lift on.

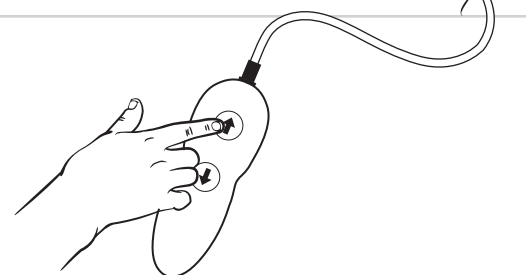

## 4 Turn Off the Lift

The lift automatically shuts off after approximately 2 minutes of non-use.




**i** In an emergency, the lift can also be shut off by positioning the Emergency On/Off switch to the "OFF" position.

## 5 Hand Control Operation


The P-440/P-600 lift hand control:

- Controls lift motion

Alternatively, you can also push the "UP" and "DOWN" buttons on the front membrane of the P-440/P-600 to lift and lower the patient.

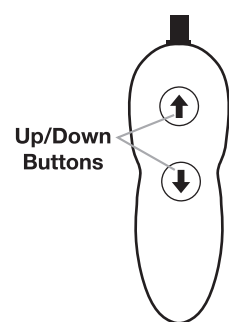


## 6 Transfer an Individual

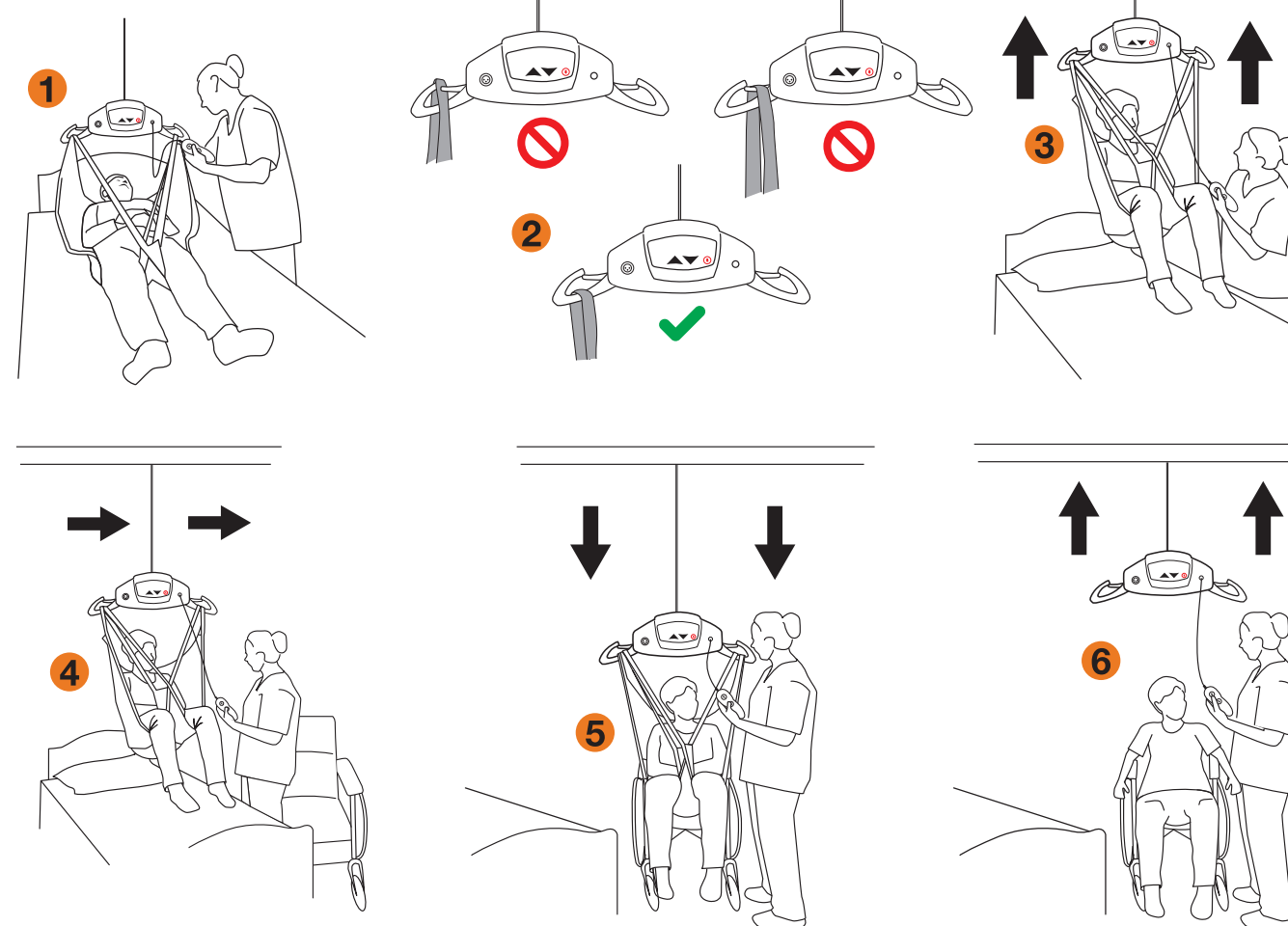

- ⚠ Never drag the lift along the track by pulling with the hand control.
- ⚠ Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.

The hand control can also be attached to the lift when not in use.

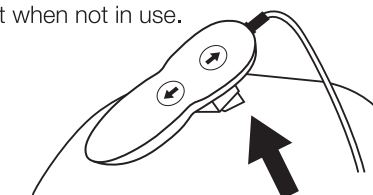

## 7 Emergency Stop



The lift unit has an Emergency Shut-off feature so that in an emergency the operator can completely cut power to the lift.

### P-440/P-600 Ceiling Lift

The P-440/P-600 is a lifting aid used by health care professionals and caregivers in homes, institutions, and long term care facilities to lift, position and transfer clients or a disabled family member. Functions such as lifting up or down are controlled by pressing buttons on the pneumatic hand control attached to the lift.

⚠ Before using the P-440/P-600 lift system, check the lift, track, and sling for any unusual wear or damage. Refer to the owner's manual for each piece of additional equipment to determine what should be checked. If anything looks unusual, contact your local Handicare dealer before use. Failure to check the lift could result in damage to the lift or serious injury to the operator or the individual being lifted.

⚠ Failure to comply with warnings in this Quick Start Guide may result in injury to the operator or the individual being lifted or transferred, as well as damage to the lift and related components. If you have any questions after reviewing this guide, contact your local authorized Handicare dealer.

⚠ Prior to attempting these steps, review the instructions for all parts and accessories; Contact your local authorized Handicare dealer if you have any questions or concerns.

HANDICARE USA, INC. 10888 Metro Court St. Louis, MO 63043 USA Tel: 866.891.6502 www.handicareusa.com	HANDICARE CANADA 485 Millway Avenue, Unit 2, Concord, ON L4K 3V4 Tel: 877.304.5438 www.handicare.ca
--	---

Handicare USA is quality certified in accordance with ISO 13485